

Recommended fluid amounts for reservoirs and tanks

| Reservoir | CAE Apollo Nursing | CAE Apollo Prehospital | CAE Ares | CAE Aria | CAE Juno | CAE Lucina | CAE Luna |
|--------------------------|--------------------|------------------------|---------------------|---|--------------------------|------------|----------|
| Airway | 40 ml | | | | 30 ml | | |
| Gastric | 60 ml | | | | 150 ml | | |
| Urinary | | | | >4 ml for priming | 60 ml fluid 80 ml air | 70 ml | 10 ml |
| Ostomy | | | | | 200 ml | | |
| Trauma/Blood Tank | 1.5 L | 1.5 L | | | | | |
| IV Prime | 60 ml | 60 ml | Primed using IV bag | 20 ml antecubital 20 ml jugular & hand | | 50 ml | 20 ml |
| Hemorrhage Tank | | | | | | 2 L | |
| Umbilical | | | | | | | 15 ml |
| Glucose Finger | | | | 0.2 ml | | | |