

IMPORTANT NOTICE ABOUT MACOS UPDATES AND MÜSE

Updating the Mac operating system (macOS) of your **Mac SCE Development Workstation** or **Mac Instructor Workstation** has an adverse effect on the **Müse** software. This affects OS X 10.8 (Mountain Lion) through macOS Mojave (10.14).

To address this, *after* the operating system has been updated, run the **Müse Patch Utility**.

NOTE: Ensure Müse is closed before proceeding.

How to Run the Müse Patch Utility

To run the Müse Patch Utility:

1. Download the Müse Patch Utility from the CAE Healthcare website:
https://caehealthcare.com/media/software_updates/MPU.zip
2. Double-click to expand the file, then double-click “Müse Patch Utility” application.
3. Click OK to continue, then enter an Administrator username and password when prompted.
4. Wait for the utility to complete.
5. Restart the computer.