

Blast Trousers



Uncontrolled hemorrhaging is the number one concern for patient survivability at the point of injury. Many of these fatalities are being prevented by the prompt application of a tourniquet. However, there are many times when a tourniquet cannot be applied or may be ineffective in controlling a life threatening hemorrhage. One of the most difficult wounds to treat on the battlefield is an uncontrolled hemorrhage from an inguinal injury. The Blast Trousers will fill this training gap.

Blast Trousers

Uncontrolled hemorrhaging is the number one concern for patient survivability at the point of injury. Many of these fatalities are being prevented by the prompt application of a tourniquet. However, there are many times when a tourniquet cannot be applied or may be ineffective in controlling a life threatening hemorrhage. One of the most difficult wounds to treat on the battlefield is an uncontrolled hemorrhage from an inguinal injury. The Blast Trousers will fill this training gap.

The Blast Trousers have been developed to replicate a life threatening bleed to the inguinal region. This is the most realistic simulation system to train Medics, EMTs, Corpsmen, and First Responders for treatment of this injury.

The Blast Trousers are used in conjunction with the Blood Pumping System (BPS) in Hyper-Realistic® scenarios. The Blast Trousers simulate blast injuries starting above the knees that are circumferential and continue to just below the naval. The wound is above any effective tourniquet placement and must be treated by clamping the vessel or packing the wound (which holds up to two rolls of gauze) and applying direct pressure.

The Blast Trousers have a life-like appearance and feel. Utilizing the BPS, these trousers are capable of bleeding out three liters of blood in just over two minutes.

The Blast Trousers are accessories that come with both versions of the Cut Suit®, the BPS Deluxe, and can be purchased separately or added to the BPS Basic.

Your worldwide
training partner
of choice



Blast Trousers

Clamping



Packing



Pressure

