

CAE**Maestro Evolve** User Guide



Proprietary Notice

This document, including the information contained herein, is confidential and/or proprietary to CAE Inc., and shall not be reproduced or disclosed in whole or in part, or used for any purpose whatsoever without the prior written authorization of CAE Inc.

Trademark Information

Trademarks and/or registered trademarks of CAE Healthcare Inc. and/or its affiliates include but are not limited to: Apollo, Ares, AresAR, Aria, Athena, BabySIM, BluePhantom, Caesar, CathLabVR, EndoVR, HPS, ICCU, InfantSIM, iStan, Juno, LapVR, LearningSpace, Lucina, LucinaAR, Luna, Maestro, METIman, Müse, NeuroVR, PediaSIM, SimEquip, Simulex, StethoSym, SymDefib, SymEyes, UniSIM, Vimedix, VimedixAR and Vivo. All other brands and product names are trademarks or registered trademarks of their respective owners. All logos, tradenames and trademarks referred to and used herein remain the property of their respective owners and may not be used, changed, copied, altered, or quoted without the written consent of the respective owner. All rights reserved.





CONTENTS

System Requirements	1
Software	1
Web Browser	1
Operating System	1
CAE Maestro Evolve Software	1
Hardware	2
Introducing Maestro Evolve	3
Login to Maestro Evolve	3
Step 1: Sign into CAE Digital Learning Solutions	3
Step 2: Obtain and Enter Verification Code	5
Step 3: Launch Maestro Evolve	8
Using CAE Maestro Evolve 1	1
CAE Maestro Application Overview	1
Simulation Mode Comparison1	1
Run on the Fly1	1
Simulated Clinical Experience (SCE)	2
Starting a Simulation	2
Run on the Fly Simulations	3
Run an SCE1	4
Run Screen Features	6
Records1	6
Checklists	8
Monitor Signals	9
Event Log	21
Reset an SCE	2
Changing Patient Status	23
Patient Status Parameters2	23
Changing a Patient's Parameters	25
Applying Conditions and Interventions2	27
Conditions	28
Medications	29
Treatments	3
Transitioning Scenario States	5
SCE Timeline Controls	6

CAEMaestro Evolve

	Using Markers.	.37
Pat	tient Display	39
	Patient Status Display	.39
	Patient Controls	.40
	Change the Control Widget Layout	.45
	Alarm State	.47
	Markers	.47
	Editing Parameters	.47
	Onset, Read-Only, and Override Indicators	.49
Ма	inaging SCEs	53
	Review an SCE	.54
	View SCE Details	.54
	SCE Editor Navigation Tabs	.54
	SCE Editor Panels	. 55
	Create a Custom SCE	.55
	Copy an SCE	. 55
	Creating a New SCE	.57
	Import or Export an SCE	.58
Usi	ing the SCE Editor	61
Usi	ing the SCE Editor	
Usi	5	.62
Usi	Patient Profile	.62 .64
Usi	Patient Profile	.62 .64 .64
Usi	Patient Profile	.62 .64 .64 .66
Usi	Patient Profile	.62 .64 .64 .66 .70
Usi	Patient Profile . Setting a Patient's Baseline . Scenarios . Add, Modify or Delete Scenario States . Checklists .	.62 .64 .64 .66 .70 .73
Usi	Patient Profile	.62 .64 .64 .70 .70 .73 .74
Usi	Patient Profile	.62 .64 .66 .70 .73 .74 .75
	Patient Profile . Setting a Patient's Baseline . Scenarios . Add, Modify or Delete Scenario States . Checklists . Patient Records . Upload Patient Records . Edit or Delete Patient Records .	.62 .64 .66 .70 .73 .74 .75 .76
	Patient Profile . Setting a Patient's Baseline . Scenarios . Add, Modify or Delete Scenario States . Checklists . Patient Records . Upload Patient Records . Edit or Delete Patient Records . Runtime Configuration .	.62 .64 .66 .70 .73 .74 .75 .76 79
	Patient Profile	.62 .64 .66 .70 .73 .75 .76 79 .79
	Patient Profile . Setting a Patient's Baseline . Scenarios . Add, Modify or Delete Scenario States . Checklists . Patient Records . Upload Patient Records . Edit or Delete Patient Records . Runtime Configuration . CAE Patient Monitor.	.62 .64 .66 .70 .73 .74 .75 .76 79 .80
	Patient Profile	.62 .64 .66 .70 .73 .73 .75 .76 79 .80 .83
	Patient Profile	.62 .64 .66 .70 .73 .74 .75 .76 79 .80 .83 .84
	Patient Profile . Setting a Patient's Baseline . Scenarios . Add, Modify or Delete Scenario States . Checklists . Patient Records . Upload Patient Records . Edit or Delete Patient Records . Runtime Configuration . tual Equipment. CAE Patient Monitor . Launch Patient Monitor . Select a Preconfigured Layout . Settings .	.62 .64 .66 .70 .73 .74 .75 .75 .79 .80 .83 .84 .89



	NIBP Cycling	93
	CAE Defibrillator and CAE AED	94
	Step 1: Set up the Patient for Electrotherapy	94
	Step 2: Set up the Defibrillator	98
	Step 3: Launch the Defibrillator	102
	Step 4: Set up the AED	104
	Step 5: Launch the AED	108
	CAE Ventilator and CAE Transport Ventilator	109
	Step 1: Set up the Patient	110
	Step 2: Set up the Ventilator	114
	Step 3: Launch the Ventilator	117
	Step 4: Set up the Transport Ventilator	121
	Step 5: Launch the Transport Ventilator	125
	CAE Anesthesia Machine	129
	Step 1: Set up the Anesthesia Machine	130
	Step 2: Launch the Anesthesia Machine	133
CA	EEmbody the Virtual Simulator	145
	Using Embody	145
	Select the Simulator	145
	Launch the Virtual Simulator	147
	Selecting the Patient	150
	Embody Features	154
	Assessments	154
	Monitoring	156
	Equipment	156
	Treatments	158
	Change Patient View	160
Le	arning Modules	161
	Import a Learning Module	
	Delete a Learning Module	162
	Create a Learning Module	163
Ad	lministrative Tools	165
	System Administration	166
	Select a Simulator	168
	Set the Default System Language	169
	Additional Administrative Tools	170

CAEMaestro Evolve

	SCE Management	.170
	Session History	.170
Sh	aring Maestro Evolve with Learners	173
	Planned Training Event	.174
	Invite Learners	.174
	Conduct Training Event	.177
	Share Virtual Equipment	.180
Ар	ppendix A - FAQ	183
	Maestro Evolve	.183
	Ventilator	.185
	Embody	.186
	Hardware Questions	.187
Ар	pendix B- Defibrillator Controls and Display Options	189
	Facilitator Controls	.189
	Learner Controls	.191
	Learner Display	.193
Ар	pendix C - AED Controls and Display Options	195
	Facilitator Controls	.195
	Learner Controls	.195
	Learner Display	.196
Ар	pendix D - Ventilator Controls and Display Options	197
	Facilitator Controls	.197
	Learner Controls	.200
	Learner Parameters Displayed	.202
	Learner Display Options	.204
	Associated Maestro Preferences	.204
Ар	pendix E - Transport Ventilator Controls and Display Options	205
	Facilitator Controls	.205
	Learner Controls	.207
	Learner Parameters Displayed	.209
	Learner Display Options	.210
	Associated Maestro Preferences	.210
Ар	pendix F - Anesthesia Machine Controls and Display	211
	Facilitator Controls	.211
	Learner Controls	.214
	Learner Parameters Displayed	.217



Appendix H - Patient Parameters	225
Appendix G - Medications	221
Associated Maestro Preferences	
Learner Display Options	





System Requirements

The following tables list the software and hardware requirements for CAE Maestro™ Evolve.

Software

Web Browser

Minimum		
Web Browser	 Google Chrome is required. Google Chrome may be downloaded free at: www.google.com/chrome/browser/desktop/index.html 	
	Note: Other web browsers which support the WebGL platform utilized by Maestro may work but are not supported at this time.	

Operating System

Operating System • Windows 10	
	• MacOS 11
	 The operating system must be current and still supported by its manufacturer.
	 Linus, iOS and Android are not supported.

Mac is a registered trademark of Apple Inc. Windows and Internet Explorer are registered trademarks of the Microsoft Corporation in the United States and/or other countries. Chrome is a registered trademark of Google Inc.

CAE Maestro Evolve Software

The following identifies the software version associated with the release of this guide.

Document	Document Version	Software Version
CAE Maestro Evolve User Guide	905K002252 v1.4	CAE Maestro Evolve v1.5

Hardware

Minimum	Optimal
• Two Intel Dual Core CPUs, 2.4 GHz	• Six Intel Dual Core CPUs, 2.4 GHz
1280x680 screen resolution	1920x1080 screen resolution
• 8 GB DDR3 RAM	• 16 GB DDR3 RAM

Required	Recommended or Not Supported
• Mouse	Two monitors are recommended.
 Keyboard - when editing SCEs (Simulated Clinical Experiences). 	• Touch screens are not supported.
 Graphics card capable of rendering WebGL applications. To determine if your graphics card is capable of rendering WebGL applications, go to: 	
http://get.webgl.org/	
• Internet connection at or above 15Mbps.	



INTRODUCING MAESTRO EVOLVE

Maestro Evolve is designed to highlight the power of Maestro for distance learning and to work with CAE Healthcare's new Virtual Equipment and Virtual Patient products. Maestro Evolve is launched directly from the cloud, providing the convenience of accessing the simulation applications from wherever the instructor or learner is located.

Login to Maestro Evolve

Follow the steps to login to Maestro Evolve.

Login Steps		
1	Sign into CAE Digital Learning Solutions	
2	Obtain and Enter Verification Code	
3	Launch Maestro Evolve	

Step 1: Sign into CAE Digital Learning Solutions

To sign into CAE Digital Learning Solutions, perform the following steps:

- A one-time registration process is necessary when access to Maestro Evolve is requested from the CAE Sales Representative. In the event you do not have a Sales Representative, contact CAE Healthcare Customer Service at: <u>https://caehealthcare.com/contact-us/</u>
- 2. Open the email received from CAE and click the link:

https://cloud.caehealthcare.com

A browser window displaying the Welcome screen appears. Enter the email address where the access link was sent to begin a one-time registration process.

CAE		
	Welcome! Sign in with your CAE account This app is restricted to invited users. Email Istan.dardman@cae.com	

Welcome Screen

3. Click the **Next** button.

The New Account Upgrade message appears.

CAE		elp 🕜
	New account upgrade Your account has been upgraded to the the new CAS user experience. You will need to reset your password in order to login to your new account. BACK NEXT	

New Account Upgrade Prompt

Note: New account registration is required for all users, including existing users, with the newest update to Maestro Evolve. After the new account registration is completed (in the following steps), the user will only see the Sign In window upon future login attempts.

4. Click **Next**. and leave the browser window open. A verification code is sent to the registered email address.






Step 2: Obtain and Enter Verification Code


Upon completion of the previous steps, a verification code is sent to the email address that was entered at the Sign In window.

1. Open the email from *noreply@caehealthcare.com*.

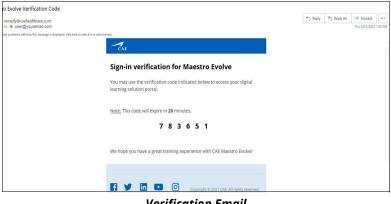

Verification Email

2. Return to the browser and enter the verification code in the Verification Code field.

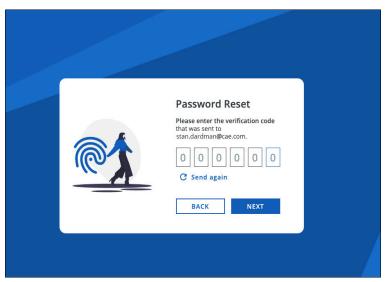

Password Reset Screen


3. Click the **Next** button.

The New Account screen appears.

CAE	Help 🕥
CAL	New Account
	Please validate your details to create a CAE account.
	CAEPortal B2C
	First Name
	Stan
	Last Name
	Dardman
	Email
	stan.dardman@cae.com
	Password
	······· ©
K	A Laster fi, Alexandra en velha 1 ar more uppercision. tettar, isoverspace fetter, number & symbol
Create a new	agree to receive email communications and a far every product updates from CAE.
CAE account	то не развис сравно сравно так САL
With your CAE account, you will be able to access a multitude of tools that will enhance your experience with CAE.	thy clicking on Next, you agree to the T <u>erms of Asias</u> Princip/Indice and <u>Contect Policy</u> . NEXT

New Account Screen


4. Click **Next**.

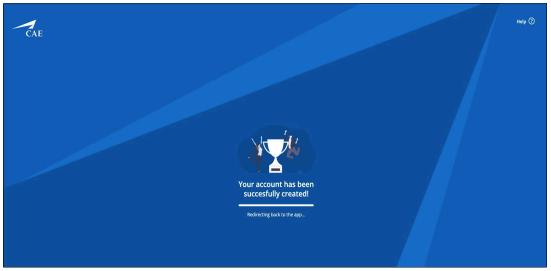

Account Successfully Created Screen



The CAE Digital Learning Solutions window appears with the Maestro Evolve module available.

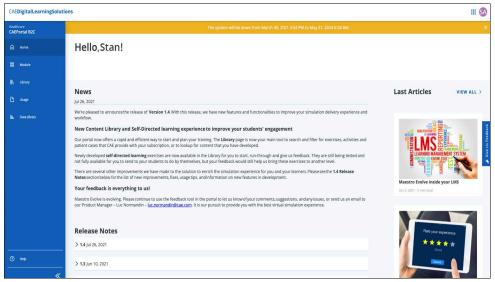

CAE Digital Learning Solutions Screen

On the Digital Learning Solutions screen, users find information regarding planned Maestro Evolve enhancements and new products, as well as information specific to the current version of Maestro Evolve.

Additional options on the CAE Digital Learning Solutions page are:

- Module Displays available modules for the registered user
- Library Displays the scenarios available for the registered user
- Usage Maestro Evolve License information
- Help Link to the CAE Healthcare website with support options

Step 3: Launch Maestro Evolve

To launch the Maestro Evolve application:

1. From the CAE Digital Learning Solutions window, on the CAEMaestro Evolve Module Menu, click **Start**.



Maestro Evolve Module Menu - Start and Invite

For more information on the "**Invite**" option, see the chapter in this guide *Sharing Maestro Evolve with Learners.*

The Start Maestro Evolve Session window appears.

Ad hoc	session				
Start pr	eviously planned	session			
				CANCEL	STAR

Start Maestro Evolve Session

2. Verify Ad hoc session is selected and click START.

For more information on the "**Start Previously Planned Session**" option, see the chapter in this guide *Sharing Maestro Evolve with Learners.*



Note: In the event Maestro Evolve has previously been started, the following Open, Share, and Stop options will appear on the Maestro Evolve Module Menu.



Maestro Evolve Module Menu - Open, Share, and Stop

- **Open** Opens Maestro Evolve, bypassing the Start Maestro Evolve Session window.
- Share -
 - Allows the Instructor to share links to virtual equipment with the learner (see the chapter in this guide *Sharing Evolve with Learners*)
 - ° Allows access to Embody, CAE's virtual simulator
- **Stop** Stops Maestro Evolve, returning to the Maestro Evolve Module Menu with Start and Invite options.

The Select Simulator window appears.

Note: For returning users, selecting a simulator may be bypassed if a simulator was previously chosen and set as the default. In this case, the Maestro Home screen appears.

\$
Ę
ights reserved.

Select Simulator Window

3. Click the drop-down list and select a simulator.



Select Simulator Drop-Down List

4. Click the **Continue** button.

The Maestro Home Screen appears.

MS CAE	Maestro \vee						\$
Home	SCE Manager	History					Ares (Standalone)
			C.	MS AE Maestr	0		
			Run on the fly (Modeled)	بخ Run on the fly (Manual)	Run an SCE		
							© 7020 CAE. All rights reserved.

Maestro Home Screen

There are three options to choose from on the Home screen:

- ° Run on the fly (Modeled)
- ° Run on the fly (Manual)
- ° Run an SCE



USING CAE MAESTRO EVOLVE

Using the CAE Maestro Evolve software, you can create and run SCEs, assess learners, and adjust system controls.

Note: For optimal performance, no other software program should be open while Maestro is running.

CAE Maestro Application Overview

Users have the following choices for managing their simulation in the CAE Maestro software:

- Run simulations "On the fly" in Manual mode
- Run simulations "On the fly" in Modeled mode using CAE Physiology
- Run preconfigured simulations using Simulated Clinical Experiences (SCEs) in Manual or Modeled mode

Simulation Mode Comparison

On the Fly vs. SCE

There are two ways to run a simulation in Maestro: On the fly and Simulated Clinical Experience (SCE). Users can start a simulation or choose to design content for the simulation and save it as an SCE.

When operating On the fly, no content is authored ahead of time and the simulation is simply controlled in real-time by the instructor.

With an SCE, some details of the simulation are defined ahead of time (e.g. states, patient profile, and other scenario background information). During an SCE, users can also make changes On the Fly as needed.

Run on the Fly

Running simulations on the fly includes no predetermined scenarios or intended progression of events, and provides facilitators with a simple way to run a simulation with no advanced programming. You can run on the fly in Manual or Modeled mode.

In Manual mode, the user drives all changes to the patient's condition and responses to interventions by making manual selections in the software.

In Modeled mode, the simulation is automatically driven by CAE Physiology (a computational model of human physiology). This model controls patient status, progression of clinical signs, and response to interventions. Modeled mode also allows the instructor to override the model's output with manual inputs.

The following virtual equipment requires Modeled mode:


- Ventilator
- Transport Ventilator
- Anesthesia Machine

Simulated Clinical Experience (SCE)

Simulated Clinical Experiences (SCEs) are process tools that let facilitators execute a learning strategy using simulation. Preconfigured CAE SCEs provide an extensive overview and outline of the learning exercise, and require minimal additional faculty development time for use. Each SCE is typically comprised of a patient and a scenario.

Starting a Simulation

Upon launching the CAE Maestro app, the Home Screen appears and users have the option to choose one of the following simulation types: Run on the fly (Modeled), Run on the fly (Manual), or Run an SCE.

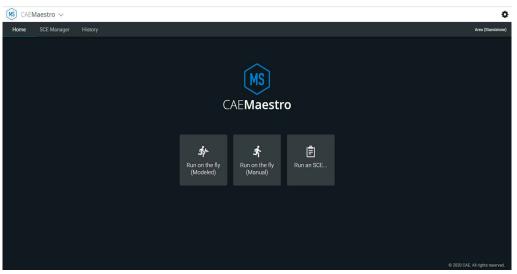

CAEMaestro Home Screen




Run on the Fly Simulations

The Run on the fly (Manual or Modeled) simulation starts a simulation without an SCE.

To begin a Run on the fly simulation:

- 1. From the Home Screen, click on the **Run on the fly** button.
- 2. Select the patient's gender.

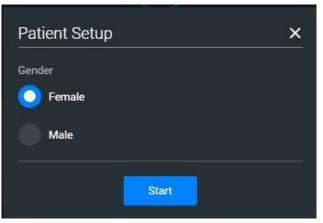

Start Screen - Select Gender

3. Click on **Start** to begin the simulation.

From the Run Screen, you can manually alter the patient's status and adjust parameters as needed for specific situations. The functionality in this mode is described in the *Run an SCE* section of this user guide.

Run an SCE

From the SCE Manager, various simulations are available.

To run an SCE:

1. From the Home Screen, click on **Run an SCE**.

The **SCE Manager** tab is selected by default.

2. Click on the **Preconfigured SCEs** tab to view the list of preloaded SCEs.

CAE Digital Learning Solutions X				- 0 ×
← → C	.canadacentral.cloudapp.azure.com/dab19420-508c-44b6-befe-ca08cb7	099ec/sce-manager		☆ <u> </u>
Home SCE Manager	History			Ares (Standalone)
Q Search	New SCE More 👻			Sort by SCE Title (A-Z)
Recently Used	SCE Title	Created by	Mode	
	Closed Head Injury and Pneumothorax	CAE	Manual	🌣 ▶ 🔤 Play
Favorites	A Heroin Overdose	CAE	Manual	ton button
Preconfigured SCEs	A Heroin Overdose	CAE	Modeled	× 🕨
User-created SCEs	Inferior-Posterior Myocardial Infarction	CAE	Manual	☆ ▶
CAE Learning Modules >	Inferior-Posterior Myocardial Infarction	CAE	Modeled	* 🕨
	Sepsis with Hypotension	CAE	Modeled	* 🕨
	Sepsis with Hypotension	CAE	Manual	*
	n a i	~		w 💌

SCE Manager Screen - Preconfigured SCEs

There are two ways to start an SCE:


° Click on the **Play** button,

OR

° Click on the SCE title to view SCE details before launching.




3. Click on the **Run SCE** button to start an SCE.

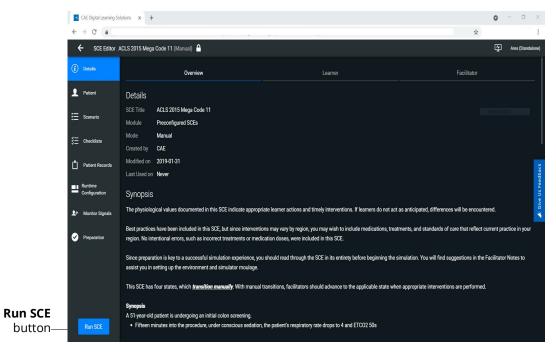

SCE Manager Details Screen - Run SCE Button


The Run Screen appears and displays the Patient Baseline state.

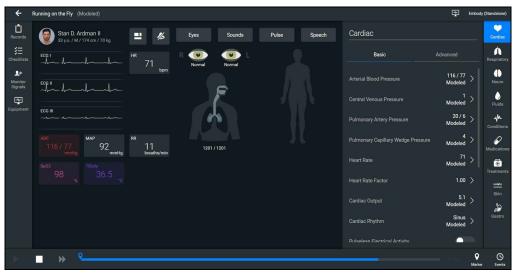

SCE Run Screen

Using CAE Maestro Evolve

4. Click on the **Scenario** icon in the upper-right corner of the Run Screen.

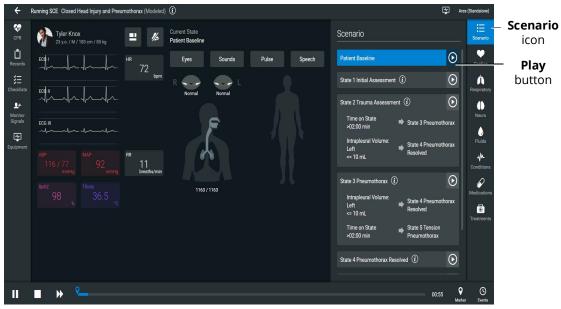

SCE Run Screen with Scenario States

The list of scenario states appears. A scenario will remain in the Patient Baseline until the **Play** button is selected in another state.

5. Click on the **Play** button to apply a state to the running SCE.

Run Screen Features

When an SCE is started or when running on the fly, the Run Screen appears and displays a series of panels that group related controls and information. The content for each control is dynamic and varies whether you run scenarios on the fly or as an SCE. Users can adjust the status of the patient while they proceed through a simulation using these controls. When running an SCE, some panels contain additional controls.

Records

You can display patient records while an SCE is running.

To display a patient record:

1. From the Maestro Run screen, select **Records**.


The Patient Files tab displays by default.

To view patient files, select **Show Library** button.

The list appears with all available patient records appears.



To view the Patient's history, select **History** tab.

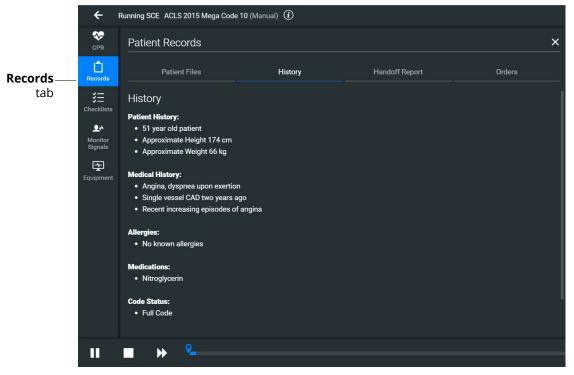

Patient Records - History

- 2. (Optional) To preview a patient record before sending to the Patient Monitor, select the record and click on the **Preview** button.
- 3. Select a patient record and click on **Send to Patient Monitor**.

The patient record opens in CAE Patient Monitor.

Note: Only one patient record displays at a time.

To stop displaying a patient record in the Patient Monitor, on the patient record click on **Remove from Patient Monitor**.

To close the Patient Record list, click on **Records**.

Checklists

To access learner checklists:

1. Click on the **Checklist** tab.

When a checklist is selected it is highlighted in blue and the content of the checklist is displayed in the right panel.

2. Click on **Begin** to launch the first checklist and start verifying steps performed by learner.

←	Running on the Fly (Modeled)	1
L Records	Checklists	X
ジ三 Checklists	Catheterization	Catheterization 01 - Begin
 ≁ Monitor Signals	Drug	
Equipment	General Assessment	
	IV Cannulation	
	Patient Management	
	Wound Management	
	Import from Library	

The Checklists Popup Screen

3. After a checklist has been completed, click on **New Instance** to begin another checklist for the next learner.

The Checklist drop-down list displays each instance completed, numbered 01 as first instance, 02 as second instance, etc. within each state.

4. Click on the **X** to close the Checklist panel.




Monitor Signals

The Monitor Signals tab gives the users control over the values and signals that appear on the simulated Patient Monitor.

Click on the **Monitor Signals** icon to define which patient indicators will be monitored.

Use the drop-down menus to define which patient parameters will be visible for learners viewing the patient monitor.

Use the toggles to turn the various values and signals on and off.

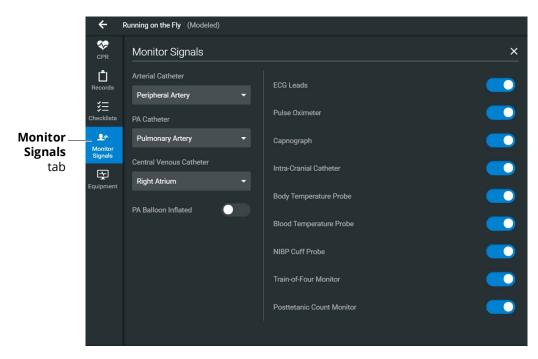

Monitor Signals

Using CAE Maestro Evolve

By default, all probes are on. When probes are turned off, it impacts different waveforms and numerical values on the patient monitor.

Probe Turned Off	Impact
ECG Leads	Hides the ECG waveform and HR value
Pulse Oximeter	Hides the PLETH waveform and SpO2 value
Capnograph	Hides the CO2 waveform and EtCO2 value
Intra-Cranial Catheter	Hides the Intra-Cranial Catheter values
Body Temperature	Hides the TBody values
Blood Temperature	Hides the TBlood values
NIBP Cuff Probe	Toggles off the Blood Pressure cuff on Embody. After launching Embody, the Blood Pressure Cuff may be toggled back on from Embody Monitoring options.

Setting the catheter placement to Atmosphere causes a flat line to appear, even when an override is used. Available catheters include:

- Arterial Catheter
- PA Catheter
- Central Venous Catheter

If the catheter placement is *None*, no trace appears at all.

Click on the **PA Balloon Inflated** toggle to inflate the patient's PA Balloon. Click on it again to deflate.




Event Log

During an SCE, all changes (manual or detected by the system) to simulation settings, learner actions, and checklist results are recorded in the Event Log.


To access the Event Log, click on the **Events** button in the bottom-right corner of the Run Screen.

The event log records the time and details of instructor and learner actions, which are color coded by category.

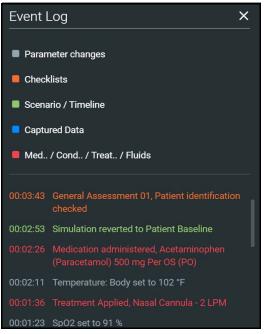

Event Log

In Manual SCEs, the administered medication information tracked in the Event Log does not have an automatic effect on patient physiology.

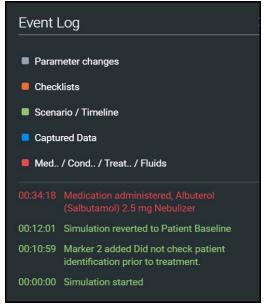

Medication Information in Event Log

Reset an SCE

Resetting an SCE brings the patient's initial status data back to its original status. The SCE time is unaffected. Reset data appears in the Event Log.


To reset patient data:

1. While running an SCE, select the first marker on the timeline.





2. Click on Revert.

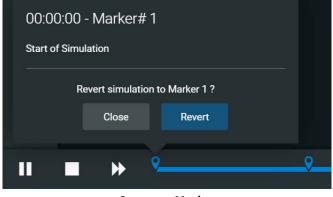

Revert to Marker

The patient returns to the Baseline state. The patient reset is indicated on the SCE timeline bar and in the Event Log.



Reset Notification

Changing Patient Status

You can adjust the patient status when running on the fly or running an SCE by:


- Using one of the parameter panels on the Run Screen to modify parameters. For more information, see *Patient Status Display* section of this user guide
- Applying a Condition, Treatment, or Medication
- Applying a Scenario state

Patient Status Parameters

From the Run Screen, select a parameter icon to display the associated parameters panel for various body systems and features. To access a parameters panel, click on the appropriate icon or button on the right side of the Run Screen for:

- Cardiovascular
- Respiratory
- Neurological
- Fluids
- Conditions

- Medications
- Treatments
- Skin (available with Embody)
- Gastro (available with Embody)

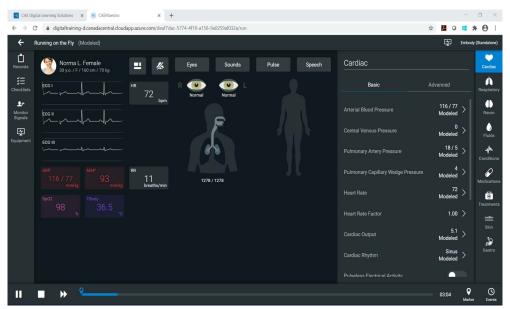

Maestro SCE Run Screen with Embody Cardiac Parameters

Types of Parameters

There are two types of parameters: numeric and discrete.

Once a parameter is selected and set, the patient's physiology changes according to the model for that parameter.

Numeric Parameters

To change numeric parameters, click in the relevant field and enter a new value in place of the existing one. Or, use the slider to move through the range of parameter values until the numeric value is established.

In Run on the fly Modeled mode, there are a larger set of parameters that are either monitored patient values, or physiological model inputs. When users set or modify monitored patient values, it overrides the output of the physiological model. To return to a physiologically modeled value, click on the **Modeled** button and click **Accept**.



Changing physiological model inputs indirectly impacts the output of the physiological model. When a modeled input is called a Factor, it acts as a multiplier. For example, Heart Rate Factor 2.0 is two times the baseline Heart Rate.

Discrete Parameters

Discrete parameters let users choose the appropriate option from a drop-down menu or toggle switch.

The following image shows the Bronchial Occlusion controls in the Respiratory parameter. The Bronchial Occlusion control is set using a discrete parameter switch that toggles between **OFF** and **ON**.




Bronchial Occlusion Parameter

Some parameters have two toggle switches or buttons, one for each side of the manikin.

Changing a Patient's Parameters

To change a patient's parameters:

1. Click on the appropriate parameters icon to view the parameter panel.

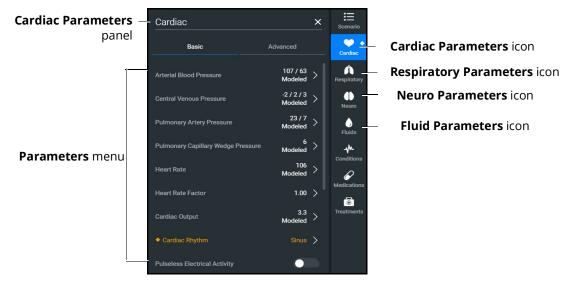

Cardiac Parameters Panel

If running on the fly or running an SCE in Modeled mode, the parameters will display the physiological responses in real-time.


Note: Patient status parameters can be toggled between modeled and override modes. Click on the arrow next to a parameter to open the control menu on the right side of the screen. Turn on the Override switch to adjust settings as desired, then click on **Accept**.

	HR		×
	Modeled		
Override Switch	O Override		
	HR (bpm)		+
	10	280	106
	Onset (mm:ss)		+
	00:00	10:00	0:00
		Cancel	Accept

Override Controls

When you override a setting, an orange diamond-shaped indicator will appear at the upper-right corner of that parameter icon to show it is no longer driven by the model. The same indicator will also appear on the parameter widget and associated waveforms.

2. Select the desired parameter. The Parameter Controls screen appears.

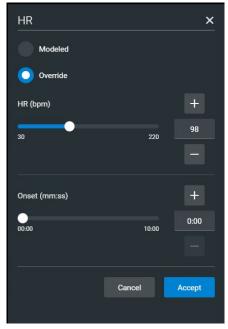

Heart Rate Parameter Controls



3. Set the new value and click **Accept**.


Applying Conditions and Interventions

Conditions or interventions, such as medications and treatments, may be applied during simulation. Once applied, conditions are reflected in the patient's physiology (if in Modeled mode) and logged. All medications and other interventions are also logged, and most also affect the patient's physiology (in Modeled mode only).

When programming an SCE, conditions, medications, and treatments likely to be used in a given simulation case can be set up as Quicklinks, which will appear in alphabetical order at the top of the panel for quick access.

Note: Quicklinks can only be added when creating or editing an SCE.

To learn more about setting up Quicklinks, see *Runtime Configuration* in *Using the SCE Editor* section of this user guide.

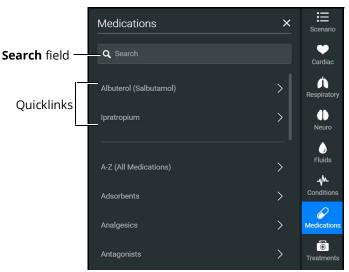


Medications Panel - Quicklinks

TIP: Click on the appropriate icon to collapse or expand the control.

Conditions

Conditions are preprogrammed pathophysiological states that incorporate one or more physiological parameters and are designed to enable multiple physiological changes more easily. Conditions are similar to scenario states in that they can contain multiple parameter changes that make it easy to quickly adjust the patient's status.

Conditions are especially useful when programming in Modeled mode, where they continuously interact with the modeled physiology and any interventions to ensure appropriate changes for the condition.

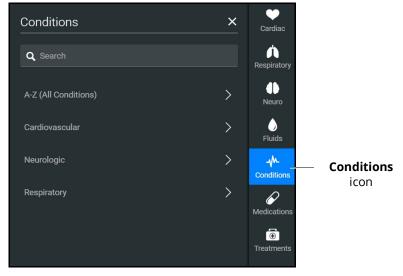

Conditions Panel

To apply a condition:

- 1. In the **Conditions** panel, click on a Quicklink, if available, or select a condition from within the listed categories or by clickingon **A-Z (All Conditions).** You can also use the Search feature to quickly find a specific condition.
- 2. Click on the name of the desired condition and select a severity, if available.
- 3. Click on **Apply condition**.





To choose a medication:

1. In the **Medications** panel, click on one of the Quicklinks, or from the Medications list, click on **A-Z (All Medications)** to browse all medications in alphabetical order, or use the Search feature to quickly find a specific medication. Medications are also grouped by category below the **A-Z (All Medications)** list option.

Medications		×
Medication List	Log	
Q Search		
A-Z (All Medications)		>
Adsorbents		>
Analgesics		>
Antagonists		>
Antibiotics		>
Antiemetics		>
Blockade Reversal		>
Cardiovascular		>
Medications	Panel	

A menu appears and shows the available medications.

2. Click on on a specific medication to choose a dose and route of administration.

The medication controls appear. The delivery method and amount options, which include pre-defined doses and custom dose options, are displayed.

Diazepam		×
Delivery Method		
Intramuscular (IM)	log only	
Intraosseous (IO)		
Intravenous (IV)		
Amount		
Custom 🔫		
		-
	Close	Administer

Medication Controls

Under **Delivery Method** options, click on the desired route of administration.

In the **Amount** drop-down menu, select a pre-defined dose or choose Custom to enter a custom dose.

Diazepam		×
Delivery Method		
Intramuscular (IM)	log only	
Intraosseous (IO)		
O Intravenous (IV)		
Amount		
1 mg 👻		
1 mg		
2 mg	se	Administer
4 mg		
10 mg		

Custom Dose

3. Click on Administer.

Note: All administered medications are added to the Event log.



In Manual mode, administered medications do not affect the patient's status. Any associated changes to the vital signs or other parameters must be manually adjusted. For more information, see *Changing Patient Status* section of this user guide.

In Modeled mode, medications affect the patient's status unless labeled *log only*. Medications are also logged in the **Log** tab on the **Medications** panel.

The Medications Log

The Medications Log, present in Modeled mode only, tracks the administered medications. If applicable, the infusion rate of the medication is also displayed.

To view the Medications Log:

- 1. From the Run Screen, click on the **Medications** control.
- 2. Click on the **Log** tab.

Medicat	tions		×	
Med	ication List	Log		- Log t
ACTIVE				
Midazolar		193.8571 mg/kg	>	
Succinylcl	noline	741.2658 mg/kg	>	
LOGGED				
00:02:26	Succinylcholine Medication Admin	istered, 1.1 mg/kg (IV)	C 2	
00:01:31	Midazolam Medication Admin	istered, 2.5 mg (IV)		
00:00:52	Midazolam Medication Admin	istered, 5 mg (IV)		

Medications Log

The medications that have previously been administered appear in the *Logged* section of the screen. Medications that are currently impacting the patient physiology appear in the *Active* section of the screen.

Click on an active medication to display medication information.



Medication Information

The normalized effector site concentration is shown next to each medication listing. This represents the amount of medication that is affecting the patient physiology.

Reset a Medication

To reset a medication:

- 1. From the Medications Log, select an active medication.
- 2. From the medication information, click on **Reset**.
- 3. When the Reset Effector Site Concentration prompt appears, click on **Reset**.

The medication is cleared from the Active list. The original administration information remains in the Medications Log, however, and the **Events** Log reflects both the administration and resetting of the medication.



Treatments


To apply a treatment:

1. Click on the **Treatments** icon and select a treatment from the Quicklinks, or select a category to see treatments organized by type, or click on **A-Z (All Treatments)**. You can also use the Search function. Navigate through the menus to locate the desired treatment.

	Treatments	×	Scenario	
Search field —	Q Search		Cardiac	
	A-Z (All Treatments)	>	Respiratory	
	Electrical Therapy	>	I Neuro	
	Invasive Procedures	>	Fluids	
	Oxygen	>	Conditions	
			Medications	
			Treatments	Treatments icon

Treatments Panel

Make further selections as needed, such as the Room Air or LPM of oxygen for the nasal cannula.

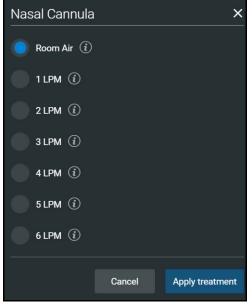

Treatments Menu

2. Select a treatment and click on **Apply treatment**.






Transitioning Scenario States

To move between scenario states from the Run Screen:

1. Click on the **Scenario** icon to expand the scenario panel.

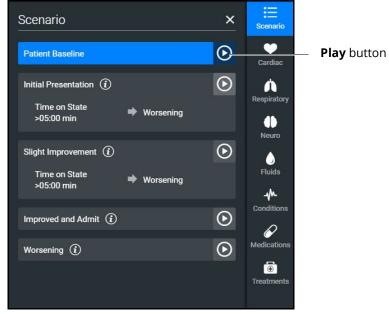

Scenario Management Panel

2. Click on the **Play** button in the desired state to advance from the current state to the desired state.

To pause or continue a scenario, click on the **Pause** or **Play** button from the Scenario Management menu.

SCE Timeline Controls

The SCE timeline controls are located at the bottom of the Run Screen.

	*	۹		06:59	Q Marker	() Events
			Timeline			

- The **Timeline** bar shows the amount of time that has elapsed and markers that have been created.
- The **Marker** button places a marker at the current point in the SCE. The marker can be used to annotate, or to reset the patient's physiology to what it was when the marker was placed. For more information, see *Using Markers* section of this user guide.
- The **Fast-Forward** button accelerates the SCE time to a 4x or 8x speed. This feature is optimal to obtain monitor readings for the learners to reference during scenarios, as well as advance the states and programmed physiology in a shorter amount of time.
- The Pause/Play button pauses the SCE (and the simulator) or starts the SCE if it has been paused. The Pause/Play button also returns the SCE time to normal speed after Fast-Forward has been selected.




Using Markers

To annotate the timeline, click the white **Marker** button to the right of the timeline.




A blue marker appears on the timeline bar and the Marker panel appears. Notes or comments may be added as needed. click **Save**.

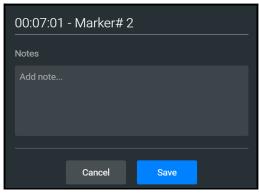

Marker Panel

To return to the patient status at the time a marker was created, click the marker on the timeline, then click **Revert**.

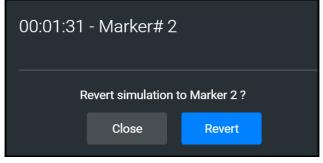

Revert to Marker #2

The patient's status returns to the selected point in the timeline.

Note: When running an SCE, the SCE time continues moving forward and does not reset to the marker time.

THIS PAGE INTENTIONALLY LEFT BLANK





PATIENT DISPLAY

The Patient Status appears when running on the fly or running an SCE.

The Patient Status displays the same information except when running an SCE:

- the Current State (e.g. Patient Baseline) appears
- the Scenario icon appears at the upper-right of the Run screen

Patient Status Display

Patient Status display appears in the main frame on the Run Screen when running on the fly or running an SCE.

The status includes numeric data and waveforms, as well as the Patient Baseline if running an SCE.

÷	Running SCE Skill Validation (Manual)	D			F Juno	
Alarm icon	86 y.o. / F / 5 π 9 in / 121 lb	= %	Current State Patient Baseline			- Patient Baseline
Waveforms		^{нк} 72	Sounds	Pulse	Speech	
*٩		/ Z bpm				
Monito Signal						
_ 😭						
Equipme	ent ECG III		a de			
Widgets -	119 / 79 мар 119 / 79 млнд	RR 11 breaths/min	1280 / 1280			
	^{Sp02} 98 ^{TBody} 97.7				4.6	

Patient Status Display

From this screen, users can:

- Click on the Alarm icon to mute or unmute alarm sounds.
- Click on waveforms, numeric widgets, and other parameters to adjust them.
- edit a layout.

Patient Controls

The patient's current state is shown on the Patient Status Display, allowing users to easily adjust parameters and patient states. The following controls are available whether running on the fly or running an SCE:

- Sounds
- Pulse
- Speech
- Respiratory

Eyes

To change Eye states:

1. From the Patient Status Display panel, tapclick on the Eyes button or the eyes illustration.



Patient Baseline - Eyes Button and Illustration





The Eye states panel appears.

Eyes						×
		Blink Sp	eed			
		Norma	al			
R ◀	7		Control Eyes Toget	her	•	L
Auto			Blink Mode		Auto	
Brisk			Light Reactivity		Brisk	
5mm			Pupil Diameter		5mm	
•	•	0	۲	•		0
Normal	Jaundice	Bloodshot	Hemorrhage Both Right	Keyhole Pi	upil Droopy Eyelids	Cataracts
				0		
	Pan	ning ON O	Consensual Pup	il Response		
	Eye	es Brightness (%)				
	0			100	-	

Eye States Panel

2. Adjust eye states including blink speed, blink mode, light reactivity, pupil diameter, and pupil states as desired by using the drop-down menus and toggles. Close the panel (by clicking the **X**) to apply the changes.

Patient Display

Sounds

To change Sounds:

- 1. From the Patient Status Display screen, click on the **Sounds** button.
- 2. Adjust Sounds including heart, lung, bowel (non-gravid patient), and Korotkoff sounds by choosing each sound tab and using the drop-down menus, toggles, and sliders as desired. Close the panel (by clicking the **X**) to apply the changes.

Sounds			
Heart	Lungs	Bowel	Korotkoff
ALL Status ON	Type Normal	▼ Volume	•
LEFT UPPER		RIGHT UPPER	
ON Norm	nal 🔻	ON Normal	-
LEFT LOWER		RIGHT LOWER	
ON ON Norm	nal 🔫	ON Normal	-
	— • —		

Patient Baseline - Sounds States Panel




Pulse

To change pulse parameters:

1. From the Patient Status Display screen, click on the **Pulse** button.

The Pulse panel appears.

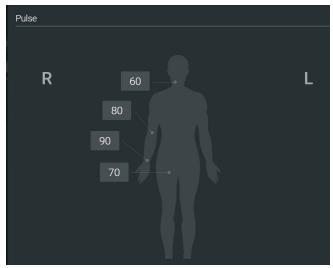

Pulse Deficit Default Values

The numeric widgets display the *pulse deficit value* for each associated location on the patient body diagram. This is the value at which the pulse will disappear if the systolic blood pressure drops below that value. The pulse deficit values default settings are the accepted clinical values, but these may be modified using the controls, if desired.

2. Click on a numeric widget, and the associated pulse parameter control window will open allowing you to adjust the pulse parameter controls.

Radial Pulse - Rig	jht	×
Status		
Intensity		
Absent Weak	Normal	Bounding
Deficit (mmHg)		+
0	300	90
		=
	Cancel	Accept

Pulse Parameter Controls

3. Click on the **Accept** button to apply the changes.

Patient Display

Speech

Embody comes equipped with many speech options and vocal speech sounds. Speech can be turned on or off and volume can be adjusted.

To choose speech and vocal speech sounds:

1. From the Patient Status Display screen, click on the **Speech** button.

The Speech panel appears.

Speech					
	Status 이	🚺 Volume 드			
VOCAL SPEECH SO	DUND				
None	-				
SPEECH SOUNDS					
Vocal Sound C	onversation Symp	otoms			
Loud cough	Short Loud Cough	Short Soft Cough			
Grunt	Soft cough				
Aching	Dull	Pressure	Sharp	Stabbing	
Ouch	Scream				

Speech Panel

- 2. To turn the speech on or off, click on the **Status** toggle. Adjust the slider next to the Status toggle to control the speech volume.
- 3. Click on the **Vocal Speech Sound** drop-down box to open the menu and select an option as desired.
- 4. Click on **Vocal Sound**, **Conversation**, or **Symptoms** to view the list of speech options. Click on a speech option as desired.



Respiratory

To adjust respiratory parameter controls:

1. From the Patient Status Display screen, click on the airway avatar.

The Respiratory Parameter Controls panel appears.



Respiratory Parameter Controls

2. Adjust the controls as desired. Close the panel (by clicking the **X**) to apply the changes.

Change the Control Widget Layout

You can change the layout of the patient vitals display and save parameters as part of the layout.

To change the control widget layout:

1. In the Run Screen, click on the **Patient Status** button.



The Edit Vitals Layout & Alarms panel appears.

Edit Vitals Layout & Alarms		×
($\widehat{{f U}}$) Drag and drop the widgets from right to left. Click on a widget to select	the parameter to display and to adjust a	larm settings.
ECG I HR S	XXXX UNIT	
ECG II		
ECG III	XXXX	
ABP AAP AR		
Tildal Volume TBody Balance TBody TB		
Load Saved Layout		Cancel Accept

Edit Vitals Layout and Alarms

- 2. Either begin modifying the baseline layout or click on **Load Saved Layout** to select another patient display layout to modify.
- 3. Drag and drop a waveform or numeric widget from the right side of the panel to an available display space on the left.

Once in place, the Select Waveform or Select Numeric Widget panel appears.

- 4. Click on a widget to modify which value will be displayed.
- 5. To delete a widget, click on the **X** to delete it.

Note: When a widget is deleted, there will be a blank space. Widgets are not rearranged.

- 6. Select or change the physiological parameter(s) as needed. Click **Save**.
- 7. When the Edit Vitals Layout & Alarms panel appears either:
 - ° Click Accept to update the layout without saving it.
 - ° Click Save to Library to save the layout

Note: When changes are applied in Runtime, the layout is stored for Runtime only. In the case of the SCE editor, the changes would be stored with the SCE so that a copy of the layout is locally stored with the SCE. This way, they are not impacted by changes to the layouts, or if the SCE is used with another simulator.





Alarm State

When an alarm is not muted and a value reaches critical state, the associated widget flashes.



Alarm State Button - Unmuted

To mute the alarms, click on the **Alarm State** button. The icon will turn blue when it is muted.

Markers

The use of markers in the timeline of the SCE helps indicate significant events or notes to revisit when debriefing the simulated clinical experience with the learner.

To add a marker:

1. Click on the **Marker** icon.

A timestamp window will appear. You can add a note if desired.

2. Click Save.

Editing Parameters


You can edit parameters in one of two ways:

- Parameter Editor
- Quick Edit

To edit a parameter in the Parameter Editor controls:

- 1. Click on a waveform or vital sign widget to open the Parameter Editor controls.
- 2. Click on the radio button to Override.

3. Use the sliders, text fields, or (+) and (-) to adjust parameters.

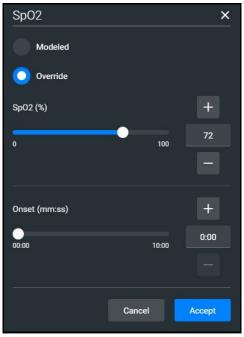

Parameter Editor Controls

4. Click Accept.

To edit a parameter using Quick Edit:

1. Click and hold a waveform or vital sign widget until (+) and (-) icons appear over the widget.



Plus/Minus Widget

- 2. Click the (+) or (-) signs to adjust the parameter.
- 3. Click outside of the Quick Edit area to save the changes and close the editor.

Panels, accessed from the Run screen, allow the application of conditions, medications and treatments during simulation. Once applied, changes are reflected in the patient's physiology and logged.



Onset, Read-Only, and Override Indicators

Onset indicators are blue, rotating circles that indicate when the value of the parameter is in transition. Read-only indicators are yellow triangles that indicate a parameter is read-only and cannot currently be modified by the user (occurs in Manual Mode only).



Onset and Read-only Indicators in Manual Mode

Override indicators are yellow diamonds indicating a user selection is currently overriding a modeled value (occurs in Modeled Mode only).



Onset and Override Indicators

THIS PAGE INTENTIONALLY LEFT BLANK



MANAGING SCES

The Maestro SCE Manager is where you can view and print information for all SCEs. You can also create, edit, copy, or delete custom SCEs and Learning Modules.

To access the SCE Manager screen, from the Maestro Home Screen click **SCE Manager**.

Search	New SCE More	e menu		Sort by
CAE Digital Learning Solutions	CAEMaestro X (ii) CAEEnbody	× +		- 0 ×
$\leftarrow \rightarrow$ C \oplus digitaltraining-	d.canadacentral.cloudapp.azure.com/dab19420-508c-44b6-befe-c	a08cb7099ec/sce-manager		☆ ⊖ :
MS CAEMaestro 🗸				0
Home SCE Marager	History			Ares (Standalone)
Q Search	New SCE More 👻			ort by SCE Title (A-Z) 👻
Recently Used	SCE Title	Created by	Mode	
	Closed Head Injury and Pneumothorax	CAE	Manual	* 🕨
Favorites	Heroin Overdose	CAE	Manual	* ►
Preconfigured SCEs	A Heroin Overdose	CAE	Modeled	* 🕨
User-created SCEs	Inferior-Posterior Myocardial Infarction	CAE	Manual	* 📐
CAE Learning Modules >	Inferior-Posterior Myocardial Infarction	CAE	Modeled	* 돈
	Sepsis with Hypotension	CAE	Modeled	* ►
	Sepsis with Hypotension	CAE	Manual	* 🕨
				*

SCE Manager Screen

The default screen shows Recently Used SCEs but you can easily locate any SCE and Module in the SCE Manager.

There are three ways to locate SCEs or modules from the SCE Manager screen:

- In the left column, select either Favorites, All, Preconfigured SCEs, User-created SCEs or User-created Modules.
- In the **Search** field, type part or all of the name of an SCE or module.
- Click on the **Sort by** drop-down to filter the list by ascending or descending order.

Review an SCE

To review details about an SCE, click on the name of an SCE, or click on the **Gear** icon and select **Review** from the drop-down menu.

Modeled	\$
Review	
Сору	
View PDF	
Add to Favorites	

SCE Gear Drop-down Menu Options

The SCE Editor screen appears.

View SCE Details

From the SCE Editor screen, you can select a tab or panel to view details.

The SCE Editor screen is where users can enter specific details for custom SCEs. For more information on entering SCE details into a new SCE, see *Create a Custom SCE* section of this user guide.

SCE Editor Navigation Tabs

• **Details:** Lists relevant information about the SCE. Allows access to the Overview, Learner, and Facilitator panels

For more information, see SCE Editor Panels.

- Patient: Includes the patient profile information
- Scenario: Includes states pertaining to the SCE
- Checklists: Includes checklists related to learner milestones for the SCE
- Patient Records: Stores patient records including but not limited to:
 - ° Lab reports
 - X-rays
 - Medical history
 - ° Healthcare Provider's orders



- Handoff reports
- **Runtime Configuration:** Allows the creation of Quicklinks for conditions, medications, and treatments
- **Monitor Signals:** Allows users to enable or disable parameters to be displayed on the patient monitor at the start of the scenario
- **Preparation:** Lists recommended equipment and supplies needed to run the SCE and manikin setup notes

SCE Editor Panels

- **Overview**: Includes information about the SCE synopsis and descriptions of the states
- **Learner**: Displays learning objectives, performance measures for each state, and preparation questions
- **Facilitator**: Includes notes, debriefing points, teaching questions and answers, as well as clinical references about the SCE

Create a Custom SCE

You can quickly create an SCE by copying an existing SCE, or by creating a new SCE and entering all of the details. Custom SCEs can be used alone or in a Learning Module. For more information, see *Learning Modules* section of this user guide.


Note: The Lock icon on any screen indicates the SCE was installed by CAE and cannot be edited or deleted.

Copy an SCE

One way to create a new SCE is to make a copy of a preconfigured (locked) SCE and then edit that as desired. To create the SCE from an existing SCE.

To copy an SCE:


1. On the SCE Manager screen, click on the **Gear** icon on the right of the desired SCE.

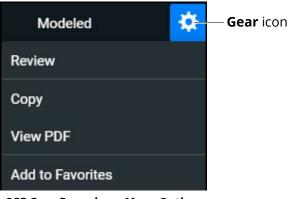

SCE Gear Drop-down Menu Options

2. Click Copy.

The Copy SCE panel appears and lists sections of the SCE you can copy.

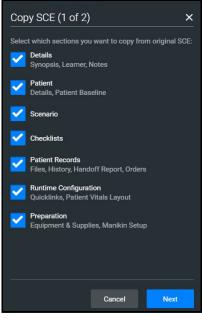

Copying SCE

- 3. Select the elements of the SCE to copy. When finished, click on **Next**. In the **SCE Title** field, enter a new SCE name.
- 4. (Optional) Assign the SCE to a Module using the drop-down menu.





5. Click Copy.

✓ Copy SCE (2 of 2)	×
SCE Title	
Copy of Test	8
Select Module	
None	- +
Created by	
Administrator	8
Mode Modeled	
Back	Сору

Saving Copied SCE

The new SCE is copied and available for selection from the SCE Manager.

Creating a New SCE

Creating a new SCE is a two-step process.


- 1. Create the SCE in the SCE Manager as described in this section.
- 2. Enter details about the SCE in the SCE Editor as described in *Using the SCE Editor* section of this user guide.

To create an SCE:

1. From the SCE Manager screen, click on **New SCE**.

The New SCE panel appears.

- 2. Type a title for the SCE in the **SCE Title** field.
- 3. (Optional) To add an SCE to a module, click on the drop-down arrow and select a module. Or, click the + icon to create a new module.

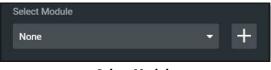

Select Module

4. Type a name for the module in the Enter New Module Name screen and click on Add.

Enter New Mo	dule Name		
	Cancel	Add	

New Module Name Screen

5. Complete the remaining fields in the New SCE screen or select options as required. When finished, click on **Create**.


The SCE Editor screen appears with the **Details** tab displayed.

- 6. In the **Overview**, **Learner**, and **Facilitator** panels, click on the **Edit** icons to enter additional information.
- 7. Click the navigation tabs to enter additional details about the SCE as needed. To learn more, see *Using the SCE Editor* section of this user guide.

Import or Export an SCE

SCEs can be imported from an external device or the hard disk drive where the SCE file is saved. The SCE file extension is *msce*.

To import from the SCE Manager screen, click on the **More** drop-down menu and click on **Import SCEs**.

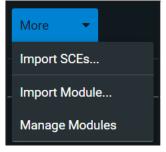


More Drop-down Menu Options



To export an SCE:

CAE

1. In the **SCE Manager**, click on the gear icon next to the desired SCE.

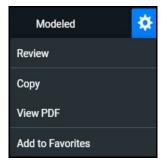

SCE Gear Drop-down Menu Options

- 2. Click **Export**.
- 3. Select the desired location for the *msce* file in your File Explorer window and click on **Save**.

The file is saved in the designated location. To view the PDF of an SCE:

- 1. Click on View PDF.
- 2. In the window pop-up, identify where the PDF file should be saved to and select **Save**. The file can then be opened and viewed from that location.

THIS PAGE INTENTIONALLY LEFT BLANK



USING THE SCE EDITOR

Use the SCE Editor to review or manage any purchased, preconfigured SCEs and to complete or edit information for custom SCEs. From the SCE Editor screen, you can also view checklists and patient records, or delete information as needed.

To access the SCE Editor, click on on the name of an SCE from the SCE Manager screen.

٢	Details	Overview	Learner	Facilitator
•	Patient	Details		
Ш	Scenario	SCE Title Diabetic Ketoacidosis with Hypoxemia Module Preconfigured SCEs		
	Checklists	Mode Modeled Created by CAE		
Ů	Patient Records	Modified on 2020-11-17 Last Used on 2021-02-19		
=	Runtime Configuration	Synopsis		
1 *	Monitor Signals	This Simulated Clinical Experience (SOE) can be placed in several locations, depending on the background of the learner or healthcare provider. The events can take place in the patient's home or in an ambulance, or the patient can be presented as a new arrival to the Emergency Department (ED).		
0	Preparation	A six-year-old female child presents with fever and cough for the past two days. The mother states that take has had tooble getting her to eat or drink since yesterday and today all she will do is like on the couch. The child responds to your questions but is teltargine and appears III. The mother has been testing the fever and cough with over-the-counter medications, however, the child coughs as but all that the card kape to get and the set of t		
		This SCE consists of four states. States 1 and 2 automatically transition based on time in state. The facilitator may manually transition when approvate interventions are performed.		
			30s to 140s, BP in the 80s/50s, RR in the 30s, SpO2 in the high 80s on room air and a temperature of 3 are equal, round and reactive to light and accommodation. The patient's breath has a fruity odor. The j	
			ata collected and document the findings, implement oxygen therapy to keep oxygen saturation greater r an isotonic fluid bolus of 20 mL/kg over 30 minutes. The learner is expected to implement measures	
		If the learner administers a crystalloid infusion of 400 mL and applies	oxygen, the facilitator should give these in the software and manually transition to State 2 Slight Imp	rovement.
		If time in state is greater than 300 seconds, the scenario automatical	y transitions to State 4 Worsening.	
		In State 2 Slight Improvement, the patient's condition improves with	a HR in the 110s to 120s, BP in the 80s to 90s/50s, RR in the upper 20s and SpO2 in the low 90s on 2 L	PM nasal cannula. The patient is lethargic.
		The learner is expected to reassess the vital signs after the fluid bolu	s, interpret the findings and document, repeat the isotonic fluid bolus of 20 mL/kg over 30 minutes, co	ntinue to monitor oxygenation and mental status, and consider a nebulizer treatment.
	Run SCE	When the learner administers the second fluid bolus, the facilitator sh	ould administer the fluid in the software and manually transition to State 3 improved and Admit.	

SCE Details

The buttons in the Navigation panel provide options to:

- Run the SCE.
- Edit the patient or scenario.
- Add checklists or patient records.
- Modify the runtime configuration including preparation information.


Some sections in the SCE Editor include a rich-text editor to allow free-form data entry.

Note: If the SCE is locked, you cannot modify or delete information.

Patient Profile

To edit the Patient Profile:


1. From the SCE Editor, click on the **Patient** tab, then click on the **Edit** button.

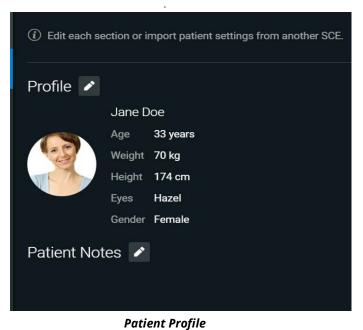

The Patient Profile Editor appears.



2. Modify the patient's name, eye color, age, gender, weight, and height in the appropriate fields.

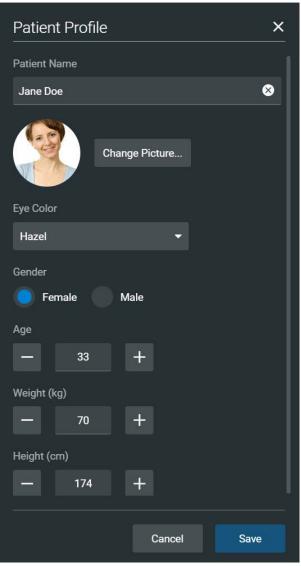

Patient Profile

3. Click on **Change Picture** to change the patient's picture.

Note: When using the Embody Simulator, the option to **Change Profile** will be available, allowing edits to all of the patient parameters on the Patient Profile panel. This option also allows for selection of the patient that is visually represented as the virtual simulator in the Embody screen. See the chapter *CAEEmbody Virtual Simulator* for more information.

4. Click on Save.

IMPORTANT: No part of the patient's profile can contain any special characters, such as / \ : * ? < > % | "

Setting a Patient's Baseline

The patient baseline is the patient's initial status at the start of an SCE.

To set the Patient's Baseline:

1. In the **Patient** section, click on the **Edit** button on the right of **Baseline**.

The Patient Baseline Editor appears.

2. Set the Patient's baseline status by modifying the desired parameters and click on **Save**.

When the SCE begins, the Patient status reflects the selected baseline settings.

To use the text editor:

- 1. Where available, click on the **Edit** button.
- 2. Enter or edit information as needed.

Note: Text can be copied into the fields from Text Editor or Notepad only.

Edit Synopsis		×
B / ⊻ Ε Ξ Ξ Ξ		
	Cancel	Save

Text Editor

3. When finished, click on **Save**.

Note: Notes can also be added to each scenario state when editing a Scenario.

Scenarios

When creating a custom SCE, users can create new scenarios or modify an existing unlocked SCE using the Scenario Editor or Live Scenario Editor.



Note: The live editor is only available within unlocked SCEs that use the optional Modeled Physiology, and allows live edits to the scenario while running the simulation.

To edit from the Scenario Editor:

1. From the SCE Manager screen, select an SCE to edit.

The SCE Editor screen appears.

- 2. Click on the **Scenario** tab.
- 3. Click on the Scenario **Edit** icon.

Note: If it is a brand new SCE, click on the **Open Scenario Editor** button to open the Scenario Editor.

The Scenario Editor appears.

Cenario Editor (Manual)				
Initial Assessment	*	State Name		
Condition Deteriorates	¢	Initial Assessment		8
Condition Detenorates	¥	Parameters	Transitions	Notes
Add State			Edit Parameters	

Scenario Editor

- 4. Click on the **Gear** icon located on the side of a state to do one or more of the following:
 - **Rename**: to rename the state
 - **Copy:** to copy the state
 - Delete: to delete the state
 - Edit Parameters: to edit the state parameters without jumping to the specified state
 - Add Transition: to add a transition from one state to another
 - Edit Note: to add state notes

Using the SCE Editor

From the Live Scenario Editor, states can be added, modified, and deleted. Patient Profile and other SCE sections can only be modified in the SCE Editor.

To edit from the Live Scenario Editor:

- 1. From the Scenario Editor screen, click on **Live Scenario Editor**. When the informational window appears, click on **Close** and the Live Scenario Editor screen opens.
- 2. Click on the **Gear** icon located on the side of a state. Then click on one or more of the following:
 - Jump to State: allows users to jump to the desired state. It is possible to perform a live edit, by selecting the Edit Live button. Once completed, parameter changes will be applied to the state. This option allows users to edit state parameters and see the resulting physiological impact on the patient.
 - **Rename**: to rename the state.
 - **Copy**: to copy the state.
 - **Delete:** to delete the state.
 - **Edit Parameters**: to edit the state parameters without jumping to the specified state.
 - Add Transition: allows transitions to be added from one state to another.
 - Edit Note: to add state notes.

At any time, the simulation may be restarted by clicking the **Restart Simulation** button at the bottom of the screen.

Add, Modify or Delete Scenario States

A scenario state is comprised of conditions and/or parameters. A state may also include automated transitions to other states and notes for the instructor.

For scenarios, users can:

- Create new states and options.
- Add conditions and/or parameters to a state, or copy a state into a new state.
- Modify or delete portions of a state such as conditions, parameters, transitions, or the entire state itself.
- Add and delete transitions.

Add Scenario States and Parameters

You can add states, conditions, and parameters in the Scenario Editor or Live Scenario Editor.





To add a scenario state:

- 1. In the Scenario Editor or Live Scenario Editor, click on Add State.
- 2. In the window that appears, enter a name for the new state. Click on **Create**.

The State window appears, where you can add parameters to the state.

Scenario	
State 1 (i)	*



To add parameters in the Live Scenario Editor:

- 1. In the Live Scenario Editor, select the desired state, and then select **Jump to State**. All changes made to controls will be added to active state.
- 2. Click on the **Gear** icon and select **Edit Live**.
- 3. Add as many parameters as needed.

Parameters appear consecutively within the state.

- 4. Move to different states to add or edit parameters as needed using the **Jump to State** feature. Repeat this step as needed through multiple states.
- 5. When finished, click on **Return** to save and exit the editor.

IMPORTANT: If the physiology of any parameter conflicts with other parameters, Maestro will retain the last parameter entered.

To add parameters in the Scenario Editor:

- 1. In the Scenario Editor, select the desired state.
- 2. On the right side of the screen, click on **Edit Parameters**.
- 3. Add conditions or parameters as needed by selecting them from the list on the left side of the screen. Once added, they appear on the right side of the screen.

Modify or Delete Scenario States or Parameters

To modify a scenario state:

- 1. In the Scenario Editor or Live Scenario Editor, select the desired scenario state. Then, click on **Edit Parameters**.
- 2. Make necessary changes. Click on **Save**.

Using the SCE Editor

To delete a scenario state in the Scenario Editor or Live Editor, click on the **Gear** icon, then click on **Delete**.

To delete parameters:

1. In the Scenario Editor or Live Scenario Editor, click on the **Information** icon to expand the state.



- 2. Click on the **Parameters** tab. Click on the **X** icon next to the parameter to delete.
- 3. Click Save.

3	
lg ≻	×
°C >	×
	°C >

Parameters Tab

Add and Delete Transitions

To add transitions, the scenario must have both an original state and a state that results from the transition. Add or delete transitions from the Scenario Editor or Live Scenario Editor.

To add transitions:

- 1. In the Scenario Editor or Live Scenario Editor, click on the **Gear** icon next to the desired state.
- 2. Click on **Add Transition**.



The New Transition panel appears.

New Transition		×
Reason for transition		
Treatment		-
Select treatment		
Defibrillation Energy		-
Criteria		
Greater than (>)		-
Defibrillation Energy		+
0	36	O
Jump to		
Condition Deteriorates		-
	Cancel	Save

New Transition Panel

3. Select a Reason for Transition from the list, then select additional options as prompted.

Example: If the administration of medication is the desired reason for transition, select Medications, then select the desired medication from the list that appears.

- 4. Once the transition option is selected (ex: Medication), enter the criteria, transition value, and state that results from the transition.
- 5. Multiple transitions may be entered for a state to create alternate pathways. Follow the same steps to make selections and assign values to the Treatment, Assessment, Vitals, Medication concentration, Fluids, Time in Scenario, and Time in State variable types.
- 6. When finished, click on **Save**.

Using the SCE Editor

To delete a transition in the Scenario Editor or Live Scenario Editor:

1. Click on the **Information** icon next to the state.



2. Select the **Transitions** tab, then click on the **Trash** button next to the transition to delete.

State Name		
Initial Assessment		8
Parameters	Transitions	Notes
lf Defibrillation Energy >0 J Jump to		✓ Î
Condition Deteriorates		-
	Add Transition	

Scenario Editor - Transition Tab

Checklists

You can create new checklists or import existing checklists from the library.

To create a new checklist:

- 1. In the SCE Editor, click on the **Checklists** tab.
- 2. When the Checklists panel appears, click on **Create New**.



The New Checklist panel appears.

CAE

New Checklist			×
Checklist Name		Checkbox Label	
Initial Assessment	8	Performs a complete assessment	⊗
Checklist Criteria			
Performs a complete assessment	Ō		
Add Checkbox Add Dropdown			
		Cancel	ave

New Checklist Panel

- 3. Enter a name for the checklist in the Checklist Name field.
- 4. (Optional) Click on Add Checkbox to add a checkbox to the checklist.
 - a. Enter a label name in the Checkbox Label field.
 - b. (Optional) Click on **Add Checkbox** to continue entering labels as needed.
 - c. Click on Save.
- 5. (Optional) Click on Add Dropdown to add list items to the checklist.
 - a. Enter a name for the drop-down list in the Dropdown Label field.
 - b. Click on **Add Option** to provide items that can be selected from the drop-down list.
 - c. Enter a name for the list item in the Dropdown Options field.
 - d. (Optional) Click on **Add Dropdown** to continue entering drop-down lists as needed.

Using the SCE Editor

- e. Click on Save.
- 6. (Optional) Rearrange the checklist order by dragging individual checklists up or down as desired
- 7. (Optional) Click on the checklist Gear icon next to the created checklist to Save to Library (or to Rename, Edit, Copy or Delete).
- 8. (Optional) Use the Edit function to:
 - a. Edit the Checklist Name.
 - b. Edit the Checkbox Label.
 - c. Rearrange the order of the Checklist Criteria by dragging any criteria up or down.
 - d. Add a Checkbox or Dropdown.

To import a checklist:

- 1. In the SCE Editor, click on the **Checklists** tab.
- 2. When the Checklists screen appears, click on **Import from Library**.



The Import Checklist panel appears and displays all checklists from all SCEs in the system.

Import Checklist	×
CAE CHECKLISTS	Identify rhythm
Cardioversion	Consider sedation
Catheterization	Safe use of defibrillator
Defibrillation	Correct cardioversion technique
Drug	Document in notes/chart
Electrical Therapy	
External Pacing	
Eye Irrigation	
General Assessment	
Intubation	
IV Administration	
	Cancel Import

Import Checklist Panel

- 3. Highlight the checklist.
- 4. Click on **Import**.

Patient Records

Patient records can be uploaded into Maestro for display in the CAE Patient Monitor software. Once uploaded, a patient record is available for use with the SCE.

Note: A single patient record file cannot exceed 20MB.

To ensure adequate space, delete patient records that are no longer needed.

To access a patient record:

1. From the SCE Editor screen, click on the **Patient Records** tab.

The Patient Records list appears with all available patient records shown.

Using the SCE Editor

2. (Optional) To preview a patient record before sending to the Patient Monitor, select the record, and click on the **Preview** button.



Preview Button

3. Select a patient record, then click on **Send to Patient Monitor**.

Upload Patient Records

You can upload the following file types:

- GIF, JPG, PNG, and XPS images
- MPEG and MOV videos
- PDF documents
- MP3 audio Files

To upload a patient record:

- 1. In the SCE Editor, click on the **Patient Record** tab.
- 2. In the panel that appears, click on **Patient Files > Add File**.



The New Patient File panel appears.

New Patient File		×
File		
Upload		
Name		
Description		
	Cancel	Create

New Patient File - Upload Panel

- 3. Click on Upload. Select the desired file and click on Open.
- 4. When the file name and description appear, modify as needed in the text fields to reflect the desired file name and description.
- 5. Click on **Create**.

Edit or Delete Patient Records

You can only change the name or description of the patient record. You cannot edit the content of the record from the Patient Record tab.

To edit a patient record name:

- 1. From the list of Patient Files, click on the **Gear** icon next to the desired file.
- 2. Click on **Edit**. Change the **Name** or **Description** as needed.
- 3. Click on Save.

To delete a patient record:

- 1. From the list of Patient Files, click on the **Gear** icon next to the desired file.
- 2. Click on **Delete**.
- 3. When the message appears to confirm your selection, click on **Delete**.


Runtime Configuration

Click on **Runtime Configuration** to access the Quicklinks setup as well as the Vitals Layout & Alarms setup screen.

Quicklinks Setup

From the Quicklinks Setup screen within the SCE Editor, conditions, medications, and treatments can be preconfigured for the SCE by creating Quicklinks. For more information about using Quicklinks, see the *Quicklinks* section of this user guide.

To navigate through available conditions, medications, and treatments, click on the **Conditions**, **Medications** and **Treatments** buttons.

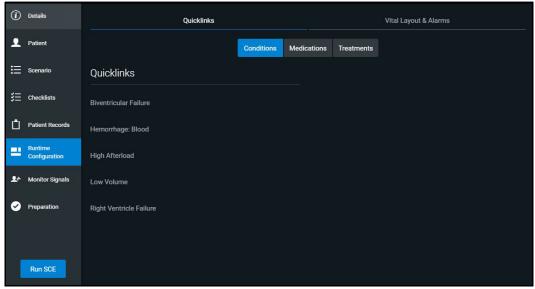

Runtime Configuration Quicklinks Setup Screen

To create a Quicklink, select an item from the **Conditions**, **Medications**, or **Treatments** lists.

To remove a Quicklink, deselect the option.

Edit Vital Layout & Alarms

To edit the vital layout and alarms, click on the Vital Layout & Alarms tab.



Any current vital widgets and alarms appear. If the SCE is new, click the **Edit** button to the right of **Vital Layouts & Alarms** and begin adding vital widgets and alarms by dragging the desired widget template from the right side of the screen to the left.

Edit Vitals Layout & Alarms			×
(\widehat{I}) Drag and drop the widgets from right to left. Click on a widget to select	the parameter to display and to adjust a	larm settings.	
	XXXX UNIT		
	XXXX		
Load Saved Layout		Cancel	Accept

Runtime Configuration Vital Layout & Alarms Screen

Upon dragging the widget, the Select Numeric Widget or Select Waveform box will appear. Select the desired vital or waveform from the available options, modify alarms and thresholds as desired, and click on **Save** when complete.

The option to load a saved layout is also available by selecting the **Load Saved Layout** button.

When all modifications are complete, select **Accept**.

THIS PAGE INTENTIONALLY LEFT BLANK



VIRTUAL EQUIPMENT

Maestro Evolve offers virtual equipment that may be used during the simulation, providing user interfaces that mirror authentic medical equipment. Running equipment, modifying settings, and applying treatments will impact the physiology of the patient status in Maestro offering a deeper learning experience. When using virtual equipment during a simulation, it may also be visually represented by launching the virtual patient.

Virtual equipment options include:

- **CAEPatient Monitor** an on-screen patient monitor with preconfigured layouts or the ability to modify settings for the user's preference. Adjustable settings, 12 Lead ECG reports, and the ability to capture a patient's status from the display are other examples of the Patient Monitor features.
- **CAEAED** a virtual automated external defibrillator that is easy to use, providing audible prompts guiding the learner on appropriate action to take.
- **CAEDefibrillator** an electrotherapy device that paces and delivers cardiac shocks to the virtual patient during the simulation, providing the learner with knowledge of how to use this life-saving device.
- **CAEVentilator** offers both invasive and non-invasive options, various modes, and a display featuring waveforms and audible alarms. Much like bedside medical equipment, the virtual ventilator allows for monitoring and treating the virtual patient with respiratory needs.
- **CAETransport Ventilator** has many of the same features as the CAE Ventilator, though providing an interface that more closely resembles equipment used for portable respiratory support.
- **CAEAnesthesia Machine** provides the learner with anesthesia technology, by delivering the ability to virtually administer medical gases, adjusting pressure and flow while monitoring and providing respiratory ventilation.

CAE Patient Monitor

Patient Monitor can run in a browser window, while the Maestro simulation is running in a separate window, and a third window may run additional virtual equipment (e.g. ventilator). When using dual monitors it allows the user to view the Patient Monitor at the same time as the simulation, enabling a view of impact to patient status as parameters like blood pressure and heart rate as they are modified. See more information in the *Appendix A - FAQ* of this guide for configuration of dual monitors.


This section will describe how to:

- Launch Patient Monitor
- Use Patient Monitor

Launch Patient Monitor

To launch the Patient Monitor:

1. Keep the Maestro Evolve window open. From the upper-right corner of the SCE Run screen, select the Maestro Application drop-down menu.

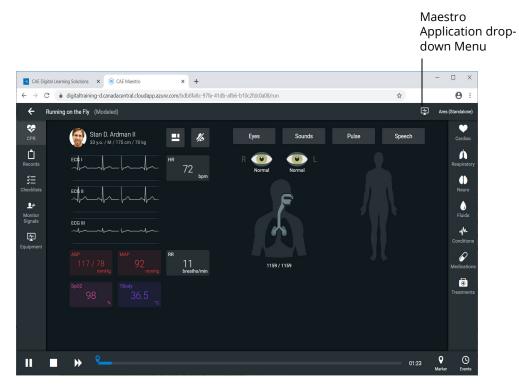

SCE Run Screen




The Maestro Application drop-down menu appears.



Maestro Application Drop-Down Menu

2. Select **CAEPatient Monitor**. The Patient Monitor appears in a separate browser tab.

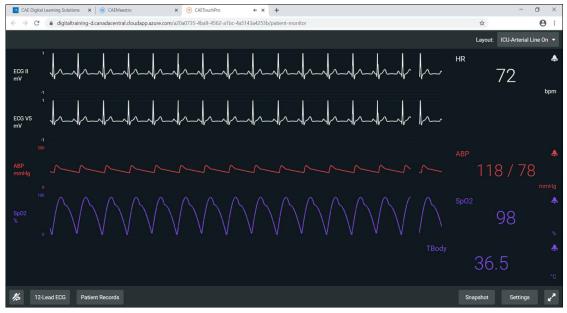

CAE Patient Monitor

Patient Monitor will display wave forms and vital signs consistent with parameters set in Maestro Evolve.

To silence all of the Patient Monitor sounds, click the **Mute** button in the lower-left corner of the screen.



When using a single monitor:

- 1. Restore or resize the Maestro Evolve screen so that some of the Windows desktop is visible.
- 2. Position the mouse pointer on the **CAEPatient Monitor** tab in the browser window.
- 3. Drag the **CAEPatient Monitor** tab outside of the active browser window. The Patient Monitor will appear in a separate window.
- 4. Minimize the Patient Monitor window.

This will allow the monitor to stay active while other equipment options are selected and utilized. The monitor window may be restored at anytime to view or modify during the simulation until the window is closed.

See Appendix A - FAQ for information on configuration of dual monitors.





Select a Preconfigured Layout

These are the preconfigured CAE Healthcare Layouts available in Maestro Evolve:

- **ICU-Arterial Line Only** waveform and numeric readouts for ECG Lead II, ECG Lead V, ABP, Pleth, and a numeric readout for Body Temperature.
- **EMS-ED-Telemetry** waveform and numeric readout for ECG Lead II and numeric readouts for SpO₂, and NIBP (noninvasive blood pressure).
- **ICU-OR No CVP** waveform and numeric readouts for ECG Lead II, ECG Lead V, ABP, PAP and Pleth, and numeric readouts for NIBP, Blood Temperature, and Body Temperature.
- **ICU-OR** waveform and numeric readouts for ECG Lead II, ECG Lead V, ABP, PAP, CVP and Pleth, and numeric readouts for NIBP, Blood Temperature, and Body Temperature.
- **Saturation-Pulse** numeric readouts for SpO₂ and pulse.

Note: Additional layouts may be available depending upon the simulator that is used.

To select a preconfigured layout:



1. Click the **Layout** drop-down menu in the upper-right corner of screen.

Layout Drop-down Menu

The Patient Monitor Settings menu appears.

2. Select a layout.

Settings

Patient Monitor settings options allow for configuration preferences. The following selections are available on the settings panel:

- Layout customize specific vital signs, colors, wave forms, numerics and specific location of the parameters on the display.
- Audio specify alarm ranges and suspension times.
- Sweeper Speed adjust intervals from 3 to 60 seconds for sweeper speed.
- Language select preferred language.

To access the Settings panel:

1. Click **Settings** in the lower-right corner of the screen.

The Settings menu appears. The **Layout** tab is displayed by default.

Settings		×
Layout	O ICU-Arterial Line Only	
Audio Setup	EMS-ED-Telemetry	
Sweeper Speed	ICU-OR NO CVP	
Language	ICU-OR	
About	Saturation-Pulse	
	Edit Save As Delete	

Settings Panel - Layout Options



Layout

To access the Layout options:

1. From the **Settings** panel, the **Layout** tab is displayed by default. From the Layout options, select a Layout. In this example, ICU-Arterial Line Only is selected.

Settings	×
Layout OILU-Arterial Line Only	
Audio Setup EMS-ED-Telemetry	
Sweeper Speed	
About Saturation-Pulse	
Edit Save As Delete	
Euit Save As Delete	

Settings Panel - Layout Options

2. Click Edit.

The Layout Edit screen appears.




ICU-Arterial Layout Edit Screen

CAE Virtual Equipment

3. Select a signal to edit its parameters.

The Insert a parameter screen appears.

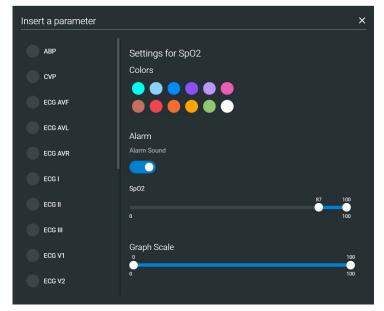

Insert a Parameter

4. Adjust the signal's **Color**, **Alarm**, and **Graph Scale** parameters as needed.

OR

Select a parameter to insert it in the place of the selected parameter.

5. When finished, click the **X** in the upper-right corner to close the panel.

The Layout Edit screen reappears.



6. Use the plus (+) buttons to add waveforms to the layout.



Layout Edit Screen

Patient Monitor adds a line to the Layout.

- 7. Select the new line to open the Insert a Parameter window.
- 8. Select the parameter to display on the new line.
- 9. Set the parameter's **Color** and **Alarms.** When finished, close the parameter panel.
- 10. Click **Done** to close the Layout Edit screen.

The Settings Layout panel opens.

- 11. Click **Edit** to resume editing the layout.
- 12. Click **Save As** to save the layout.

Type a name for the layout, then click **Save**.

CAE Virtual Equipment


Audio Setup

To silence all sounds, click the **Mute** button in the lower-left corner of the screen.



To set up the audio for Patient Monitor:

- 1. Click **Settings** in the lower-right corner of the Patient Monitor screen.
- 2. From the Settings panel, select the **Audio Setup** tab.

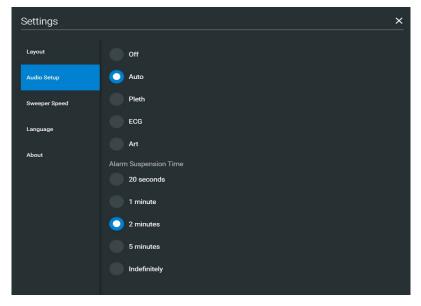

Settings - Audio Setup Options

- 3. (Optional) Select a waveform to set it as the pulse sound trigger.
- 4. (Optional) Select an **Alarm Suspension Time** to disable the selected waveform for an indicated amount of time.
- 5. When finished, click **X** to close the window.



Snapshot

The snapshot tool captures an image of the data that is displayed on the screen. The image may then be printed or saved. This function is unavailable if the device is connected to a wireless network where users cannot print a report.

To capture an image:

1. Click **Snapshot** at the lower-right corner of the Patient Monitor screen.

The Snapshots window appears showing the live data available to capture.

Snapsho	ots			×
	HR	ABP	Sp02	TBody
Current	71 bpm	119 / 79 mmHg	98 %	36.5 ℃
00:29:03	72 bpm	119 / 79 mmHg	98 %	36.5 ℃
				o 🖍 🕂

Snapshots Window


- 2. To take another snapshot, click the **Capture Snapshot** (camera) button at the lower-right corner of the screen.
- 3. (Optional) Click the **Full Screen View** button at the lower-right corner of the screen.

Click the **Esc** key to exit full screen view.

12-Lead ECG


To view a 12-lead ECG report:

1. From patient monitor controls at the lower-left corner of the screen, select **12-Lead ECG** .

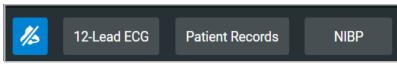


Patient Monitor Controls

The report appears.

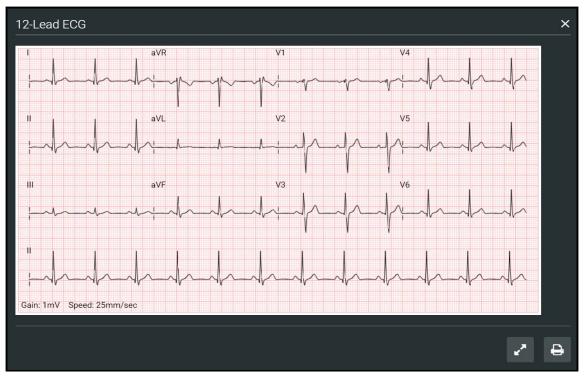

12-Lead ECG Report

2. (Optional) To view the full screen, click the **Full Screen View** button at the lower-right corner of the 12-lead ECG report.

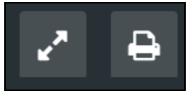

Full Screen View and Print Buttons

3. Click the **Esc** key to return to the inset view.



4. Click the **Print** button to print the report.

The Print Settings window appears.

10/16/2020	CAEPatientMonitor	<u></u>	Print	2 sha	ets of paper
1			r mit	a sne	era or haber
H~			Destination	- NPI616F34 (HP	Laser、 🔻
wh			Pages	All	*
<u>j</u>			Copies	1	
p.			Layout	Portrait	*
Gain: 1mV Speed: 25mm/sec					
			More settings		~
				Print	Cance

Print Settings Window

- 5. Select the print settings on the right side of the window.
- 6. Click **Print** to print the report.

Patient Record

To view Patient Records, click **Patient Record** from patient monitor controls at the lower-left of the screen.

The Patient Records window appears.

Patient Files	History	Handoff Report	Orders
Classic Findings in Pneumor	nia Image	POCUS obtained from EMSimCases.com and contributed to EMSIMcases.com by http://www.thepocus atlas.com/pulmonary	Z
Classic Findings in Pneumor	nia Image	and contributed to EMSIMcases.com by http://www.thepocus	

Patient Records Window

This information provides access to the same Patient Files, History, Handoff Report, and Orders information contained in Maestro.



NIBP Cycling

When non-invasive blood pressure (NIBP) is displayed, the patient's NIBP can be updated at specified intervals using NIBP Cycling.


To set NIBP cycling:

- 1. Select **NIBP** from patient monitor controls at the lower-left of the screen.
- 2. Click Manual.

OR

3. Click Select.

Auto cycling options will appear 1 minute to 60 minutes.

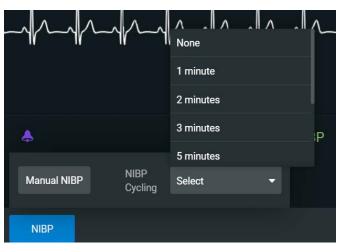

NIBP Auto Cycling Options

4. Select the desired interval for the cycling.

The NIBP Cycling timer will appear at the lower-center of the patient monitor screen.



NIBP Cycling Timer

Note: Manual NIBP can be used at any time during cycling, however it turns off auto-cycling.

CAE Defibrillator and CAE AED

Listed below are the steps to get started using CAE Defibrillator and CAE AED.

Note: Only one electrotherapy device may be used at a time. For example, if the AED is active, and the Defibrillator is powered on, the AED will automatically power off.

Defibri	Defibrillator and AED Steps		
1	Set up the Patient for Electrotherapy		
2	Set up the Defibrillator		
3	Launch the Defibrillator		
4	Set up the AED		
5	Launch the AED		

Step 1: Set up the Patient for Electrotherapy

To perform a realistic and effective cardiac electrotherapy simulation, parameters are set to reflect a patient's condition that would require use of a Defibrillator or AED. This section provides information on modifying and activating the appropriate parameters.

In the case that Maestro Evolve is running but another virtual device is in the active window, it may be necessary to switch windows.

To switch windows:

1. If a virtual device window (e.g. Patient Monitor or Ventilator) is open, minimize the device window.

OR

At the top of the browser window, select the **CAE Maestro** tab.


	CAE Mae Tab 	stro	
CAE Digital Learning Solution 🗙	CAEMaestro ×	(B) CAEEmbody	× 阃 CAEAED
\leftrightarrow \rightarrow C \bullet digitaltrainin	g-d.canadacentral.cloudapp.azure.co	om/edf68cff-c4a2-4e95-819b-	d3604924b6b4/run
← Running on the Fly	(Modeled)		
	Ardman II	🔏 Eyes	Sounds

Browser Window Tabs

Tip: In Windows 10, holding down on the **ALT** key and pressing the **Tab** key will allow you to alternate between open windows.



The SCE Run Screen appears.

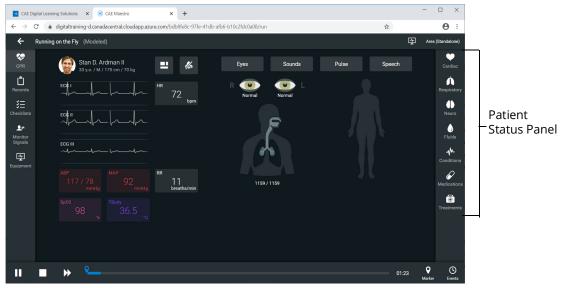


SCE Run Screen

2. From Patient Status panel, on the right side of the SCE Run Screen, click the **Cardiac Parameters** icon.



The Cardiac Parameters panel appears.

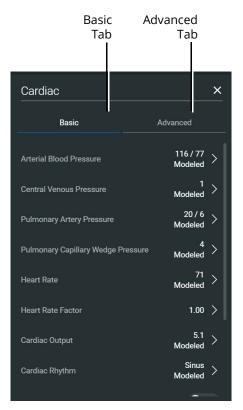


Cardiac Parameters Panel

3. From the Cardiac Parameters panel, select **Cardiac Rhythm**.

The Cardiac Rhythm Options appear.



4. Toggle Modeled to **Override**.

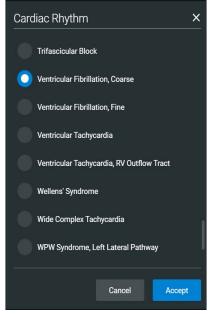

Cardiac Rhythm Options

5. The arrhythmias in the following table result in a shock advised state for the Defibrillator and AED.

Cardiac Rhythm Parameters - Arrhythmias				
Basic	Coarse Ventricular Fibrillation			
	Fine Ventricular Fibrillation			
	Torsade de Pointes			
	Ventricular Tachycardia			


To change the Maestro default modeled cardiac parameters to one of the suggested arrhythmias:

- a. From the Cardiac Rhythm Options, scroll through the list to find the desired arrhythmia.
- b. Select the Arrhythmia.
- c. Click Accept.

Step 2: Set up the Defibrillator

To setup the Defibrillator, select probes and enabling the power as follows:

1. From the left panel of the SCE Run screen, select the **Equipment** icon.

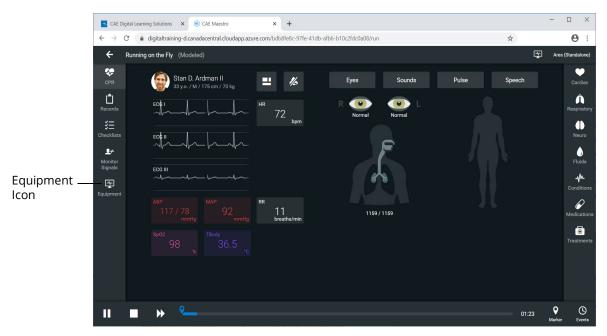

SCE Run Screen







The Equipment parameters appear.

Select	🖲 Ventilator 🖉		X
Equipment Parameters	General		Probes
drop-down	Power		Breathing Circuit
	Ventilation Type	Invasive 🔻	Pulse Oximeter SpO2 (Plethysmograph)
	Ventilation Mode	PCV -	CO2 Attachment (Capnograph)
	Ventilation Status	Standby 🝷	Edi Catheter
	Pressure		NAVA
	Pi (cmH2O)		NAVA Mode
		20 +	NAVA Level (cmH2O/uV)
			— 0.0 +
			0.0 15.0
	n 80	0 +	Edi Trigger (uV)

Equipment Parameters

Click the **Select Equipment Parameters** drop-down indicator to the right of the equipment type at the top of the list.

 Ventilator 	ø 🗸
Defibrillator	×
(iii) AED	1
Transport Ventilator	1
Anesthesia Machine	1

Select Equipment Parameters Drop-Down

2. Select **Defibrillator** from the list.

The Defibrillator parameters appear for editing.

Defibrillator	<i>*</i> ~		🔺 = Availabl		×
General		Probes			
Power	•	ECG Cables		None	
Learner Layout Selection	Generic Defibrillator 🔻	ECG Cable Placement		Correct	
Energy Select (J)		Therapy Pads Reversal			
2 360	- 200 +	🔺 Therapy Pads			
Apply Energy Change		Pulse Oximeter SpO2 (Plethysn	nograph)		
		NIBP Cuff			
Pacing		ABP Catheter			
Pacer		CO2 Sample Line			
Pacing Rate (bpm)	- 60 +	Temperature Probe			•

Defibrillator Equipment Parameters

3. From the Probes options, select the drop-down list for **ECG Cables**.


•

ECG Cables Drop-Down List

- 4. Select **ECG 12 Lead** from the drop-down list.
- 5. From the Probes options, select the drop-down list for **ECG Cable Placement**.
- 6. Select **Correct** from the drop-down list.
- 7. Ensure that the Therapy Pads Reversal is disabled.
- 8. Enable the following probes:
 - Therapy Pads
 - ^o Pulse Oximeter SPO₂ (Plethysmograph)
 - ° NIBP Cuff
 - ° ABP Catheter
 - ° CO₂ Sample Line
 - ° Temperature Probe



The Probes options will appear as shown in the following image.

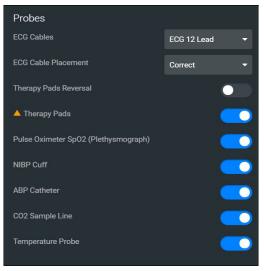


Defibrillator Probes Enabled

9. From the General parameters, verify that the **Power** indicator is **enabled**.

General					
Power					
Learner Layout Selection			Gen	ieric Defibr	illator 🔻
Energy Select (J)					
2	360	-		200	+
Apply Energy Change					

General Equipment Parameters - Power Enabled

Note: Once the power is enabled to the Defibrillator, the simulation connection status will change at the top of the Equipment Parameters Panel. This same status icon will be displayed at the top of the defibrillator screen, when running the defibrillator.

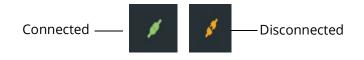


Simulation Connection Status

10. Click the **X** in the upper-right corner to close the Equipment parameters panel and return to the SCE Run screen.

Step 3: Launch the Defibrillator


To launch the Defibrillator:

1. From the top right of the SCE Run screen, select the Maestro Application drop-down menu.

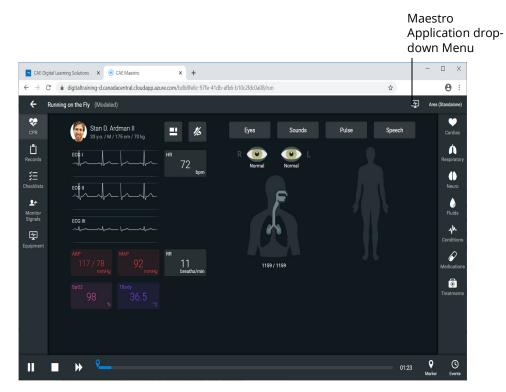

SCE Run Screen


The Maestro Application drop-down menu appears.

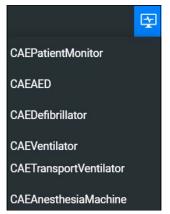

- Maestro Application Drop-Down Menu
- 2. From the drop-down menu, select **CAEDefibrillator**.



After the system has initialized, the CAE Defibrillator screen appears.

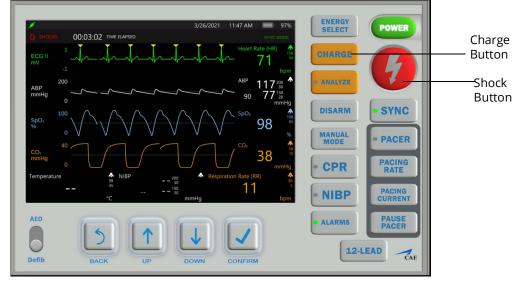

CAE Defibrillator


Note: Enable all probes as described in *Step 2: Set up the Defibrillator*. Having probes enabled will ensure appropriate patient vital signs are displayed and monitored by the equipment during the simulation.

See Appendix B - Defibrillator Controls and Display Options for more information.

Step 4: Set up the AED

To set up the AED:

- 1. Only one electrotherapy device may be used at a time. If the Defibrillator is running, close the Defibrillator by clicking on the **X** in the upper-right corner of the Defibrillator browser tab.
- 2. From the left panel of the SCE Run screen, select the **Equipment** icon.

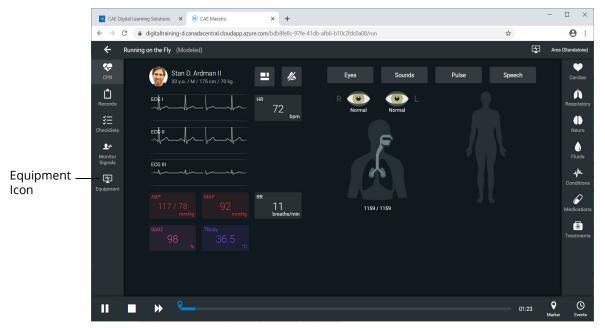

SCE Run Screen







The Equipment parameters appear.

Select	()) Ventilator	<u>/ v</u>	×
Equipment Parameters	General		Probes
drop-down	Power		Breathing Circuit
	Ventilation Type	Invasive 🔻	Pulse Oximeter SpO2 (Plethysmograph)
	Ventilation Mode	PCV -	CO2 Attachment (Capnograph)
	Ventilation Status	Standby 🔫	Edi Catheter
	Pressure		NAVA
	Pi (cmH2O)		NAVA Mode
	0 80	- 20 +	NAVA Level (cmH20/uV)
	Pi Backup (cmH2O)		0.0 15.0 - 0.0 +
	H		Edi Trigger (uV)
	n 80		

Equipment Parameters

3. Click the **Select Equipment Parameters** drop-down indicator to the right of the equipment type at the top of the list.

(n) Ventilator	¢ 🗸
Defibrillator	1
AB AED	1
Transport Ventilator	1
Anesthesia Machine	1

Select Equipment Parameters Drop-Down

4. Select **AED** from the list.

The AED parameters appear for editing.

AED	× ×		×
General		Probes	
Power		Therapy Pads	
Learner Layout Selection	Generic Defibrillator 👻		
Energy Select (J)		Voice	
	- 200 +	Instructions	
2 360			
Apply Energy Change			
CPR			
Metronome			
Metronome Rate (bpm)			
100 120	- 100 +		
Apply Metronome Change			I

AED Equipment Parameters

5. From the Probes options, ensure that the Therapy Pads Reversal is disabled.

6. Enable the Therapy Pads.


7. From the Voice options, **Enable** the **Instructions**.

The Probes and Voice options appear as shown in the following image.

Probes	
Therapy Pads	
Voice	
Instructions	

AED Probes and Voice Options

8. From the General parameters, verify that the **Power** indicator is **enabled**.

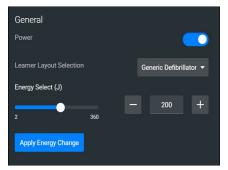

General Equipment Parameters - Power Enabled



Note: Once the power is enabled to the AED, the simulation connection status will change at the top of the Equipment Parameters Panel. This same status icon will be displayed at the top of the AED screen, when running the AED.




Simulation Connection Status

9. Click the **X** in the upper-right corner to close the Equipment parameters panel and return to the SCE Run screen.

Step 5: Launch the AED


To launch the AED:

1. From the upper-right corner of the SCE Run screen, select the Maestro Application dropdown menu.

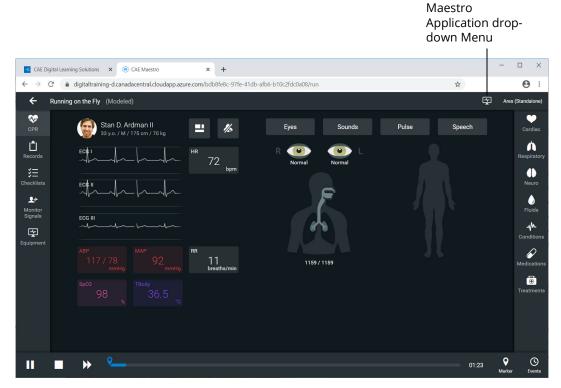

SCE Run Screen

The Maestro Application drop-down menu appears.

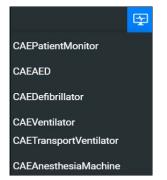


Maestro Application Drop-Down Menu

2. Select CAEAED.



After the system initializes, the CAE AED appears in a separate browser tab. When voice instructions are enabled, audible instructions will be heard, in addition to the textual instructions that appear on the screen.

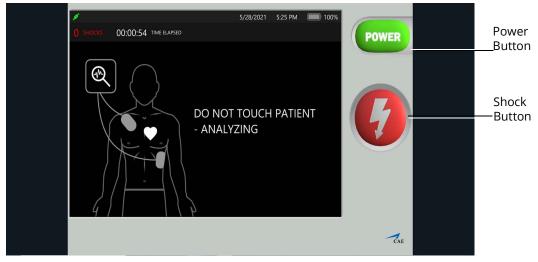

CAE AED

Note: Enable all probes as described in *Step 4: Set up the AED*. Having probes enabled will ensure appropriate patient vitals are displayed and monitored by the equipment during the simulation.

See Appendix C - AED Controls and Display Options for more information.

CAE Ventilator and CAE Transport Ventilator

Listed below are the steps to use the virtual ventilation equipment included with Maestro Evolve.

Note: Only one ventilation device may be used at a time. For example, if the Transport Ventilator is active, and the Ventilator or Anesthesia Machine is powered on, the Transport Ventilator will automatically power off.

Ventila	Ventilation Equipment Steps	
1	Set up the patient for ventilation	
2	Set up the Ventilator	
3	Launch the Ventilator	
4	Set up the Transport Ventilator	
5	Launch the Transport Ventilator	

Step 1: Set up the Patient

To perform a realistic and effective ventilation simulation, parameters are set to reflect a patient's condition that would require ventilation. This section provides information on modifying and activating the appropriate parameters.

- These settings should be set in CAE Maestro Evolve prior to powering on and using the ventilator.
- The same suggested patient parameters may be used for CAE Ventilator or CAE Transport Ventilator.

In the case that Maestro Evolve is running but another virtual device is in the active window, it may be necessary to switch windows.

To switch windows:

1. If a virtual device window (e.g. the Patient Monitor) is open, minimize the device window.

OR

At the top of the browser window, select the **CAE Maestro** tab.




Browser Window Tabs

Tip: In Windows 10, hold down on the **ALT** key and press the **Tab** key to alternate between open windows.

Note: Select a modeled simulation for the Ventilator to function appropriately.



The SCE Run Screen appears.

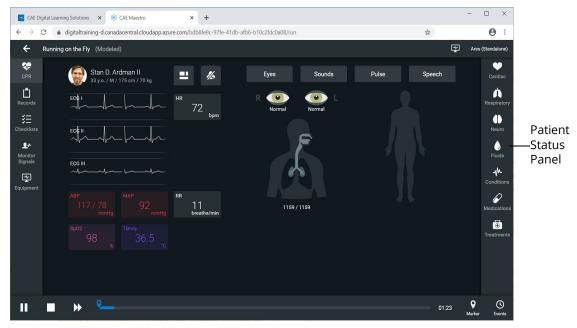


SCE Run Screen

2. From the Patient Status panel on right side of the SCE Run screen, click the **Respiratory Parameters** icon.



The Respiratory Parameters panel appears.

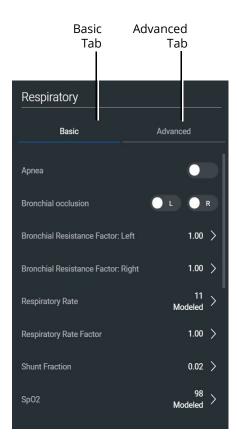

Respiratory Parameters Panel




3. Set the following Respiratory Parameters in Maestro Evolve:

Respiratory P	Parameters	Setting
Basic	Bronchial Resistance Factors: Left	5.0
	Bronchial Resistance Factors: Right	5.0
	Apnea	 For testing <i>control</i> mode, enable Apnea. For testing <i>support</i> mode,
		disable Apnea.
Advanced	Chest Wall Compliance Factor	.50
	Lung Compliance Factor: Left	.50
	Lung Compliance Factor: Right	.50

Step 2: Set up the Ventilator

Prior to running a simulation using the Ventilator, there are setup steps involved; selecting probes and enabling the power as follows:

1. From the left panel of the SCE Run Screen, select the **Equipment** icon.

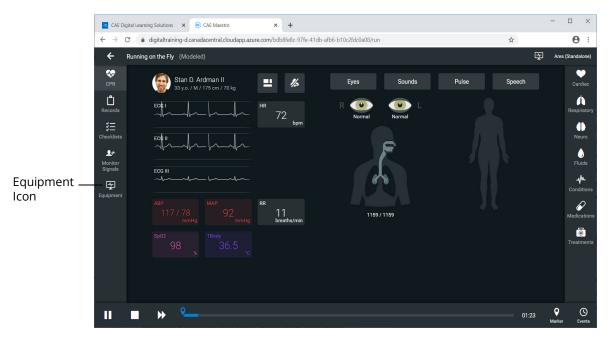

SCE Run Screen







The Equipment parameters appear.

Select	🖲 Ventilator	<u>/ v</u>	×
Equipment Parameters	General		Probes
drop-down	Power		Breathing Circuit
	Ventilation Type	Invasive 🔻	Pulse Oximeter Sp02 (Plethysmograph)
	Ventilation Mode	PCV 👻	CO2 Attachment (Capnograph)
	Ventilation Status	Standby 👻	Edi Catheter
	Pressure		NAVA
	Pi (cmH2O)		NAVA Mode
		- 20 +	NAVA Level (cmH20/uV)
	Pi Backup (cmH2O)		0.0 +
			Edi Trigger (uV)
	n RN		

Equipment Parameters

2. Click the **Select Equipment Parameters** drop-down indicator to the right of the equipment type at the top of the list.

Wentilator	1 v
Defibrillator	1
AED	1
Transport Ventilator	1
(Anesthesia Machine	1

Select Equipment Parameters Drop-Down

3. Scroll down the list and select **Ventilator**.


CAE Virtual Equipment

- 4. From the Probes options, **enable** the following **probes**:
 - ° Breathing Circuit
 - ^o Pulse Oximeter SPO₂
 - ° CO₂ Attachment




Ventilator Probes Enabled

5. From the General parameters, verify that the **Power** indicator is **enabled**.

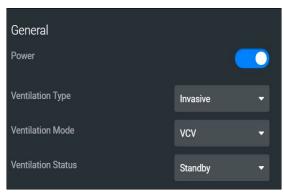

General Equipment Parameters - Power Enabled

Note: Once the power is enabled to the ventilator, the simulation connection status will change at the top of the Equipment parameters panel. This same status icon will be displayed at the top of the ventilator screen, when running the ventilator.

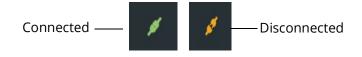

Simulation Connection Status


6. Click the **X** in the upper-right corner to close the Equipment parameters panel and return to the SCE Run screen.



Step 3: Launch the Ventilator


To launch the Ventilator:

1. From the upper-right corner of the SCE Run screen, select the Maestro Application dropdown menu.

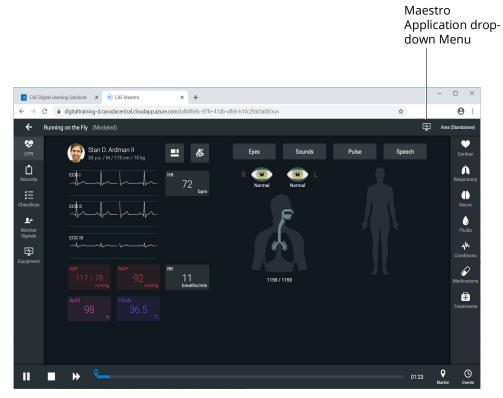

SCE Run Screen


The Maestro Application drop-down menu appears.

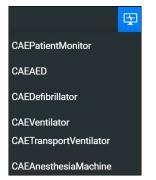

Maestro Application Drop-Down Menu

2. From the drop-down menu, select **CAEVentilator**.

The System Initializing Screen will appear in a separate browser tab.

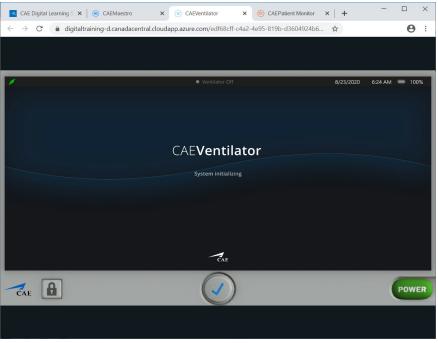

System Initializing Screen

3. Arrange the open browser tabs for the most effective viewing:

When using dual monitors:

- a. Position the mouse pointer on the **CAEVentilator** browser tab.
- b. Drag the Ventilator window to the second monitor display, leaving the Maestro Evolve window in the first display.

When using a single monitor:

- a. Restore or resize the Maestro Evolve screen so that some of the Windows desktop is visible.
- b. Position the mouse pointer on the **CAEVentilator** tab in the browser window.
- c. Drag the Ventilator window outside of the active browser window. The Ventilator will appear in a separate window.



After the system has initialized, the Ventilator Setup Standby screen appears.

Ventilator Setup Standby Screen


4. Change the parameter values as needed. After positioning the cursor in one of the parameter value fields, and selecting the value for editing, additional control buttons will appear at the bottom of the screen.



Value Select Buttons

- 5. Use the **plus (+)** and **minus (-)** buttons at the bottom center of the screen to adjust the value lower or higher as needed.
- 6. Click the **Check mark symbol** in the center to finalize the change in value.
- 7. When finished making all changes on the Setup screen, click the **Start Ventilation** button.

The Ventilator screen appears.

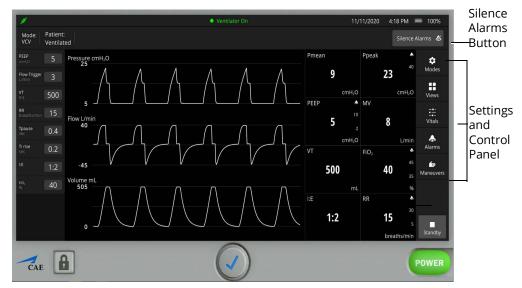

Ventilator Screen - Ventilator On

To silence the alarms, from the upper-right corner of the ventilator screen, click the **Silence Alarms** button.



Silence Alarms Button

There are additional settings that represent a typical ventilator used in a clinical setting. Status and parameters are displayed on the Ventilator screen based on options selected from the Settings and Controls panel at the right of the screen. Options to select from are:

- Modes
- Views
- Vitals
- Alarms
- Maneuvers

Refer to Appendix D- Ventilator Controls and Display Options for further details.

Note:. Enable all probes as described in *Step 2: Set up the Ventilator*. Having probes enabled will ensure appropriate patient vital signs are displayed and monitored by the equipment during the simulation.

The ventilator screen may be minimized and continue to run during the simulation. This is especially helpful if using a single monitor, allowing for a full view of the Maestro Evolve SCE Run screen and the patient status display.





Step 4: Set up the Transport Ventilator

Prior to running a simulation using the Transport Ventilator, there are setup steps involved; selecting probes and enabling the power as follows:

- CAE Digital Learning Solutions 🗙 🛞 CAE Maestro × + ← → C 🌘 digitaltraining-d.canadacentral.cloudapp.azure.com/bdb8fe8c-97fe-41db-afb6-b10c2fdc0a08/rur ☆ **e** : Running on the Fly (Modeled) 4 Ŧ • Y Stan D. Ardman II Sounds Pulse Eyes Ĉ A Neuro EC¢ ٥ <u>.</u> ECG III Equipment _ Ţ lcon Equipmer P 11 breaths/min • (5) 9 • 01:23 SCE Run Screen
- 1. From the left panel of the SCE Run Screen, select the **Equipment** icon.



CAE Virtual Equipment

The Equipment parameters appear.

Select	() Ventilator	/ <u>v</u>	×
Equipment Parameters	General		Probes
drop-down	Power		Breathing Circuit
	Ventilation Type	Invasive 🔻	Pulse Oximeter SpO2 (Plethysmograph)
	Ventilation Mode	PCV -	CO2 Attachment (Capnograph)
	Ventilation Status	Standby -	Edi Catheter
	Pressure		NAVA
	Pi (cmH20)		NAVA Mode
		- 20 +	NAVA Level (cmH20/uV)
	0 80		— 0.0 +
			0.0 15.0
			Edi Trigger (uV)

Equipment Parameters

2. Click the **Select Equipment Parameters** drop-down indicator to the right of the equipment type at the top of the list.

()) Ventilator	\$ v
Defibrillator	1
AED AED	1
Transport Ventilator	(X)
(Anesthesia Machine	1

Select Equipment Parameters Drop-Down

3. Scroll down the list and select **Transport Ventilator**.



The Transport Ventilator parameters appear for editing.

Transport Ventilator	<i>ø</i> •	×
General		Probes
Power		Breathing Circuit
Ventilation Type	Invasive 👻	Pulse Oximeter SpO2 (Plethysmograph)
Ventilation Mode	VCV -	CO2 Attachment (Capnograph)
Ventilation Status	Standby -	
		Complications
		Breathing Circuit Disconnect
Pressure		
		Leak (%)
		● 0 +
		0 100
		Apply Complications Change


Transport Ventilator Equipment Parameters

- 4. From the Probes options, **enable** the following **probes**:
 - ° Breathing Circuit
 - Pulse Oximeter SPO₂
 - ° CO₂ Attachment




Transport Ventilator Probes Enabled

5. From the General parameters, verify that the **Power** indicator is **enabled**.

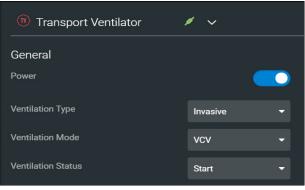

General Equipment Parameters - Power Enabled

Note: Once the power is enabled to the transport ventilator, the simulation connection status will change at the top of the Equipment parameters panel. This same status icon will be displayed at the top of the transport ventilator screen, when running the ventilator.

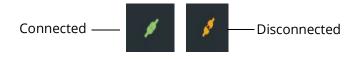




6. Click the **X** in the upper-right corner to close the Equipment parameters panel and return to the SCE Run screen.



Step 5: Launch the Transport Ventilator

To launch the Transport Ventilator:

1. From the upper-right corner of the SCE Run screen, select the Maestro Application dropdown menu.

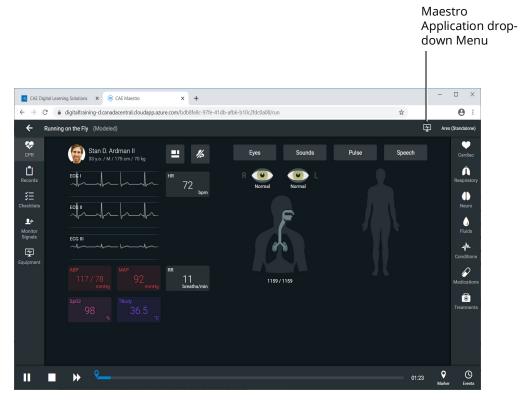

SCE Run Screen

The Maestro Application drop-down menu appears.



Maestro Application Drop-Down Menu

2. From the drop-down menu, select **CAETransportVentilator**.

The System Initializing Screen will appear in a separate browser tab.

CAE Digital Learning Solutions X 🛞 CAEMaestro X 🛞 CAETiansport/ientilator. (Beta) X +		- 0	×
E 3 C 0 û digitatraining-ganadezentral.doudapp.azure.com/dos/6a/5-4ed2-4507-a174-33eb52a050d/transport.vent	¢	• * 0	9 :
The second se			
🖉 🛛 🔍 🖉 Ventilator Off 11/11/2020 4:34 PM			
CAETransportVentilator			
System initializing			
CAE			

System Initializing Screen

3. Arrange the open browser tabs for the most effective viewing:

When using dual monitors:

- a. Position the mouse pointer on the **CAETransport Ventilator** browser tab.
- b. Drag the Transport Ventilator window to the second monitor display, leaving the Maestro Evolve window in the first display.

When using a single monitor:

- a. Restore or resize the Maestro Evolve screen so that some of the Windows desktop is visible.
- b. Position the mouse pointer on the **CAETransport Ventilator** tab in the browser window.
- c. Drag the Transport Ventilator window outside of the active browser window. The Ventilator will appear in a separate window.



After the system has initialized, the Transport Ventilator Setup Standby screen appears.



Transport Ventilator Setup Standby Screen

4. Change the parameter values as needed. After positioning the cursor in one of the parameter value fields, and selecting the value for editing, additional control buttons will appear at the bottom of the screen.



Value Select Buttons

- 5. Use the **plus (+)** and **minus (-)** buttons at the bottom center of the screen to adjust the value lower or higher as needed.
- 6. Click the **Check mark symbol** in the center to finalize the change in value.
- 7. When finished making all changes on the Setup screen, click the **Start Ventilation** button.

The Ventilator screen will appear.



Transport Ventilator Screen - Ventilator On

To silence the alarms, from the upper-right corner of the transport ventilator screen, click the **Silence Alarms** button.



Silence Alarms Button

There are additional settings that represent a typical ventilator used in a clinical setting. Status and parameters are displayed on the Ventilator screen based on options selected from the Settings and Controls panel at the right of the screen. Options to select from are:

- Modes
- Views
- Vitals
- Alarms

Refer to Appendix E- Transport Ventilator Controls and Display Options for further details.

Note: Enable all probes as described in *Step 4: Set up the Transport Ventilator*. Having probes enabled will ensure appropriate patient vital signs are displayed and monitored by the equipment during the simulation.

The transport ventilator screen may be minimized and continue to run during the simulation. This is especially helpful if using a single monitor, allowing for a full view of the Maestro Evolve SCE Run screen and the patient status display.



CAE Anesthesia Machine

Listed below are the steps to begin using CAEAnesthesia Machine.

Note: Only one ventilation device may be used at a time. For example, if the Transport Ventilator is active, and the Ventilator or Anesthesia Machine is powered on, the Transport Ventilator will automatically power off. However, both an electrotherapy device and a ventilation device may be used in parallel.

Unlike the Ventilator and Transport Ventilator, it is not necessary to establish specific patient parameters in the setup process.


Note: Select a modeled simulation for the Anesthesia Machine to function appropriately.

Anesth	esia Machine Steps
1	Set up the Anesthesia Machine
2	Launch the Anesthesia Machine

Step 1: Set up the Anesthesia Machine

Prior to running the Anesthesia Machine, there are setup steps involved; selecting probes and setting gas parameters as follows:

1. From the left panel of the SCE Run Screen, select the **Equipment** icon.

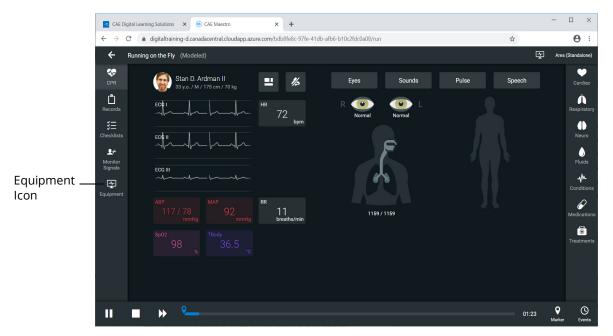

SCE Run Screen





The Equipment parameters appears.

Select Equipment	🛞 Ventilator	/ ~	×
Parameters drop-down	General		Probes
	Power		Breathing Circuit
	Ventilation Type	Invasive 🔻	Pulse Oximeter SpO2 (Plethysmograph)
	Ventilation Mode	PCV -	CO2 Attachment (Capnograph)
	Ventilation Status	Standby –	Edi Catheter
	Pressure		NAVA
	Pi (cmH2O)		NAVA Mode
	0 80	- 20 +	NAVA Level (cmH2O/uV)
	Pi Backup (cmH2O)		0.0 15.0 - 0.0 +
	n 80	- 0 +	Edi Trigger (uV)

Equipment Parameters

2. Click the **Select Equipment Parameters** drop-down indicator to the right of the equipment type at the top of the list.

(1) Ventilator	× ~
Defibrillator	1
AD AED	1
(1) Transport Ventilator	1
Anesthesia Machine	1

Select Equipment Parameters Drop-Down

3. Scroll down the list and select **Anesthesia Machine**.

The Anesthesia Machine parameters appear for editing.

Anesthesia Machine	¢ ~	× 1
General		Probes
Power		Breathing Circuit
Ventilation Type	Invasive 🔫	Pulse Oximeter SpO2 (Plethysmograph)
Ventilation Mode	PCV 👻	CO2 Attachment (Capnograph)
Ventilation Status	Standby 🔫	
Manual Ventilation	•	Complications
Auxiliary Common Gas Outlet (ACGO) Oper		Breathing Circuit Disconnect
		0.0 100.0 +
Pressure		
Pi (cmH2O)		Soda Lime Canister Consumed
0 125	- 20 +	Apply Complications Change

Anesthesia Machine Equipment Parameters

- 4. From the Probes options, **enable** the following **probes**:
 - ° Breathing Circuit
 - Pulse Oximeter SPO₂
 - ° CO₂ Attachment



Anesthesia Probes Enabled

5. Click the **X** to close the Equipment parameters panel and return to the SCE Run screen.






Step 2: Launch the Anesthesia Machine


To launch the Anesthesia Machine:

1. From the upper-right corner of the SCE Run screen, select the Maestro Application dropdown menu.

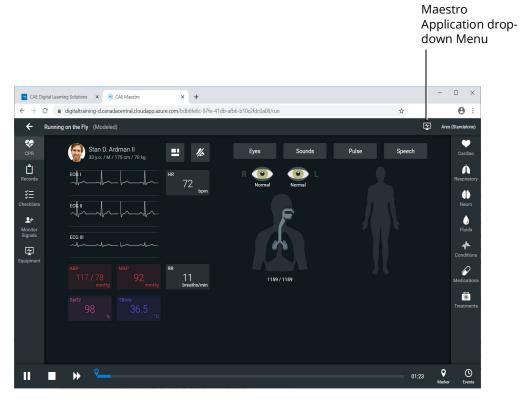

SCE Run Screen

The Maestro Application drop-down menu appears.

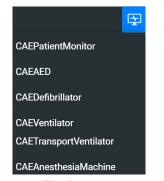

Maestro Application Drop-Down Menu

2. From the drop-down menu, select **CAEAnesthesiaMachine**.

The Anesthesia Machine Screen will appear in a separate browser tab.



Anesthesia Machine Screen - Machine Off

3. Arrange the open browser tabs for the most effective viewing:

When using dual monitors:

- a. Position the mouse pointer on the **CAEAnesthesia Machine** browser tab.
- b. Drag the Anesthesia Machine window to the second monitor display, leaving the Maestro Evolve window in the first display.

When using a single monitor:

- a. Restore or resize the Maestro Evolve screen so that some of the Windows desktop is visible.
- b. Position the mouse pointer on the **CAEAnesthesia Machine** tab in the browser window.
- c. Drag the Anesthesia Machine window outside of the active browser window. The Anesthesia Machine will appear in a separate window.



Power On the Anesthesia Machine

To turn on the Anesthesia Machine, click on the **Power** button in the lower-right corner of the Anesthesia Machine screen.



Power Button

The System Initializing Screen will appear in a separate browser tab.



System Initializing Screen

After the system has initialized, the Anesthesia Setup Standby screen appears.

1	Anesthesia Machine On	6/1/2021 6:53 PM 📼 100%
	Standby - Patient not Ventilated	
	Setup	
	Ventilation Type Ventilation Setup	
	Invasive PEEP 5 Ti rise 0.2	
	Non-invasive Pi Ventilation Mode Control 20	
- 36 - 12 - 8 - 12 - 32 - 9 - 6 - 9 - 28 - 6 - 4 - 6	VCV SIMV VC Row Trigger 3	
-40 -10 -10 -10 30 -20 -10 -10 31 -1 -4 -9 32 -1 -4 -6 33 -1 -1 -1 34 -1 -1 -1 34 -1 -1 -1 34 -1 -1 -1 35 -1.5 -6.5 -6.1	PCV RR CPAP+PS CPAP+PS	Start
	CPAP+PS Start Ventilation	Ventilation
0 L ^{/min} 0.2 L ^{/min} 0 L ^{/min} 0 L ^{/min} AlR		Button
O ₂ N ₂ O AIR		Tap to view soda time canister
CAE		Power Button

Anesthesia Setup Standby Screen

Setting Ventilation Parameter Values

To change the ventilation parameter values:

1. Select the value for editing by positioning the cursor in one of the parameter value fields.

Additional control buttons appear at the bottom of the screen.



Value Select Buttons

- 2. Use the + and buttons at the bottom center of the screen to adjust the value lower or higher as needed.
- 3. Click the **Check mark symbol** in the center to finalize the change in value.



Setting Gas Flow Volume


To set gas flow volume:



1. From the Gas controls on the lower-left of the Anesthesia screen, click on the O_2 control.

The O_2 controls are activated.

Click on the **plus (+)** control until the O₂ flow gauge displays 5 L per minute. Use the minus (-) control if necessary to decrease the value.

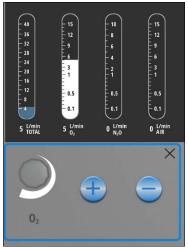

O₂ Gas Set at 5.0

- 3. Click on the \mathbf{X} to close the O₂ control.
- 4. Click on the N_2O control.
- 5. Adjust the N_2O flow to 1 L per minute.
- 6. Click on the **X** to close the N_2O control.
- 7. Click on the Air control.
- 8. Adjust the Air flow gauge to 10 L per minute.
- 9. Click on the **X** to close the Air flow control.

Gauges will reflect volumes set.



Gases Flow Volume Set

Setting Anesthestic Agent Volumes

To set anesthestic agent volume percentages:

1. From the lower-left of the Anesthesia screen, select one of the anesthestic agents. In this example, **Isoflurane** is selected.



Anesthestic Agents



The anesthestic agent controls will appear.



Isoflurane Controls

- 2. Click on the **plus (+)** to increase the amount of the anesthestic agent. In this example, **Isoflurane** is increased to 2 (2.0%). Depending upon the agent selected, the ranges will vary.
- 3. Click on the **X** to close the Anesthestic Agents Controls window.

The Anesthestic Agents that were not selected will be shown locked, preventing the administration of additional agents. The agent chosen will display the value previously set using the controls.

2	0 📾	0 🖬
	1 mm	ALL IX
Isoflurane	Sevoflurane	Desflurane

Anesthestic Agents - Set and Locked

O2 Flush and Auxillary Common Gas Outlet



O₂ Flush

Clicking on the O_2 + Flush control pushes 100% O_2 into the breathing circuit.

CAE Virtual Equipment

To open and close the Auxillary Common Gas Outlet (ACGO):

1. Click on the **ACGO Valve** control.

The valve control is activated.



ACGO Valve Control Closed

2. Clilck on the **Open** or **Close** option.


The red marker indicates the setting for the valve.

Note: If the valve is left open, the gases will not go to the main breathing circuit causing erratic patient vitals on the Maestro Run Screen display.

3. Click on the **X** to close the ACGO valve control.

Manual Ventilation Controls

On the lower-right side of the Anesthesia screen are the controls for the APL valve, the Manual Switch and the Rebreather Bag.

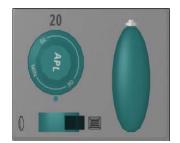

APL Valve, Manual Swtich and Rebreather Bag Controls



Adjusting the APL Valve

To adjust the Adjustable Pressure Limiting (APL) valve:

1. Click on the **APL Valve** control.

The valve is activated.



APL Valve Control


- 2. Clilck on the **plus (+) or minus (-)** option, to increase or decrease amount of pressure. The value will be displayed in the control.
- 3. Click on the **X** to close the APL control.

Controlling Manual Ventilation

To control manual ventilation:

1. Click on the **Manual Switch** button on the left of the control panel.

The switch control is activated.

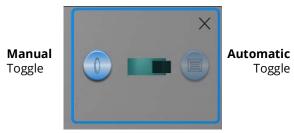

Manual Switch Control

- 2. Clilck on the Manual or Automatic option.
- 3. Click on the X to close the Manual Switch control panel.


While the ventilation is set to manual, air may be administered using the rebreather bag.

CAE Virtual Equipment

To use the rebreather bag:

1. Click on the **Rebreather Bag** control.

The control is activated.

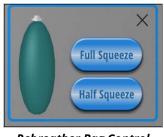

Rebreather Bag Control

- 2. Select Full Squeeze or Half Squeeze.
- 3. Click on the X to close the Rebreather Bag control.

Soda Lime Canister

The soda lime canister is a visual indicator of how much CO2 is absorbed through the breathing circuit. If the canister is completely consumed, it will turn to purple.

Soda Lime Canister


To view the soda lime canister, click the canister icon in the lower-right corner of the Anesthesia screen.

Start Ventilation

When finished making all changes on the Setup screen, click the **Start Ventilation** button.



The Anesthesia Machine screen will appear.

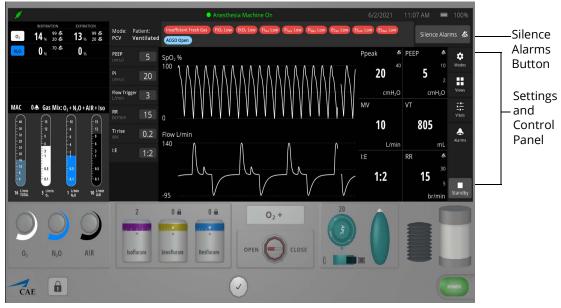

Anesthesia Machine Screen - Power On

To silence the alarms, from the upper-right corner of the ventilator screen, click the **Silence Alarms** button.



Silence Alarms Button

There are additional settings that represent a typical anesthesia machine used in a clinical setting. Status and parameters are displayed on the anesthesia screen based on options selected from the Settings and Controls panel at the right of the screen. Options to select from are:

- Modes
- Views
- Vitals
- Alarms


Refer to Appendix F - Anesthesia Machine Controls and Display Options for further details.

Note: Enable all probes as described in *Step 5: Set up the Anesthesia Machine*. Having probes enabled will ensure appropriate patient vital signs are displayed and monitored by the equipment during the simulation.

CAE Virtual Equipment

The Anesthesia Machine screen may be minimized and continue to run during the simulation. This is especially helpful if using a single monitor, allowing for a full view of the Maestro Evolve SCE Run screen and the patient status display.

Once the power is enabled to the anesthesia machine, the simulation connection status will change at the top of the Equipment parameters panel. This same status icon will be displayed at the top of the Anesthesia screen, when running the Anesthesia Machine.

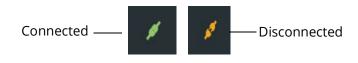

Simulation Connection Status



CAEEMBODY THE VIRTUAL SIMULATOR

Embody, CAE's virtual simulator, is designed with advanced physiology, interfacing with Maestro and all of the virtual equipment features. When parameters are modified in Maestro Evolve, the virtual simulator reflects the adjusted conditions and vital signs.

Using Embody

Embody may be used in one of two ways:

- As the Selected Simulator Selecting Embody as the preferred simulator for the simulation, prior to starting the simulation.
- During the Simulation in Progress Embody may also be used to virtually represent the simulation in progress, although a different patient simulator (e.g. Ares, Juno, etc.) was selected prior to starting the simulation.

Note: Not all Embody functionality will be available if another simulator was selected prior to starting the simulation.

Select the Simulator

Selecting Embody as the simulator for the simulation will provide these additional features:

- Additional patient parameters; skin and gastro.
- Ability to select from multiple male and female patient profiles to represent the virtual simulator when creating and editing SCEs.

To select Embody as the preferred simulator for the simulation:

1. After logging into Maestro, from the Select a Simulator screen, click the drop-down list and select **Embody**.



Select Simulator Drop-Down List

The Maestro Home Screen appears.

2. Select Run on the fly (Modeled), Run on the fly (Manual), or Run an SCE.

The SCE Run screen appears, featuring the additional Embody patient parameters.



SCE Run Screen - Embody



Launch the Virtual Simulator

To launch the virtual simulator:

1. Keep the Maestro Evolve window open. At the top of the browser window, select the **CAE Digital Learning Solutions** tab.

CAE Digital Learning Solutions Tab ———	CAE Digital Learning Solutions 🗙 🙉 CAE Maestro 🗙 +
	\leftarrow \rightarrow C ($$ digital training-d.canadacentral.cloud app.azure.com/b763af0a-060d-4717-854f-f1cf
	Running on the Fly (Modeled)
	CPR 🔂 Stan D. Ardman II 🛄 🌿 Eyes
	Browser Window Tabs

The CAE Learning Digital Solutions page appears.

2. From the Maestro Evolve Module Menu on the CAE Digital Learning Solutions page, select **Share**.



Maestro Evolve Module Menu - Open, Share, and Stop

The Share Link to Maestro Evolve window appears.

Access Code	
Q0QZ	co COPY CODE
Patient Monitor	
https://digitaltraining-q.canadace	co COPY LINK
Defibrillator	
https://digitaltraining-q.canadace	co COPY LINK
AED	
https://digitaltraining-q.canadace	co COPY LINK
Ventilator	
https://digitaltraining-q.canadace	co COPY LINK
Embody	
https://cae-hc-cace-web-q-embo	co COPY LINK
Transport Ventilator	
https://digitaltraining-q.canadace	co COPY LINK
Anesthesia Machine	
https://digitaltraining-q.canadace	co COPY LINK
Share the following links with your s	tudents so they can view patient status
Share the following links with your s	adents so they can view patient status

Share Link to Maestro Evolve

- 3. Find **Embody i**n the list of options.
- 4. Click the **COPY LINK** button associated with Embody.
- 5. Click **DONE**.
- 6. At the top of the browser window, select the + to open a new window.

	CAE Digital Learning Solutions X 🛞 CAE Maestro X 🕇	Add New Window
Address Bar	← → C	
	← Running on the Fly (Modeled)	
	CPR 🔂 Stan D. Ardman II 💻 🏂 Eyes	
	Browser Window Tabs	

7. In the blank address bar of the new window, paste the copied link from Maestro Evolve and press Enter.

Note: This can be done by positioning the cursor in the address bar and using the keyboard shortcut **CTRL+V**.





The CAEEmbody virtual simulator appears in a separate browser tab.

CAE Embody

Note: If the chosen simulator was a simulator other than Embody, the same steps would be followed to launch the virtual simulator in a separate browser window.

Selecting the Patient

If Embody is chosen as the preferred simulator, multiple patient profiles are available when creating and editing SCEs.

When creating or editing the SCE:

1. From the SCE Navigation tabs on the left side of the window, select **Patient**.

	SCE Editor	Asthma - Chronic (I	Manual)		Embody (Standalone)
i	Details	Edit each se	ection or import patient settings from another SCE.		Import from SCE
£	Patient	Profile 🖍		Baseline 🖌	
≣	Scenario		A.Day Star	Modified Parameters	
絙	Checklists		Age 26 years Weight 70 kg	MONITOR SiGNALS Arterial Catheter position	None
Û	Patient Records	U	Height 160 cm Eyes Blue	Blood Temperature Probe	Off
=	Runtime Configuration		Gender Female	Body Temperature Probe	Off
1 *	Monitor Signals	Patient Not	tes 🧭	Capnograph	Off
0	Preparation			Central Venous Catheter position	None
				ECG Leads	Off
				Intra-Cranial Catheter	Off
				NIBP Cuff Probe	Off
	Run SCE			På Catheter position	None

SCE Editor - Editing a Patient Profile

2. Select the **Profile** edit icon.



The Select a Patient Profile Preset window appears. The gender selected at the beginning of the simulation will determine the initial profile options available (male or female). In the following example, Male was chosen as the gender initially at the beginning of the simulation. In the event a female gender profile is preferred, the gender may later be changed after selecting a profile.



Select a Patient Profile Window

- 3. Identify the preferred patient profile.
- 4. Click on the associated **Select** button.

The selected patient profile appears.

Patient Profile ×	
Patient Name	
Ohi T. Hertz 🛛 😣	
Change Profile	
Eye Color	
Blue 👻	
Gender Female O Male	
Age	
- 56 +	
Weight (kg)	
- 118 +	
Cancel Save	
Changes to the patients height and weight will not be reflected	ed

Patient Profile

Use the **drop-down list** or **plus (+)** and **minus (-)** buttons to make patient parameters changes to: eye color, age, weight, or height.

To change the gender:

- 1. Select the preferred gender.
- 2. Click on Change Profile.
- 3. The Select a Patient Profile window appears and a preferred selection may be made.



The selected patient profile is represented as the virtual patient when Embody is launched from the browser window.



CAE Embody - Selected Patient

Note: The patient represented in the Embody window will reflect the eye colors, age, weight, and height that were original to the profile. Changes made to parameters will not be morphed to the image.

More information is provided regarding creating and editing SCEs in the previous chapters of this guide:

- Managing SCEs
- Using the SCE Editor

Embody Features


Features are available for the learner to assess, monitor, and treat the virtual patient. The following are options are available from the panel at the left side of the Embody screen:

- Assessments
- Monitoring
- Equipment
- Treatments

Assessments

Selecting Assessments provides the following options:

- Pulse
- Skin Temperature
- Pupils
- Heart
- Lungs
- Abdomen

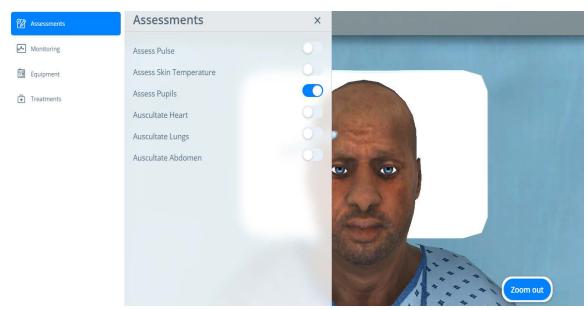

Assessment Options

In the example shown, the Assess Pupils option is toggled on. The patient view is modified so the pupils are more visible and a penlight is provided for use during the assessment. Click on the **X** to close the Assessments options panel, providing a more complete view of the patient.





To use the assessment features:

1. Identify the desired option to assess and toggle on.

Depending upon which option is selected, detection points (e.g. pulse) or tools (e.g. penlight or stethoscope) will be displayed on the virtual patient.

2. Click on one of the detection points or move the tool to the area chosen for assessment.

After selecting the assessment point, a corresponding widget will appear displaying data from the chosen assessment.



Assess Pulse Widget

Audible sounds can be heard by the learner when the Embody link to the simulation has been shared by the Instructor. Sounds are not audible by the learners when the Instructor is sharing the simulation via the Instructor's machine.


See the *Share Virtual Equipment* section of this guide for more information on how to share components of the simulation with learners.

Click on the **X** to close the widget.

Monitoring

Selecting Monitoring provides the following options:

- 5-Lead ECG
- Pulse Oximeter
- Blood Pressure Cuff

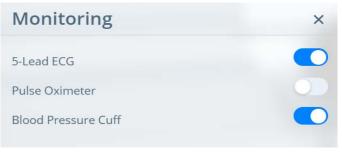

Monitoring Options

To use the monitoring features:

Identify the desired option to monitor and toggle on.

Depending upon which option is selected, equipment (e.g. pulse oximeter) will be displayed on the virtual patient. The corresponding monitoring option in the Maestro window will be modified to synchronize with the changes made in Embody.

Equipment

Equipment options include the virtual equipment applications found in Maestro and bed calibration options.

Equipment	×
Patient Monitor	🖸 Launch
Ventilator	🖸 Launch
Defibrillator	🖸 Launch
Bed	Please select 🔻

Equipment Options



Virtual Equipment

The following types of virtual equipment may be launched from the Embody virtual patient window:

- Patient Monitor
- Ventilator
- Defibrillator

Launching the equipment from Embody allows the learner to use the equipment in the current simulation, without having the links provided from the instructor, and without having access to the Maestro interface.

To launch the virtual equipment, click on the associated **Launch** button and the equipment chosen will appear in a separate browser window.

For more information on the devices, see the *Virtual Equipment* section of this User Guide. There are also Appendices specific to each device detailing parameters and controls.

Bed Calibration

To adjust the patient's bed calibration:

1. From the Equipment options, click on the drop down list to the right of **Bed**.



The calibration options appear.

Bed Calibration Options

2. Select one of the bed calibration options.


Embody will display the patient using the new bed calibration selected.

Treatments

Selecting Treatments provides the following options:

- Oxygen Devices
- Noninvasive or Invasive Ventilation

Selecting invasive or noninvasive will determine the enabled respiratory treatments.

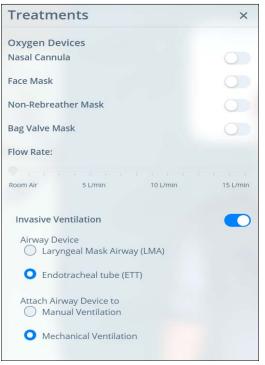

Treatment Options

To select and apply treatments:

- 1. If desired, enable an **Oxygen Device**.
 - a. Adjust the **Flow Rate** using the slider control.
- 2. Using the toggle switch, toggle off for **Noninvasive** or toggle on for **Invasive** Ventilation.
 - a. Select the desired option for **Airway Device**.
 - b. Select the desired option for Attach Airway Device to.
- 3. Click on the **X** to close the Treatment window.



After selecting and enabling treatment, the virtual patient will appear with the selected device. In the following example the patient is displayed with the Mechanical Ventilator.



Patient with Mechanical Ventilator

Change Patient View

The view of the patient may be adjusted by selecting any of the buttons in the right-center of the Embody screen.



Patient View Buttons


When assessment options are chosen which automatically provide close up views of the patient, the standard patient view buttons are replaced by a **Zoom Out** option.



LEARNING MODULES

Learning Modules are comprised of multiple SCEs. Some Learning Modules are predefined and included in Maestro Evolve. Users can also import or delete existing Learning Modules, or create new Learning Modules in Maestro.

From the SCE Manager screen, select **CAE Learning Modules** and the Learning Modules Screen appears.

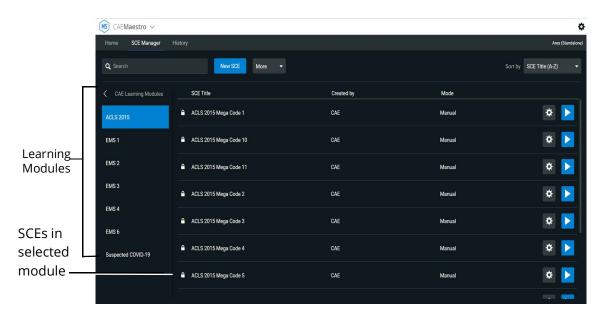


Learning Modules Screen

Import a Learning Module

To import a learning module in Maestro:

- 1. From the SCE Manager screen, click the **More** button.
- 2. From the drop-down menu, select Import Module.

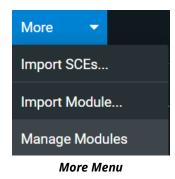


Learning Modules

- 3. Locate the correct learning module file on the external storage device or hard disk drive where the learning module is located. The file extension is *mlm*.
- 4. Select the file and click **Open.**

Delete a Learning Module

To delete a learning module from Maestro:

- 1. From the SCE Manager screen, select the **More** button.
- 2. From the drop-down menu, select **Manage Modules**.

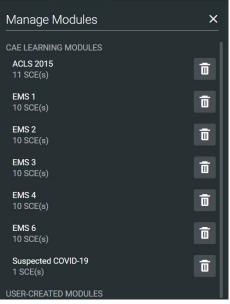

Manage Modules Screen

- 3. Click the **Trash** icon next to the module to be deleted.
- 4. When the message appears to confirm the selection, click **Delete All**.

All of the SCEs in the learning module will be deleted.



Create a Learning Module

To create a learning module:

1. From the SCE Manager screen, select **New SCE.**

The New SCE screen appears.

New SCE		×
SCE Title		
Select Module		
None		• +
Created by		
Administrator		8
Mode		
O Manual Mode	eled	
Patient Name		
Patient Gender		
Female Male		
	Cancel	

New SCE Screen

- 2. Type a title for the SCE in the **SCE Title** field.
- 3. To select a module to add the SCE to, click on the dropdown arrow, then select a module. Or, select the + icon to create a new module.

If creating a new module, type a name for it in the Enter New Module Name screen. Then, click **Add**.

Enter New Mo	dule Name		
	Cancel	Add	

New Module Name Screen

4. Complete the remaining fields or select options as required. Click **Create**.



ADMINISTRATIVE TOOLS

The Maestro software includes administrative tools to access and manage system settings, preferences, and library content. There are features that allow for managing and creating SCEs. Historical data associated with the simulation may be viewed or exported, serving as an effective debriefing tool.

Access Administrative tools menu via the **Gear** icon located on the upper-right corner of the Home page.



Administrative Tools Menu

From the Administrative tools menu select one of the following:

- System: Allows the user to set Preferences, access Performance Metrics and License Manager, Maintenance, and manage Content Library.
- **Select Simulator:** Allows the user to change the simulator selection from Ares, Juno or Embody. The simulator may also be chosen as the default simulator.
- Language: Change the Maestro language from the default English.
- **User Guide**: Provides a link to the online User Guide.
- **Support**: Provides a link to CAE support and contact information.
- About: Provides Maestro version.

System Administration

From the System Administration screen, users can access system information, set preferences, manage licenses, maintenance information, and the content library for Maestro.

To access the System Administration:

- 1. From the Home screen, click the **Gear** icon.
- 2. From the drop-down menu, select System.

CAEMaestro

CAEMaestro

CAEMaestro

CAEMaestro

CAEMaestro

Sinulator

Sandalone

Software

Version 22 CLOUD (19)

Version 22 CLOUD (19)

Disk Space 100 GB Available (25% free)

System Time Fri Sep 11 2020 17:05:50 -0400

The System Administration screen appears.

System Administration





From this screen, users can select:

- **About**: View simulator type, software version and system configuration.
- **Preferences**: Allows the user to change settings for different software features.

(i) About	Language
Preferences	English •
Hit Performance Metrics	Sweeper Speed
Orr License Manager	Default 3 Seconds 6 Seconds 15 Seconds 30 Seconds 60 Seconds
A Maintenance	Units Patient Profile
Content Library	Metric Imperial
	Pressure 0 mmHg kPa Temperature 0 °C °F Energy (electrical therapy) 0 Vs

System Preferences

• License Manager: Provides access license information. From here, users can activate or deactivate licenses.

÷			
(i) About	License Manager		
Preferences	Ares -		
부터 Performance Metrics	ACLS 2015	✓ Valid License	Deactivate
Maintenance	EMS 1	Valid License	Deactivate
Content Library	EMS 2	✓ Valid License	Deactivate
	EMS 3	Valid License	Deactivate
	EMS 4	Valid License	Deactivate
	EMS 6	✓ Valid License	Deactivate

License Manager

- Content Library: Manage patient files or custom speech audio files.
 - Patient Files See the section Using the SCE Editor for details on managing patient files.
 - ^o Custom Speech Audio Files The user can create new speech files or import existing files, search and sort the files.

Patient Files		 Custom Speech Files
Q Search	Sort by Title (A-Z)	
Create New Import		


Custom Speech Files Tab

Select a Simulator

Users can select a simulator and set it as the default when using Maestro.

To change the simulator type:

1. From the Home screen, click the **Gear** icon and select **Select Simulator**.

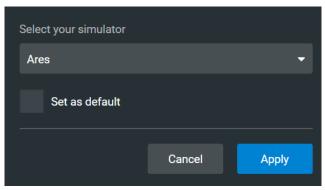

Select Simulator Screen

- 2. (Optional) Select Set as default.
- 3. Click Apply.






Set the Default System Language

To set Maestro's language:

- 1. From the Home screen, click the **Gear** icon.
- 2. Select Language.

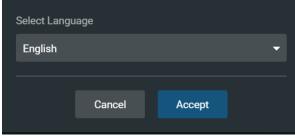

Select Language Screen

- 3. Click on the drop-down list to select the desired language.
- 4. Click Accept.

Note: Language can also be set from System > Preferences.

Additional Administrative Tools

Users can access the CAEMaestro User Guide, support information, or system information.

From the Home screen, click the **Gear** icon, and then select one of the following:

- User Guide: a link is provided where a copy of the user guide may be downloaded.
- **Support**: for CAE Support contact information.
- About: to access information about the Maestro software version.

SCE Management

The SCEs in Maestro are managed in the SCE Manager screen. For more information on managing and modifying settings of the SCEs using the SCE Manager, see the *Using Maestro Evolve* section of this user guide.

Session History

Session histories are retained so that the user may view or export various types of data associated with the simulation.

From the Home screen, select **History.**


The History screen appears displaying a log of simulation event data.

Home	SCE Manager Histe	ory				Ţ	Ares (Standalone)
Q Searc		In	nport		Sort by	Start Time	
Sta	irt Time	Duration	Simulator	Session Name	SCE Title		
20	19-12-30 11:45	Omin	Standalone	Ares session	Running on the Fly (Manual)		*
20	19-12-30 11:45	Omin	Standalone	Ares session	Running on the Fly (Manual)		\$
20	19-12-30 11:28	16min	Standalone	Ares session	Running on the Fly (Modeled)		\$
20	19-12-30 11:27	1min	Standalone	Ares session	Running on the Fly (Manual)		\$
20	19-12-27 13:45	Omin	Standalone	Ares session	Skill Assessment (Manual)		\$
20	19-12-27 13:43	Omin	Standalone	Ares session	Skill Assessment (Manual)		\$
20	19-12-24 17:15	Omin	Standalone	Ares session	Skill Assessment (Manual)		\$
Sel	ect All				Delet	e	

History Screen



Click the **Gear** icon for the History options menu.

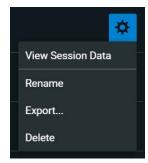


History Options Menu

To view session data:

- 1. Click the **Gear** icon for the desired simulation session.
- 2. Select View Session Data.

The Event Log appears. Click the **Physiological Data** or **CPR Data** tab to view associated data for the SCE.

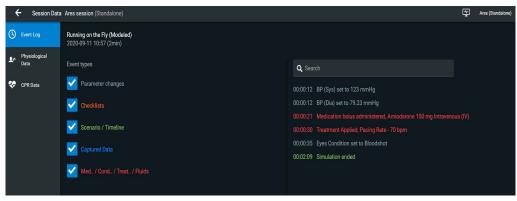



Users can also export data to a CSV or MSS file (session file in Maestro format) and store it on an external device. To export data from the History screen:

- 1. Select the **Gear** icon for the desired simulation session.
- 2. Select **Export**.

A CSV export will automatically generate Excel spreadsheet files in the user's Downloads folder. A separate worksheet will be created for the events, physiological data, and summary of the session.

THIS PAGE INTENTIONALLY LEFT BLANK



SHARING MAESTRO EVOLVE WITH LEARNERS

Maestro Evolve provides capabilities for sharing simulated clinical experiences with learners in several ways:

- A planned training event
 - ° Invite participants in advance
 - ° Conduct virtual training
- Share simulations of virtual medical equipment
 - ° Ad-hoc in a classroom setting or virtual training
 - ° During a planned virtual training event

Planned Training Event

For planned training events, Maestro Evolve provides the capability to invite the participants in advance, and to later conduct the actual training.

Invite Learners

To invite the learners to a training event:

1. From the CAE Digital Learning Solutions window, on the CAEMaestro Evolve Module Menu, click **Invite**.



Maestro Evolve Module Menu - Start and Invite

Note: In the event Maestro Evolve has previously been started, the following Open, Share, and Stop options will appear on the Maestro Evolve Module Menu.



Maestro Evolve Module Menu - Open, Share, and Stop

To return back to the Module Menu that has the Invite option, click **Stop**. Then click **Invite**.



The Invite Participants window appears.


sue.rnhealth@university.com, bob.emt@un	iversity.com, doc.md@university.com		
Subject			
Deep Dive in Ventilator Functions			
Message			
We will be learning ventilator functions next attending the virtual training. I am looking f Maestro Evolve learning experience. If you h			
Date	Time		
02/26/2021	05:00 PM (0		
	CANCEL SEN		

Invite Participants Window

- 2. Complete the information required.
 - Email addresses of participants
 - ° Subject
 - ° Date of Training Event
 - ° Time of Training Event
- 3. Click SEND.

The invited participants will receive an email containing the information specified in Maestro Evolve. Additionally, the email will contain links to the:

- Instructor's email address (or the individual who completed the invitation in Maestro Evolve) and
- Training event

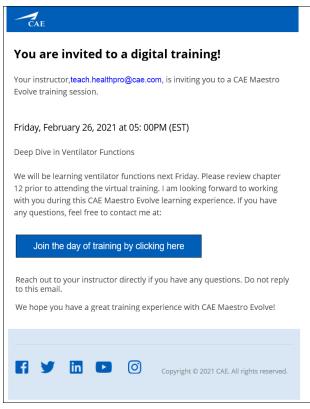

Invitation Email to Participant





Conduct Training Event

Participant

On the day of the training, the participant will need to:

- 1. Open the invitation email sent from the instructor.
- 2. Click the link to join the training.

The participant will be directed to a "lobby" page displaying a system generated access code.

	8GUN
S	tay on this page while the session is being launched. You will be redirected automatically when th ession starts. Meanwhile, identify yourself so that your instructor knows who is ready to start:
ſ	:mail* suern@university.com
I	'ull name (Optional)
ſ	Sue R. Egisterednurs

Training Session Lobby Page

- 3. The participant is directed to enter their:
 - Email address and
 - ° Name
- 4. Click the **I'm ready** box.

Instructor

On the day of the training event, when the Instructor logs into Maestro Evolve,

1. From the CAE Digital Learning Solutions window, on the CAEMaestro Evolve Module Menu, click **Start**.



Maestro Evolve Module Menu - Start and Invite

The Start Maestro Evolve Session window appears.

Start Maestro Evolve Session
You are about to start a new Maestro Evolve session, what would you like to do today:
Ad hoc session
Start previously planned session Tuesday, February 02, 2021, 5:00 pm - 3RIP 🗸
Participants link to join session https://cae-hc-cace-web-d- digitallearningsolutionswebapp.azurewebsites.net/lobby?accesscode=3RIP © COPY LINK
CANCEL START

Start Maestro Evolve Session - Previously Planned

- 2. Select Start previously planned session.
- 3. If there is more than one previously planned training session, click on the drop-down list to select the specific date and time for the current training event.
- 4. For participants that may not have received an invitation, the instructor may select the **COPY LINK** button. This will allow the instructor to paste the link in an email or chat window for sending to the intended participant.
- 5. Click **START**.

Once the instructor has started the session, participants waiting in the lobby will be directed to the Embody screen.



6. Click **Open.** The Select Simulator window appears.

CNE	\$
E	Ð
[MS]	
CAE Maestro	
Select your simulator	
Ares •	
Set as default	
Continue	
© 2020 CAE. All rights reserve	i.

Select Simulator Window

7. Continue with the simulation as planned to meet the learning objectives of the training event.

Share Virtual Equipment

Maestro Evolve allows the Instructor to share selected components of the simulation experience with learners.

To share virtual equipment with learners:

- 1. If the instructor has not begun a simulation, login to Maestro Evolve. Begin the simulation, setting appropriate parameters.
- 2. Keep the Maestro Evolve window open. At the top of the browser window, select the **CAE Digital Learning Solutions** tab.

CAE Digital Learning Solutions × 🙉 CAE Maestro × +
$\leftarrow \ \ \rightarrow \ \ \mathbb{C} (\ \ \textbf{a} \ \ \textbf{digital training-d.canadacentral.cloud app.azure.com/b763af0a-060d-4717-854f-f1cf})$
Running on the Fly (Modeled)
CPR Stan D. Ardman II I K Eyes

Browser Window Tabs

The CAE Learning Digital Solutions page appears.

3. From the Maestro Evolve Module Menu on the CAE Digital Learning Solutions page, select **Share**.



Maestro Evolve Module Menu - Open, Share, and Stop



The Share Link to Maestro Evolve window appears.

Access Code	c> COPY CODE	
	e corr code	
Patient Monitor		
https://digitaltraining-q.canadace	co COPY LINK	
Defibrillator		
https://digitaltraining-q.canadace	co COPY LINK	
AED		
https://digitaltraining-q.canadace	co COPY LINK	
Ventilator		
https://digitaltraining-q.canadace	co COPY LINK	
Embody		
https://cae-hc-cace-web-q-embo	co COPY LINK	
Transport Ventilator		
https://digitaltraining-q.canadace	co COPY LINK	
Anesthesia Machine		
https://digitaltraining-q.canadace	co COPY LINK	
Share the following links with your stu	dents so they can view patient status	

Share Link to Maestro Evolve

- 4. Select the virtual learning application to be shared with the learner(s). For example, Defibrillator or Ventilator.
- 5. Click the COPY LINK button associated with the selected virtual equipment.
- 6. Click **DONE**.
- 7. Paste the link into an application which would allow for communication with the student. Some examples are:
 - ° Email
 - ° Chat window

Note: This can be done by positioning the cursor in the application of choice and using the keyboard shortcut **CTRL+V**.

8. When the learner clicks the link, the application associated with the selected virtual equipment will launch, with the characteristics of the simulation in progress. The learner may then interact with the simulation from their device.

THIS PAGE INTENTIONALLY LEFT BLANK



APPENDIX A - FAQ

This Frequently Asked Questions (FAQ) section addresses some common concerns and inquiries. For any questions not addressed here, please contact CAE Healthcare Customer Service. Contact information is provided on the last page of this guide.

Maestro Evolve

Q: When attempting to Sign In, I enter the code sent to me via email and a verification failed error appears.


A: At the Enter Verification Code prompt window, click the **Send New Code** button. An email will be sent with a unique code.

- 1. Open the new email.
- 2. Retry signing in by entering the new code at the Sign In window.

Q: I am able to sign in and launch Maestro, but I am receiving a *Bad Gateway* error message when attempting to launch one or more of the virtual applications.

A: Close the browser window, relaunch the browser, and clear the data cache. For example, using Google Chrome:


- 1. Select the menu options from the top-right corner of the Google Chrome window.
- 2. Select More Tools > Clear Browsing Data.

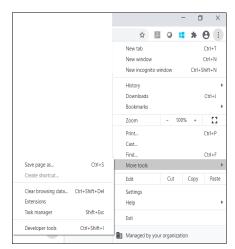

Google Chrome More Tools

- 3. Select the **Clear Data** button.
- 4. Retry signing in and launching the virtual application.

Q: I received a Connection Failed error message.

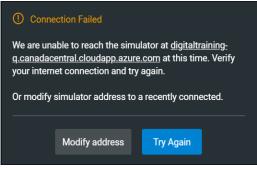

Connection Failed Error Message

A: Check the Internet connection and sign in again to Maestro Evolve.



Ventilator

Q: I am able to launch the Ventilator application. However, the screen is blank.

A: Confirm that the Ventilator power is enabled in Equipment options.

Q: I am able to launch the Ventilator application. However, I'm not seeing the data as expected.

A: Confirm that the probes are enabled and the ventilator is connected to the patient.


- 1. From the SCE Run screen, select the Equipment icon from the left control panel.
- 2. On the Ventilator Equipment parameters panel, enable probes and power.

Q: I am able to sign in and launch Maestro, but I am unable to see many of the controls that are referred to the guide.

A: It may be necessary to resize your browser window.

Q: I am able to launch the Ventilator application. However, when I click on the power button, the screen is blank and nothing happens.

A: Confirm that Maestro is running in the modeled mode. In the top left of the SCE Run screen, the Simulation Mode will be identified.

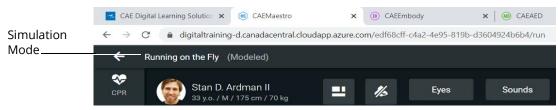

Running on the Fly in Modeled Mode

If modeled mode is not indicated, restart the simulation:

- 1. Close all open windows and exit Maestro Evolve.
- 2. Relaunch Maestro.
- 3. Select Run On The Fly (Modeled) from the Maestro Home screen.

Q: I have started a simulation but CAEVentilator, CAETransportVentilator, or CAEAnesthesiaMachine are not available options on the Maestro Application drop-down menu.

A: All ventilation equipment requires Maestro running in Modeled Mode. Verify that the simulation mode identified in the top left of the SCE Run screen is **Run On The Fly (Modeled).**

Embody

Q: I do not see the Embody option on the Maestro application drop-down menu. How do I launch Embody?

A: Embody is now launched from the Share Link options. See the section of this guide *Launch the Virtual Simulator* for more information.





Hardware Questions

Q: Can I connect multiple monitors to the system?

A: Yes, using multiple monitors is optional.

When a secondary monitor is connected, the software defaults to dual monitor configuration. If the dual monitor software is not set up, both monitors have the same image.

To configure the monitors.

- 1. Click the Windows Notification button on the bottom right hand corner of the monitor.
- 2. Click on **Project**. If you do not see the Project option, click **Expand** to view more options.

L ⁴	^m	ŵ	e ⁿ
Tablet mode	Network	All settings	
<i>₽</i> }>		I	
Airplane mode		Alarms only	
` *	÷.	*	~~~~~~~~~~~~~~~~~~~~~~~~~~~~~~~~~~~~~~~
50%	Night light	Not connected	VPN
φ	Ş	Ð	⊊ ∎
Battery saver	Screen snip	Project	Connect
	x ^e ^		4:51 PM /10/2020

Windows Notifications and Settings

3. Click on **Extend**.



Projection Options

THIS PAGE INTENTIONALLY LEFT BLANK



APPENDIX B- DEFIBRILLATOR CONTROLS AND DISPLAY OPTIONS

Available controls will vary based on the user's role. The following tables provide the available selections and defaults (when applicable) for:

- Facilitator Controls
- Learner Controls
- Learner Parameters Displayed
- Associated Maestro Preferences

Facilitator Controls

Defibrillator Facilitator Controls			
Parameter	Available Selections	Default	
ECG Cables	None	None	
	ECG 3 Lead		
	ECG 12 Lead		
ECG Placement	Correct	Correct	
	Incorrect		
Therapy Pads Reversal (Available in AED Mode)	On /Off	Off	
Therapy Pads (Available in AED Mode)	On /Off	Off	
Pulse Oximeter SpO ₂ (Plethysmograph)	On /Off	Off	
NIBP Cuff	On /Off	Off	
ABP Catheter	On /Off	Off	
CO ₂ Sample Line	On /Off	Off	
Temperature Probe	On /Off	Off	
Learner Layout Selection	Generic Defibrillator	Generic Defibrillator	
Metronome Rate (Available in AED mode)	100-120 beats per min.	100 beats per min.	
Power	On /Off	Off	
Energy Select	2-360 J	200 J	

Appendix B - Defibrillator Controls

CAEMaestro Evolve

Defibrillator Facilitator Controls			
Parameter	Available Selections	Default	
Pacing Rate	40-170 beats per min.	60 beats per min.	
Pacing Current	0-200 mA	0 mA	
AED Mode Metronome (Available in AED Mode)	On/Off	On	
AED Mode Metronome C:V	30:2	30:2	
Ratio	15:2		
Voice Instructions (Available in AED Mode)	On/Off	On	



Learner Controls

Parameter Power	Available Selections/Action	Default
Power		Beruart
	On/Off	Off
Analyze	Activates automated external defibrillator (AED) mode/analyzing status.	N/A
Manual Mode	Exits AED mode if analyze button is pressed.	N/A
Energy Select	2-360 J	200 J
Charge	Charges defibrillator to selected energy level.	N/A
Shock	Administers shock.	N/A
Disarm	Disarms energy for shock.	N/A
Sync	On/Off	Off
Pacer	On/Off	Off
Pause Pacer	On (Requires Hold)	N/A
	Off (Hold Released)	
Pacing Rate	40-170 ppm	60 ppm
Pacing Current	0-200 mA	0 MA
NIBP	Manual	Manual
	1 minute	
	2 minutes	
	3 minutes	
	5 minutes	
	10 minutes	
	15 minutes	
	30 minutes	
	60 minutes	
12-lead	12-lead report generated	N/A
Up	Navigation button	N/A
Down	Navigation button	N/A

Defibrillator Learner Controls		
Parameter	Available Selections/Action	Default
Back	Navigation button	N/A
Confirm	Navigation button	N/A
Scale	Adjustment of scale level for waveforms.	N/A
Color	Adjustment of color for vitals.	N/A
Layout	Adjustment of display layout of the vitals.	N/A
CPR	Off	Off
	30:2	
	15:2	
Alarms	Off	On
	On	
	Suspend for 20 seconds	
	Suspend for 1 minute	
	Suspend for 2 minutes	
	Suspend for 5 minutes	
Alarm Level	Available for each vital with an alarm.	
Alarm Sound	On/Off	On
HR/PR Detection Sound	On/Off	Off
Display Lock	Locked/Unlocked	Unlocked



Learner Display

Defibrillator Learner Display		
Parameter	Values	
Temperature	24.8-45.2 °C	
Pulse Rate (PR) from SpO ₂	25-240 bpm	
SpO ₂	0-100%	
NIBP	Systolic: 30-255 mmHg Diastolic: 15-220 mmHg Mean: 20-235 mmHg	
Pulse Rate (PR) from NIBP	30-240 bpm	
АВР	Systolic: 0-300 mmHg Diastolic: 0-300 mmHg Mean: 0-300 mmHg	
Pulse Rate (PR) from ABP	30-240 bpm	
End-tidal CO2 (EtCO ₂)	0-100 mmHg	
Respiration Rate (RR)	0-100 bpm (breaths/min)	
Heart Rate (HR) from ECG	20-300 bpm	
STJ Level	-10.00-10.00 mm	
PR Interval	0.000-1.000 s	
QRS Interval	0.000-1.000 s	
QT/QTc Interval	0.000-1.000 s	
P-QRS-T Axes	-360-360 °	
Shocks Delivered	0-50	
Gain	1 mV	
Speed	25 mm/sec	
Status Messages - Active mode, instructions, indications	N/A	
Elapsed Time	hh:mm:ss	
Current Date and Time	dd/mm/yyyy, hh:mm AM/PM	
Battery Status	N/A	
Connection Status	N/A	
Alarm Indicators	N/A	

Associated Maestro Preferences

Maestro Preferences Associated with Defibrillator		
Parameter	Available Selections	Default
Energy (electrical therapy)	J	J
	Ws	
CO ₂	mmHg	mmHg
	kPa	
	%	



APPENDIX C - AED CONTROLS AND DISPLAY OPTIONS

Available controls will vary based on the user's role. The following tables provide the available selections and defaults (when applicable) for:

- Facilitator Controls
- Learner Controls
- Learner Parameters Displayed
- Associated Maestro Preferences

Facilitator Controls

AED Facilitator Controls		
Parameter	Available Selections Default	
Therapy Pads	On /Off	Off
Learner Layout Selection	Generic Defibrillator Generic Defibrillator	
Metronome Rate	100-120 beats per min. 100 beats per r	
Power	On /Off	Off
Energy Select	2-360 J 200 J	
Metronome	On/Off On	
Voice Instructions	On/Off	On

Learner Controls

AED Learner Controls		
Parameter Available Selections/Action Default		Default
Power	On/Off	Off
Shock	Administers shock	N/A

Learner Display

AED Learner Display		
Parameter	Values	
Shocks Delivered	0-50	
Status Messages - instructions, indications	N/A	
Elapsed Time	hh:mm:ss	
Current Date and Time	dd/mm/yyyy, hh:mm AM/PM	
Battery Status	N/A	
Connection Status	N/A	
Alarm Indicators	N/A	

Associated Maestro Preferences

Maestro Preferences Associated with AED		
Parameter	Available Selections	Default
Energy (electrical therapy)	J	J
	Ws	



APPENDIX D - VENTILATOR CONTROLS AND DISPLAY OPTIONS

Available controls will vary based on the user's role and the equipment that is used (e.g. not all controls are available when using the Transport Ventilator that may be available in the Ventilator). The following tables provide selections and defaults (when applicable) for:

- Facilitator Controls
- Learner Controls
- Learner Parameters Displayed
- Learner Display Options
- Associated Maestro Preferences

Facilitator Controls

Ventilator Facilitator Controls		
Parameter	Selections	Default
Breathing Circuit	On /Off	Off
Pulse Oximeter SpO ₂ - Plethysmograph	On /Off	Off
CO ₂ Attachment - Capnograph	On /Off	Off
Edi Catheter	On /Off	Off
Breathing Circuit Disconnect	On /Off	Off
Leak	0-100%	0%
NAVA Mode - Neurally Adjusted Ventilatory Assist Mode	Enabled/Disabled	Disabled
Power	On /Off	Off
Pi - Inspiratory Pressure	0-80 cmH ₂ O	20 cmH ₂ O
Pi Backup	0-80 cmH ₂ O	20 cmH ₂ O
ΔPsupp - Support Pressure	0-80 cmH ₂ O	20 cmH ₂ O
PEEP - Positive End Expiratory Pressure	0-50 cmH ₂ O	5 cmH ₂ O
End Inspiration - Percent of Peak Flow	0-70%	30%
Flow Trigger	0.2-20.0 L/min	3 L/min
VT - Tidal Volume	100-4000 ml	500 ml

Appendix D - Ventilator Controls

CAEMaestro Evolve

Ventilator Facilitator Controls		
Parameter	Selections	Default
VT Backup	100-4000 ml	500 ml
FiO ₂ - Oxygen Concentration of the Delivered Gas	0-100%	40%
Tpause - Pause Time	0.0-1.5 seconds	0s
I:E Inspiratory time:expiratory time ratio	1.0:1.0 to 1.0:4.0 (I:E)	1:2
l:E Inspiratory time:expiratory time ratio backup	1.0:1.0 to 1.0:4.0 (I:E)	1:2
Ti - Inspiratory Time (non-functional)	0.1-5.0 seconds	N/A
Ti Backup (non-functional)	0.1-5.0 seconds	N/A
RR - Respiratory Rate	4-150 breaths per minute	15 breaths per min.
RR Backup	4-150 breaths per minute	15 breaths per min
Ti rise - Inspiratory Rise Time	0.1-0.4 seconds	0.2s
Ta - Apnea Time	15-45 seconds	20s
Edi Trigger (Depends on if NAVA mode is enabled or disabled)	0.1-2.0 μV	0.5 μV
NAVA Level (Depends on if NAVA mode is enabled or disabled)	0-15 cmH ₂ O/μV	0.5 cmH ₂ O/μV
Type of Ventilation	Invasive	Invasive
	Non-invasive	
Ventilation Mode	 Volume-controlled Ventilation (VCV) Pressure-controlled Ventilation (PCV) Continuous Positive Airway Pressure + Pressure Support (CPAP+PS) Volume Support Ventilation (VSV) Neurally Adjusted Ventilatory Assist (NAVA) Synchronized Intermittent-Mandatory Ventilation Volume Control (SIMV VC) 	Volume-controlled Ventilation (VCV)



Ventilator Facilitator Controls		
Parameter Selections Default		Default
Ventilation Status	Start	Standby
	Standby	

Learner Controls

Ventilator Learner Controls		
Parameter	Selections	Default
Pi - Inspiratory Pressure	0-80 cmH ₂ O	20 cmH ₂ O
Pi Backup	0-80 cmH ₂ O	20 cmH ₂ O
ΔPsupp - Support Pressure	0-80 cmH ₂ O	20 cmH ₂ O
PEEP - Positive end-expiratory Pressure	0-50 cmH ₂ O	5 cmH ₂ O
End Inspiration - Percent of Peak Flow	0-70%	30%
Flow Trigger	0.2-20.0 L/min	3.0 L/min
VT - Tidal Volume	100-1500ml	500 ml
VT Backup	100-1500 ml	500 ml
FiO ₂ - Oxygen Concentration of the Delivered Gas	21-100%	40%
Tpause- Pause Time	0.0-1.5 seconds	0s
Inspiratory Hold	Input Control	N/A
Expiratory Hold	Input Control	N/A
I:E Inspiratory time:expiratory time ratio	1.0:1.0 to 1.0:4.0 (I:E)	1:2
I:E Inspiratory time:expiratory time ratio backup	1.0:1.0 to 1.0:4.0 (I:E)	1:2
Ti rise - Inspiratory Rise Time	0.1-0.4 seconds	0.2s
Tapnea - Apnea Time	15-45 seconds	20s
Edi Trigger (Depends on if NAVA mode is enabled or disabled)	0.1-2.0 μV	0.5 μV
NAVA Level (Depends on if NAVA mode is enabled or disabled)	0-15 cmH ₂ O/μV	0.5 cmH ₂ O/μV
Type of Ventilation	Invasive	Invasive
	Non-invasive	



Ventilator Learner Controls		
Parameter	Selections	Default
Ventilation Mode	 Volume-controlled Ventilation (VCV) Pressure-controlled Ventilation (PCV) Continuous Positive Airway Pressure + Pressure Support (CPAP+PS) Volume Support Ventilation (VSV) Neurally Adjusted Ventilatory Assist (NAVA) Synchronized Intermittent-Mandatory Ventilation Volume Control (SIMV VC) 	Volume-controlled Ventilation (VCV)
Ventilation Setup	Setup options will vary depending upon the ventilation mode selected.	N/A
Ventilation Status	Start	Standby
	Standby	
Power	On/Off	On
Display Lock/Unlock	Locked/Unlocked	Unlocked
Navigation Dial and Select	Navigate and select the highlighted option.	N/A
Scale	Adjustment of scale level for vital signs.	N/A
Layout	Adjustment of display layout of the vital signs.	N/A
Alarms Silencer	Silences all alarms for 2 minutes.	N/A
Alarm Level	Available for indicated vital signs.	N/A
Alarm Sound (Silences for 2 minutes)	On/Off	On

Learner Parameters Displayed

Ventilator Learner Display		
Parameter	Values	
Pmean - Mean airway pressure	-20-200 cmH ₂ O	
Ppeak - Peak airway pressure	-20-200 cmH ₂ O	
Pplateau	-20-200 cmH ₂ O	
PEEP - Positive end-expiratory Pressure	-20-200 cmH ₂ O	
Pi - Inspiratory Pressure (waveform value)	-20-200 cmH ₂ O	
Insp Flow - Peak Inspiratory Flow	-400-400 L/min	
Exp Flow - Peak Expiratory Flow	-400-400 L/min	
Flow Trig	0.2 -20.0	
MVe - Expiratory Minute Volume	0-100 L/min	
MVi - Inspiratory Minute Volume	0-100 L/min	
MV - Minute Volume	0-100 L/min	
MVspont - Spontaneous Expiratory Minute Volume	0-100 L/min	
VTe - Expiratory Tidal Volume	0-6000 ml	
VTi - Inspiratory Tidal Volume	0-6000 ml	
Leak - Leakage percent	0-100%	
VT - Tidal Volume	0-6000 ml	
FiO ₂ - Oxygen Concentration of the Delivered Gas	0-100%	
RRsp - Spontaneous Respiratory Rate	0-150 breaths per minute	
Te - Expiratory Time	0.10-0.80 seconds	
I:E - Inspiratory:Expiratory Ratio	1.0:600 to 150:1.0 (I:E)	
Tpause- Pause Time	0.0-1.5 seconds	
Ti - Inspiratory Time	0.2-10.0 seconds	
RR- Respiratory Rate	0-150 breaths per minute	
Ti rise - Inspiratory Rise Time	0.0-0.4 seconds	
Cstat - Static Compliance	0-500 ml/ cmH ₂ O	
Cdyn - Dynamic Compliance	0-200 ml/ cmH ₂ O	



Appendix D - Ventilator Controls

Ventilator Learner Display		
Parameter	Values	
Ri - Inspiratory Flow Resistance	0-500 cmH ₂ O/(L/s)	
Re - Expiratory Flow Resistance	0-500 cmH ₂ O/(L/s)	
RSB - Rapid Shallow Breathing Index	0.1-600.0 1/(L*min)	
RCe- Expiratory Time Constant	0-3 seconds	
RCi - Inspiratory Time Constant	0-3 seconds	
EtCO ₂ - End Tidal CO ₂	0-100 mmHg	
VeCO ₂ - Exhaled CO ₂ Volume	0-1000 ml	
ViCO ₂ - Inspired CO ₂ Volume	0-1000 ml	
RREtCO ₂ - Respiratory Rate From EtCO ₂	0-100 breaths per minute	
SpO ₂ - Oxygen Saturation	0-100%	
PRSpO ₂ - Pulse rate from SpO ₂ probe	0-275 beats per minute	
Edipeak	0-50 μV	
Edimin	0-50 μV	
Pressure Waveform	-20 to 200 cmH ₂ O	
Flow Waveform	-400 to 400 L/min	
Volume Waveform	0 to 6000 ml	
Pressure Volume Loop	-20 to 200 cmH ₂ O and 0 to 6000 ml	
Flow Pressure Loop	-400 to 400 L/min and -20 to 200 cmH ₂ O	
Volume Flow Loop	0 to 6000 ml and -400 to 400 L/min	
Edi Waveform	0-50 μV	
SpO ₂ (Plethysmograph) Waveform	0-100%	
CO ₂ (Capnograph) Waveform	0-100%	
Status Messages Area - Alarms, Stand-by or Active, Ventilation Details, Patient Information	N/A	
Alarm Indicators	N/A	
Trigger Indicators - Flow and Edi Waveform Trigger Indicators	N/A	
Current Date and Time	N/A	
Battery Status	N/A	

Ventilator Learner Display	
Parameter Values	
Connection Status	N/A

Learner Display Options

Ventilator Display Option	Selections
Waves and Numerics	3 Waveforms and 8 Numerics
Loop Graph, Waves and Numerics	3 Loops and 2 Waveforms and 8 Numerics
All Numerics	35 Numerics
Waves and Numerics	4 Waveforms and 8 Numerics

Associated Maestro Preferences

Maestro Preferences Associated with Ventilator			
Parameter	Selections	Default	
Airway Pressure	cmH ₂ O	cmH ₂ O	
	mBar		
CO ₂	mmHg	mmHg	
	kPa		
	%		



APPENDIX E - TRANSPORT VENTILATOR CONTROLS AND DISPLAY OPTIONS

Available controls will vary based on the user's role and the equipment that is used (e.g. not all controls are available when using the Transport Ventilator that may be available in the Ventilator). The following tables provide selections and defaults (when applicable) for:

- Facilitator Controls
- Learner Controls
- Learner Parameters Displayed
- Learner Display Options
- Associated Maestro Preferences

Facilitator Controls

Transport Ventilator Facilitator Controls		
Parameter	Selections	Default
Breathing Circuit	On /Off	Off
Pulse Oximeter SpO ₂ - Plethysmograph	On /Off	Off
CO ₂ Attachment - Capnograph	On /Off	Off
Breathing Circuit Disconnect	On /Off	Off
Leak	0-100%	0%
Power	On /Off	Off
Pi - Inspiratory Pressure	0-80cmH ₂ O	20 cmH ₂ O
Pi Backup	0-80cmH ₂ O	20 cmH ₂ O
ΔPsupp - Support Pressure	0-80cmH ₂ O	20 cmH ₂ O
PEEP - Positive End Expiratory Pressure	0-50 cmH ₂ O	5 cmH ₂ O
End Inspiration - Percent of Peak Flow	0-70%	30%
Flow Trigger	0.2-20.0 L/min	3 L/min
VT - Tidal Volume	100-1500 ml	500 ml
VT Backup	100-1500 ml	500 ml
FiO ₂ - Oxygen Concentration of the Delivered Gas	0-100%	40%

Appendix E - Transport Ventilator Controls CAEMaestro Evolve

Parameter	Selections	Default
Tpause - Pause Time	0.0-1.5 seconds	0s
I:E Inspiratory time:expiratory time ratio	1.0:1.0 to 1.0:4.0 (I:E)	1:2
I:E Inspiratory time:expiratory time ratio backup	1.0:1.0 to 1.0:4.0 (I:E)	1:2
RR - Respiratory Rate	4-150 breaths per minute	15 breaths per min.
RR Backup	4-150 breaths per minute	15 breaths per min
Ti rise - Inspiratory Rise Time	0.1-0.4 seconds	0.2s
Ta - Apnea Time	15-45 seconds	20s
Type of Ventilation	Invasive	Invasive
	Non-invasive	
Ventilation Mode	 Volume-controlled Ventilation (VCV) Pressure-controlled Ventilation (PCV) Continuous Positive Airway Pressure + Pressure Support (CPAP + PS) Volume Support Ventilation (VSV) Neurally Adjusted Ventilatory Assist (NAVA) Synchronized Intermittent-Mandatory Ventilation Volume Control (SIMV VC) 	Volume-controlled Ventilation (VCV)
Ventilation Status	Start	Standby
	Standby	



Learner Controls

Transport Ventilator Learner Controls		
Parameter	Selections	Default
Pi - Inspiratory Pressure	0-80cmH ₂ O	20 cmH ₂ O
Pi Backup	0-80cmH ₂ O	20 cmH ₂ Om,
ΔPsupp - Support Pressure	0-80cmH ₂ O	20 cmH ₂ O
PEEP - Positive end-expiratory Pressure	0-50 cmH ₂ O	5 cmH ₂ O
End Inspiration - Percent of Peak Flow	0-70%	30%
Flow Trigger	0.2-20.0 L/min	3.0 L/min
VT - Tidal Volume	100-1500 ml	500 ml
VT Backup	100-1500 ml	500 ml
FiO ₂ - Oxygen Concentration of the Delivered Gas	21-100%	40%
Tpause- Pause Time	0.0-1.5 seconds	0s
I:E Inspiratory time:expiratory time ratio	1.0:1.0 to 1.0:4.0 (I:E)	1:2
I:E Inspiratory time:expiratory time ratio backup	1.0:1.0 to 1.0:4.0 (I:E)	1:2
Ti rise - Inspiratory Rise Time	0.1-0.4 seconds	0.2s
Tapnea - Apnea Time	15-45 seconds	20s
Type of Ventilation	Invasive	Invasive
	Non-invasive	

Appendix E - Transport Ventilator Controls CAEMaestro Evolve

Transport Ventilator Learner Controls			
Parameter	Selections	Default	
Ventilation Mode	 Volume-controlled Ventilation (VCV) Pressure-controlled Ventilation (PCV) Continuous Positive Airway Pressure + Pressure Support (CPAP + PS) Volume Support Ventilation (VSV) Neurally Adjusted Ventilatory Assist (NAVA) Synchronized Intermittent-Mandatory Ventilation Volume Control (SIMV VC) 	Volume-controlled Ventilation (VCV)	
Ventilation Setup	Setup options will vary depending upon the ventilation mode selected.	N/A	
Ventilation Status	Start Standby	Standby	
Power	On/Off	On	
Display Lock/Unlock	Locked/Unlocked	Unlocked	
Navigation Dial and Select	Navigate and select the highlighted option.	N/A	
Scale	Adjustment of scale level for vital signs.	N/A	
Layout	Adjustment of display layout of the vital signs.	N/A	
Alarms Silencer	Silences all alarms for 2 minutes.	N/A	
Alarm Level	Available for each vital sign.	N/A	
Alarm Sound (Silences for 2 minutes)	On/Off	On	



Learner Parameters Displayed

Transport Ventilator Learner Display		
Parameter	Values	
Pmean - Mean airway pressure	-20-200 cmH ₂ O	
Ppeak - Peak airway pressure	-20-200 cmH ₂ O	
PEEP - Positive end-expiratory Pressure	-20-200 cmH ₂ O	
Insp Flow - Peak Inspiratory Flow	-400-400 L/min	
Exp Flow - Peak Expiratory Flow	-400-400 L/min	
MVe - Expiratory Minute Volume	0-100 L/min	
MVi - Inspiratory Minute Volume	0-100 L/min	
MV - Minute Volume	0-100 L/min	
VTe - Expiratory Tidal Volume	0-6000 ml	
VTi - Inspiratory Tidal Volume	0-6000 ml	
Leak - Leakage percent	0-100%	
VT - Tidal Volume	0-6000 ml	
FiO ₂ - Oxygen Concentration of the Delivered Gas	0-100%	
Te - Expiratory Time	0.10-0.80 seconds	
I:E - Inspiratory:Expiratory Ratio	1.0:600 to 150:1.0 (I:E)	
Tpause- Pause Time	0.0-1.5 seconds	
Ti - Inspiratory Time	0.2-10.0 seconds	
RR- Respiratory Rate	0-150 breaths per minute	
Ti rise - Inspiratory Rise Time	0.0-0.4 seconds	
EtCO ₂ - End Tidal CO ₂	0-100 mmHg	
RREtCO ₂ - Respiratory Rate From EtCO ₂	0-100 breaths per minute	
SpO ₂ - Oxygen Saturation	0-100%	
PRSpO ₂ - Pulse rate from SpO ₂ probe	0-275 beats per minute	
Pressure Waveform	-20 to 200 cmH ₂ O	
Flow Waveform	-400 to 400 L/min	
Volume Waveform	0-6000 ml	

Appendix E - Transport Ventilator Controls CAEMaestro Evolve

Transport Ventilator Learner Display			
Parameter	Values		
Pressure Volume Loop	-20 to 200 cmH ₂ O to 0-6000 ml		
Pressure Flow Loop	-20 to 200 cmH ₂ O to -400 to 400 L/min		
Volume Flow Loop	0-6000 ml to -400 to 400 L/min		
SpO ₂ (Plethysmograph) Waveform	0-100%		
CO ₂ (Capnograph) Waveform	0-100%		
Status Messages Area - Alarms, Stand-by or Active, Ventilation Details, Patient Information	N/A		
Alarm Indicators	N/A		
Trigger Indicators - Flow and Edi Waveform Trigger Indicators	N/A		
Current Date and Time	N/A		
Battery Status	N/A		
Connection Status	N/A		

Learner Display Options

Transport Ventilator Display Option	Selections
Waves and Numerics	2 Waveforms and 6 Numerics
Loop Graph and Numerics	2 Loops and 6 Numerics
All Numerics	23 Numerics

Associated Maestro Preferences

Maestro Preferences Associated with Transport Ventilator		
Parameter	Selections	Default
Airway Pressure	cmH ₂ O	cmH ₂ O
	mBar	
CO ₂	mmHg	mmHg
	kPa	
	%	



APPENDIX F - ANESTHESIA MACHINE CONTROLS AND DISPLAY

Available controls will vary based on the user's role and the equipment that is used (e.g. not all controls are available when using the Transport Ventilator that may be available in the Ventilator). The following tables provide selections and defaults (when applicable) for:

- Facilitator Controls
- Learner Controls
- Learner Parameters Displayed
- Learner Display Options
- Associated Maestro Preferences

Facilitator Controls

Anesthesia Machine Facilitator Controls		
Parameter	Selections	Default
Breathing Circuit	On /Off	Off
Pulse Oximeter SpO ₂ - Plethysmograph	On /Off	Off
CO ₂ Attachment - Capnograph	On /Off	Off
Breathing Circuit Disconnect	On /Off	Off
Leak	0-100%	0%
Power	On /Off	Off
Pi - Inspiratory Pressure	0-80 cmH ₂ O	20 cmH ₂ O
Pi Backup	0-80 cmH ₂ O	20 cmH ₂ O
ΔPsupp - Support Pressure	0-80 cmH ₂ O	20 cmH ₂ O
PEEP - Positive End Expiratory Pressure	0-70 cmH ₂ O	5 cmH ₂ O
End Inspiration - Percent of Peak Flow	0-70%	30%
Flow Trigger	0.2-20.0 L/min	3 L/min
VT - Tidal Volume	100-1500 ml	500 ml
VT Backup	100-1500 ml	500 ml
Tpause - Pause Time	0.0-1.5 seconds	Os
I:E Inspiratory time:expiratory time ratio	1.0:1.0 to 1.0:4.0 (I:E)	1:2

Appendix F - Anesthesia Machine Controls CAEMaestro Evolve

Anesthesia Machine Facilitator Controls			
Parameter	Selections	Default	
l:E Inspiratory time:expiratory time ratio backup	1.0:1.0 to 1.0:4.0 (l:E)	1:2	
RR - Respiratory Rate	4-150 breaths per minute	15 breaths per min.	
RR Backup	4-150 breaths per minute	15 breaths per min	
Ti rise - Inspiratory Rise Time	0.1-0.4 seconds	0.2s	
Ta - Apnea Time	15-45 seconds	20s	
Type of Ventilation	Invasive Non-invasive	Invasive	
Ventilation Mode	 Volume-controlled Ventilation (VCV) Pressure-controlled Ventilation (PCV) Continuous Positive Airway Pressure + Pressure Support (CPAP+PS) Volume Support Ventilation (VSV) Synchronized Intermittent-Mandatory Ventilation Volume Control (SIMV VC) 	Pressure-controlled Ventilation (PCV)	
Ventilation Status	Start Standby	Standby	
Soda Lime Canister Consumed	On Off	Off	
Auxiliary Common Gas Outlet (ACGO) Open	On Off	Off	
APL	0-70 cmH ₂ O	0 cmH ₂ O	
O ₂ Flow	0.2-15.0 L/min	0.2 L/min	
N ₂ O Flow	0.0-10.0 L/min	0.0 L/min	
Air Flow	0.0-15.0 L/min	0.0 L/min	
Volume Percent of Isoflurane	0.0-6.0%	0%	
Volume Percent of Sevoflurane	0.0-8.0%	0%	



Appendix F - Anesthesia Machine Controls

Anesthesia Machine Facilitator Controls			
Parameter	Selections	Default	
Volume Percent of Desflurane	0.0-6.0%	0%	
Manual Ventilation	On	Off	
	Off		

Learner Controls

Anesthesia Machine Learner Controls			
Parameter	Selections	Default	
Pi - Inspiratory Pressure	0-80 cmH ₂ O	20 cmH ₂ O	
Pi Backup	0-80 cmH ₂ O	20 cmH ₂ O	
ΔPsupp - Support Pressure	0-80 cmH ₂ O	20 cmH ₂ O	
PEEP - Positive end-expiratory Pressure	0-70 cmH ₂ O	5 cmH ₂ O	
End Inspiration - Percent of Peak Flow	0-70%	30%	
Flow Trigger	0.2-20.0 L/min	3.0 L/min	
VT - Tidal Volume	100-4000 ml	500 ml	
VT Backup	100-4000 ml	500 ml	
FiO ₂ - Oxygen Concentration of the Delivered Gas	21-100%	40%	
Tpause- Pause Time	0.0-1.5 seconds	0s	
Inspiratory Hold (Rebreather Bag)	Input Control	N/A	
I:E Inspiratory time:expiratory time ratio	1.0:1.0 to 1.0:4.0 (I:E)	1:2	
I:E Inspiratory time:expiratory time ratio backup	1.0:1.0 to 1.0:4.0 (I:E)	1:2	
Ti rise - Inspiratory Rise Time	0.1-0.4 seconds	0.2s	
Tapnea - Apnea Time	15-45 seconds	20s	
Type of Ventilation	Invasive	Invasive	
Ventilation Mode	 Non-invasive Volume-controlled Ventilation (VCV) Pressure-controlled Ventilation (PCV) Continuous Positive Airway Pressure + Pressure Support (CPAP+PS) Volume Support Ventilation (VSV) Synchronized Intermittent-Mandatory Ventilation Volume Control (SIMV VC) 	Pressure-controlled Ventilation (PCV)	



Appendix F - Anesthesia Machine Controls

Anesthesia Machine Learner Controls			
Parameter	Selections	Default	
Ventilation Setup	Setup options will vary depending upon the ventilation mode selected.	N/A	
Ventilation Status	Start	Standby	
	Standby		
Power	On/Off	On	
Display Lock/Unlock	Locked/Unlocked	Unlocked	
Navigation Dial and Select	Navigate and select the highlighted option.	N/A	
Scale	Adjustment of scale level for vital signs.	N/A	
Layout	Adjustment of display layout of the vital signs.	N/A	
Alarms Silencer	Silences all alarms for 2 minutes.	N/A	
Alarm Level	Available for each vital sign.	N/A	
Alarm Sound (Silences for 2 minutes)	On/Off	On	
Rebreather Bag	Full Squeeze	N/A	
	Half Squeeze		
Soda Lime Canister Reveal	On	Off	
	Off		
Auxiliary Common Gas Outlet (ACGO)	On	Off	
Open	Off		
APL	0-70 cmH ₂ O	0 cmH ₂ O	
O ₂ Flow	0.2-15.0 L/min	0.2 L/min	
N ₂ O Flow	0.0-10.0 L/min	0.0 L/min	
Air Flow	0.0-15.0 L/min	0.0 L/min	
Volume Percent of Isoflurane	0.0-6.0%	0%	
Volume Percent of Sevoflurane	0.0-8.0%	0%	
Volume Percent of Desflurane	0.0-6.0%	0%	
0 ₂ +	On	Off	
	Off		

Appendix F - Anesthesia Machine Controls CAEMaestro Evolve

Anesthesia Machine Learner Controls				
Parameter Selections Default				
Manual Ventilation	On	Off		
Off				



Learner Parameters Displayed

Anesthesia Machine Learner Display		
Parameter	Values	
Pmean - Mean airway pressure	-20-200 cmH ₂ O	
Ppeak - Peak airway pressure	-20-200 cmH ₂ O	
Pplateau	-20-200 cmH ₂ O	
PEEP - Positive end-expiratory Pressure	-20-200 cmH ₂ O	
Insp Flow - Peak Inspiratory Flow	-400-400 L/min	
Exp Flow - Peak Expiratory Flow	-400-400 L/min	
MVe - Expiratory Minute Volume	0-100 L/min	
MVi - Inspiratory Minute Volume	0-100 L/min	
MV - Minute Volume	0-100 L/min	
MVspont - Spontaneous Expiratory Minute Volume	0-100 L/min	
VTe - Expiratory Tidal Volume	0-6000 ml	
VTi - Inspiratory Tidal Volume	0-6000 ml	
Leak - Leakage percent	0-100%	
VT - Tidal Volume	0-6000 ml	
RRsp - Spontaneous Respiratory Rate	0-150 breaths per minute	
Te - Expiratory Time	0.10-0.80 seconds	
I:E - Inspiratory:Expiratory Ratio	1.0:600 to 150:1.0 (I:E)	
Tpause- Pause Time	0.0-1.5 seconds	
Ti - Inspiratory Time	0.2-10.0 seconds	
RR- Respiratory Rate	0-150 breaths per minute	
Ti rise - Inspiratory Rise Time	0.0-0.4 seconds	
Cstat - Static Compliance	0-500 ml/ cmH ₂ O	
Cdyn - Dynamic Compliance	0-200 ml/ cmH ₂ O	
Ri - Inspiratory Flow Resistance	0-500 cmH ₂ O/(L/s)	
Re - Expiratory Flow Resistance	0-500 cmH ₂ O/(L/s)	
RSB - Rapid Shallow Breathing Index	0.1-600.0 1/(L*min)	
RCe- Expiratory Time Constant	0-3 seconds	

Appendix F - Anesthesia Machine Controls CAEMaestro Evolve

Anesthesia Machine Learner Display		
Parameter	Values	
RCi - Inspiratory Time Constant	0-3 seconds	
EtCO ₂ - End Tidal CO ₂	0-100 mmHg	
VeCO ₂ - Exhaled CO ₂ Volume	0-1000 ml	
ViCO ₂ - Inspired CO ₂ Volume	Х	
RREtCO ₂ - Respiratory Rate From EtCO ₂	0-100 breaths per minute	
SpO ₂ - Oxygen Saturation	0-100%	
PRSpO ₂ - Pulse rate from SpO ₂ probe	0-275 beats per minute	
Pressure Waveform	-20 to 200 cmH ₂ O	
Flow Waveform	-400 to 400 L/min	
Volume Waveform	0 to 6000 ml	
Pressure Volume Loop	-20 to 200 cmH ₂ O and 0-6000 ml	
Flow Pressure Loop	400 to 400 L/min and -20 to 200 cmH ₂ O	
Volume Flow Loop	0 to 6000 ml and -400 to 400 L/min	
SpO ₂ (Plethysmograph) Waveform	0-100%	
CO ₂ (Capnograph) Waveform	0-100%	
Fraction of inspired oxygen	0-100%	
End tidal oxygen	0-100%	
Fraction of inspired nitrous oxide	0-100%	
End tidal nitrous oxide	0-100%	
Fraction of inspired isoflurane	0.0-6.0%	
End tidal isoflurane	0.0-6.0%	
Fraction of inspired sevoflurane	0.0-8.0%	
End tidal sevoflurane	0.0-8.0%	
Fraction of inspired desflurane	0.0-20.0%	
End tidal desflurane	0.0-20.0%	
Minimum alveolar concentration	0.0-100.0 MAC	
TOTAL - Total gas flow	0.0-40.0 L/min	
O ₂ - Oxygen gas flow	0.2-15.0 L/min	



Anesthesia Machine Learner Display			
Parameter	Values		
N ₂ O - N ₂ O gas flow	0.0-10.0 L/min		
Air - Air gas flow	0.0-15.0 L/min		
Status Messages Area - Alarms, Stand-by or Active, Ventilation Details, Patient Information	N/A		
Alarm Indicators	N/A		
Trigger Indicators - Flow and Edi Waveform Trigger Indicators	N/A		
Current Date and Time	N/A		
Battery Status	N/A		
Connection Status	N/A		

Learner Display Options

Ventilation Display Option	Selections
Waves and Numerics	2 Waveforms and 6 Numerics
Loop Graph and Numerics	2 Loops and 6 Numerics
All Numerics	35 Numerics

Associated Maestro Preferences

Maestro Preferences Associated with Anesthesia Machine						
Parameter Selections Default						
Airway Pressure	cmH ₂ O	cmH ₂ O				
	mBar					
CO ₂	mmHg	mmHg				
	kPa					
	%					
Color Preference	ISO	ISO				
	USA					

THIS PAGE INTENTIONALLY LEFT BLANK



APPENDIX G - MEDICATIONS

The following table describes the medications available for administration in the Maestro software. Each medication is listed along with:

- Any effects on the patient's physiology (modeled or log-only)
- The medication category
- The units used for dosages
- The route of administration.

Medication	Modeled or	Category	Dosage (Dosage Options	
	Log Only		Units	Route	
Acetaminophen	L	Analgesic	mg	РО	
Activated Charcoal	L	Adsorbant	g	РО	
Adenosine	М	Cardiovascular Emergency	mg	IV	
Albuterol	L	Respiratory	mg	Inhaler Nebulizer	
Alfentanil	М	Analgesic Narcotic	mcg/kg	IV	
Amiodarone	М	Cardiovascular Emergency	mg mg/min	IV	
Aspirin	L	Cardiovascular Analgesics NSAID Metabolic	mg	PO	
Atropine	М	Cardiovascular Emergency	mg	IO, IV	
Betamethasone	М	Corticosteriod	mg	IM	
Bicarbonate	М	Cardiovascular Emergency Toxicology	mEq	IV	
Bretylium	М	Cardivascular	mg/min mg/kg	IV	
Calcium Chloride	М	Cardiovascular Emergency Toxicology	g mg	IV, IO	
Captopril	L	Cardiovascular	mg	PO	

Appendix G - Medications

CAEMaestro Evolve

Medication	Modeled or	Category	Dosage Options	
	Log Only		Units	Route
Cisatracurium	М	Neuromuscular Blocker	mg/kg mg	IV
Curare (d-Tubocurarine)	М	Neuromuscular Blocker	mg	IM, IV
Dextrose 50%	L	Anti-Hypoglycemic Metabolic	mg	IV
Diazepam	М	Hypnotic	mg	IO
Digoxin	М	Cardiovascular	mcg	IV
Diltiazem	М	Cardiovascular	mg	IV, IO
Diphenhydramine	L	Histamine Blocker	mg	IV, IO, IM
Dobutamine	M	Cardiovascular Emergency	mcg/kg/min	IV, IO
Dopamine	М	Cardiovascular	mcg/kg/min	IV, IO
Doxacurium	М	Neuromuscular Blocker	mg/kg	IV
Droperidol	М	Antagonists Antiemetics	mg	IV
Edrophonium	М	Antiemetics	mg	IM, IV
Epinephrine	L	Cardiovascular	mg	IM, IV, SC
Epinephrine 1:1,000	L	Respiratory	mg	IM
Epinephrine 1:10,000	М	Cardiovascular	mcg	IO, IV
Esmolol	L	Cardiovascular	mg	IV
Etomidate	L	Hypnotic	mg/kg	IV
Fentanyl	L	Analgesic Narcotic	mcg	IM, IO, PO, Transdermal
Flumazenil	L	Antagonist	mg	IV
Furosemide	L	Diuretic	mg	IO, IV, PO
Glucagon	L	Metabolic Toxology	mg	IO, IV, PO
Glycoipyrrolate	М	Antagonist	mg/kg	IM, IV
Hydralazine	L	Cardiovascular	mg	IV, IM
pratropium	L	Respiratory	mcg	Nebulizer
soproterenol	М	Cardiovascular	mg	IM, IO, IV



Appendix G - Medications

Medication	Modeled or Log Only	Category	Dosage Options	
			Units	Route
Ketamine	М	Analgesic Hypnotic	mg	IM, IO, IV, Intranasal
Labetalol	М	Cardiovascular	mg	IV
Lidocaine	М	Cardiovascular	mg	IM, IV, IO
Lorazepam	М	Hypnotic	mg	IV
Magnesium Sulfate	L	Emergency Tocolytics	g g/hr	IV, IM
Mannitol	L	Diuretic	g	IV
Mark 1 Kit	L	Antagonist Toxicology	Kit	IM
Meloxicam	L	Analgesic NSIAID	mg	PO
Meperidine	М	Analgesic Narcotic	mg	IV
Methohexital	М	Hypnotic	mg/kg	IV
Methyldopa	L	Cardiovascular	mg	PO
Methylprednisolone	L	Corticosteroid	mg/kg	IV/IO, IM
Metocurine	М	Neuromuscular Blocker	mg	IV
Metoprolol	М	Cardiovascular	mg	IV
Midazolam	М	Hypnotic	mg/kg	IM, IO, IV
Mivacurium	М	Neuromuscular Blocker	mg/kg	IV
Morphine	М	Analgesic Emergency Narcotic	mg	IM, IO, IV
Naloxone	М	Antagonist Toxicology	mg	IM, IO, IV
Neostigmine	М	Antagonist Neuromuscular Blockade	mg	IV
Nitroglycerin	М	Cardiovascular Emergency	mcg/min	IV Paste Patch SL Spray
Nitroprusside	М	Cardiovascular	mcg/kg/min	IV, IO

Appendix G - Medications

CAEMaestro Evolve

Medication	Modeled or	Category	Dosage Options	
	Log Only		Units	Route
Norepinephrine	М	Cardiovascular	mcg/min	IV,IO
Odansetron	L	Antiemetics	mg	IV
Oral Glucose	L	Metabolic	g	PO
Pancuronium	М	Neuromuscular Blocker	mg/kg	IV
Phentolamine	М	Cardiovascular	mg	IV
Phenylephrine	М	Cardiovascular	mcg/min	IV
Prednisone	L	Corticosteroid	mg	IV
Procainamide	М	Cardiovascular	mg/min	IV
Promethazine	L	Histamine Blocker	mg	IV
Propofol	М	Hipnotic	mg	IV
Propranolol	М	Cardiovascular	mg	IV
Quindine	М	Cardiovascular Narcotic	mg/kg	IV
Remifentanil	М	Analgesics	mcg/kg mcg/kg/min	IV
Rocuronium	М	Neuromuscular Blocker	mg/kg mcg/kg/min	IV
Succinylcholine	М	Neuromuscular Blocker	mg/kg	IO, IV
Sufentanil	М	Analegesic Narcotic	mcg/kg	IV
Sugammadex	L	Blockagde Reversal Neuromuscular Blocker	mg/kg	IV
Terbutaline	L	Respiratory	mcg/min	IV, PO, SC
Thiopental	L	Hypnotic	mg mg/kg	IV
Urapidil	L	Cardiovascular	mg mg/min	IV
Vasopressin	М	Cardiovascular	units munits/min	IV, IO
Vecuronium	М	Neuromuscular Blocker	mg/kg/min	IO, IV
Verapamil	М	Cardiovascular Emergency	mg	IO, IV



APPENDIX H - PATIENT PARAMETERS

The Maestro software has a number of parameters that control the physiological features of the simulation. The parameters are grouped by category: Cardiovascular, Respiratory, Neurological, Fluids, Sounds, and Pulse.

Parameter Descriptions

The following is a brief description of each parameter. Each parameter description lists the default settings for the Stan D. Ardman and Norma L. Female patients as well as the ranges, if available, for all patients.

Cardiovascular: Basic Parameters

Arterial Blood Pressure

The **Blood Pressure** parameter is used to set the value of the blood pressure(Manual mode) or override the physiological modeling for blood pressure in Modeled mode. The systolic and diastolic blood pressures can both be set to fixed numeric values, regardless of interventions performed. Resetting the parameter to **Modeled** returns control of the underlying blood pressure to the physiological models. The set change can also be programmed to occur over time using the onset control.

Default: 116/77

Range:

- ° Systolic: 0 mmHg 300 mmHg
- ° Diastolic: 0 mmHg 300 mmHg

Central Venous Pressure (CVP)

The **CVP** parameter is used to set the CVP baseline and atrial contraction amplitude to fixed numeric values. Once set, intravascular volume changes have no effect on the CVP. In addition, once an override is applied, changes in tidal volume have no effect on the CVP waveform with the exception of an apneic patient where the minimum and maximum would be the same value since there is no inspiration or expiration. Depending on the volume status of the patient, the minimum/maximum value can be shifted up or down.

The available CVP controls are as follows:

- Minimum Diastolic: Baseline of the CVP at the end of an inspiration
- Maximum Diastolic: Baseline of the CVP at the end of an exhalation
- Pulse Amplitude: Size of the CVP wave during atrial contraction

For the override to take effect, the **Central Venous Catheter** must be set to the *Intrathoracic Vein*.

Appendix H - Patient Parameters

For example, with the minimum diastolic set to 5 mmHg, maximum diastolic set to 15 mmHg and pulse amplitude set to 2 mmHg, the CVP baseline is 15 mmHg, dipping to 5 mmHg with each inhalation, and the amplitude of the wave is 2 mmHg with each atrial contraction. The CVP baseline remains the same even in the event of intravascular volume changes and the depth of each dip due to inhalation remains at 5 mmHg even in the event of tidal volume changes. However, if the respiratory rate increases or decreases, the frequency of the dips will show a corresponding increase or decrease. The set change can also be programmed to occur over time using the onset control.

Default: Modeled or Manual mode as follows

- ° Minimum Diastolic: -2 mmHg
- ^o Maximum Diastolic: 2 mmHg
- ° Pulse Amplitude: 3 mmHg

Range:

- ° Minimum Diastolic: -10 mmHg 25 mmHg
- Maximum Diastolic: -10 mmHg 25 mmHg
- ^o Pulse Amplitude: 0 mmHg 50 mmHg

Pulmonary Artery Pressure (PAP)

The **PAP** parameter is used to set the pulmonary artery pressure or to override the physiological modeling for pulmonary artery pressure in Modeled mode. The systolic and diastolic pressures can both be set to fixed numeric values, regardless of interventions performed. Resetting the parameter to **Modeled** returns control of the underlying pulmonary artery pressure to the physiological model. The set change can also be programmed to occur over time using the onset control.

Default: Modeled or 18/6 (Manual mode)

Range:

- ° Systolic: 0 mmHg 50 mmHg
- ^o Diastolic: 0 mmHg 50 mmHg

Pulmonary Capillary Wedge Pressure (PCWP)

The **PCWP** parameter is used to display the patient's pulmonary capillary wedge pressure. It is used to simulate the pressure as measured by wedging a pulmonary catheter with an inflated balloon into a small pulmonary arterial branch. The set change can also be programmed to occur over time using the onset control.

Default: Modeled or 6 mmHg

Range: -10 mmHg - 100 mmHg



Heart Rate

The **Heart Rate** parameter is used to set the heart rate to a given (fixed) number of beats per minute. Once the heart rate is set to a numeric value, administered drugs or intravascular volume changes have no effect on the heart rate but continue to influence other components of the physiological models (in Modeled mode). Use this parameter to "fix" or set the heart rate to a specific number. In Manual mode, the set change can also be programmed to occur over time using the onset control.

Default: Modeled or 71 bpm (Manual mode)

Range: beats per minute - beats per minute

Cardiac Rhythm

The **Cardiac Rhythm** parameter is used to change the patient's underlying cardiac rhythm displayed on the Patient Status Display or the CAE Patient Monitor. To change the cardiac rhythm, click the **Cardiac Rhythm** parameter and select the desired rhythm from the available list. If a number appears following the cardiac rhythm on the list, this overrides the heart rate to the rate indicated.

Default: Sinus
Α
Asystole
Atrial Enlargement, Left
Atrial Enlargement, Right
Atrial Fibrillation
Atrial Flutter
Atrial Tachycardia
AV Block, First-Degree
AV Block, Second-Degree, Mobitz I
AV Block, Second-Degree, Mobitz II
AV Block, Third-Degree
AV Block, Third-Degree (Wide Complex)
В
Bundle Branch Block, Incomplete Right
Bundle Branch Block, Left
Bundle Branch Block, Right

Appendix H - Patient Parameters

Н

Hypercalcemia

Hyperkalemia (Mild)

Hyperkalemia (Moderate)

Hyperkalemia (Severe)

Hypertrophy, Biventricular

Hypertrophy, Left Ventricular

Hypertrophy, Right Ventricular

Hypocalcemia

Hypokalemia

Hypothermia

I

Idioventricular

J

Junctional

L

Long QT Syndrome

Low QRS voltage

Μ

Modeled

Myocardial Ischemia, Mild

Myocardial Ischemia, Moderate

Myocardial Ischemia, Severe

Ν

NSTEMI

Ρ

Paroxysmal Junctional Tachycardia

Pericarditis



Premature Atrial Contraction

S

Sinus

ST Elevation with Chest Pain

ST Segment Elevation

STEMI Anterior

STEMI Anterolatera

STEMI Inferior

STEMI Lateral

STEMI LBBB

STEMI Posterior

STEMI Septal

Supraventricular Tachycardia

Т

Torsade de Pointes

Trifascicular Block

V

Ventricular Fibrillation, Coarse

Ventricular Fibrillation, Fine

Ventricular Tachycardia

Ventricular Tachycardia, RV Outflow Tract

W

Wellen's Syndrome

Wide Complex Tachycardia

WPW Syndrome, Left Lateral Pathway

Cardiac Output

The **Cardiac Output** parameter displays the volume of blood pumped by the heart per minute. **Cardiac Output** is a function of heart rate (the number of heart beats per minute) and stroke

Appendix H - Patient Parameters

volume (the volume of blood pumped out of the heart with each beat). **Cardiac Output** does not affect the rest of the physiology. For example, if cardiac output is set to zero, it will be shown on the CAE Patient Monitor as zero, but the patient will still have a blood pressure and pulses. In Manual mode, the set change can also be programmed to occcur over time using the onset control.

Default: Modeled or5.25.9 L/min :

Range: 0 L/min - 30 L/min

Pulseless Electrical Activity

The **Pulseless Electrical Activity** parameter triggers a clinical condition characterized by unresponsiveness and lack of palpable pulse in the presence of organized cardiac electrical activity. It is either ON or OFF.

Default: OFF

PVC Probability

The **PVC Probability** parameter represents the percentage of cardiac cycles containing a premature ventricular contraction (contraction of the ventricles that occurs earlier than usual due to abnormal electrical activity of the ventricles).

The **PVC Probability** is used to set the frequency of PVCs within any cardiac rhythm. The set change can also be programmed to occur over time using the onset control.

Default: Modeled or 0 (Manual mode)

Range: 0% - 90%

Hemoglobin

The **Hemoglobin** parameter is used to set a fixed hemoglobin value. Hemoglobin does not affect the rest of the physiology. The value set will be shown on the CAE Patient Monitor if selected in one of the numeric fields. In Manual mode, the set change can also be programmed to occur over time using the onset control.

Default: Modeled or 14.4 (Manual mode)

Range: 10.0 g/dL - 20.0 g/dL

Respiratory: Basic Parameters

Apnea

The **Apnea** parameter triggers a clinical condition characterized by no spontaneous breathing. Lung sounds and vocal sounds will cease when apnea is enabled. It is either ON or OFF.

Default: OFF





Bronchial Occlusion (Left and Right)

Turning on the **Bronchial Occlusion** parameter completely obstructs the right or left bronchi, simulating a lower airway obstruction (e.g., mucus plug).

Right and left bronchi can be occluded individually.

Improper intubation creates a mainstem occlusion, yielding an inability to ventilate the lungs. However, the right and left bronchi are not occluded individually.

Default: OFF

Respiratory Rate

The **Respiratory Rate** parameter is used to set the respiratory rate to a given number of breaths per minute. In Modeled mode, when the Respiratory Rate is set, arterial oxygen and carbon dioxide values have no effect on the resulting respiratory rate, but continue to influence other components of the physiological models. The patient continues to breathe at the set number of breaths per minute, regardless of the arterial oxygen or carbon dioxide levels.

For example, when the respiratory rate is set to 10 breaths per minute, the respiratory rate remains at 10 breaths per minute, regardless of arterial oxygen or carbon dioxide levels.

Default: Modeled or (Manual mode)

Range: 0 breaths per minute - 40 breaths per minute

SpO₂

The SpO₂ parameter overrides the physiology and sets a fixed numeric value, regardless of the oxygen applied. Resetting to **Modeled** returns control of the underlying SpO₂ to the physiological models. If SpO₂ is set to less than 75%, the eyes will automatically close when in the Auto mode. In Manual mode, the set change can also be programmed to occur over time using the onset control.

Default: 98% **Range:** 0% - 100%

EtCO₂

The $EtCO_2$ parameter is used to set the end-tidal CO_2 to a fixed numeric value, measured in mmHg, regardless of the minute ventilation. The end exhalation point of the capnogram waveform will also reflect the set end-tidal CO_2 value. Setting the $EtCO_2$ has no effect on the arterial carbon dioxide values (PaCO₂), respiratory rate or tidal volume.

For example, when the $EtCO_2$ is set to 50 mmHg, the numeric end-tidal CO_2 will display 50 mmHg and the capnogram waveform rises to an end-tidal of 50 mmHg. However, the respiratory rate and tidal

Appendix H - Patient Parameters

volume will remain the same unless the Respiratory Rate and/or the Tidal Volume parameter(s) is adjusted. When in Manual mode, the set change can also be programmed to occur over time using the onset control.

Default: Modeled

Range: 0 mmHg – 100 mmHg

Tidal Volume

The **Tidal Volume** parameter sets the tidal volume to a given volume per breath. Once Tidal Volume is set to a numeric value, arterial oxygen and carbon dioxide values have no effect on the tidal volume, but continue to influence other components of the physiological models (in Modeled mode).

For example, with the tidal volume set to 600 mL, the tidal volume remains a constant (set) 600 mL even in the event of falling arterial oxygen levels. In such situations, the patient can only respond to arterial oxygen or carbon dioxide levels by altering the respiratory rate. In Manual mode, the set change can also be programmed to occur over time using the onset control.

Default: Modeled or 500mL (Manual mode)

Range: 0 mL - 2500mL

I to E Ratio (1:X)

The **I to E Ratio (1:X)** parameter can be used to change the ratio of inspiratory time:expiratory time (I:E). The I to E Ratio does not affect physiology. The set change also be programmed to occur over time using the onset control.

Default: 2.0

Range: 0.5 - 7.0

Intrapleural Volume (Vol): (Left and Right)

The **Intrapleural Vol** parameters allow intrapleural volume to accumulate, for example, as happens during pneumothorax, hydrothorax or hemothorax.

To simulate a pneumothorax, set the corresponding **Intrapleural Vol** to a value greater than 0 mL. Breath sounds are automatically diminished on the appropriate side due to decreased ventilation of the affected lung.

Default: 0 mL

Range: 0 mL - 2500 mL



Neurological

Eyes

Blink Mode

Condition

Eight (8) eye conditions are available for selection in the neurological patient controls window.

Default: None

Options:

- Normal (both eyes)
- ^o Jaundice (both eyes)
- Bloodshot (both eyes)
- Hemorrhage (right eye)
- ^o Hemorrhage (both eyes)
- Keyhole Pupil (right eye)
- ^o Droopy Eyelids (both eyes)
- ^o Cataracts (both eyes)

Control Eyes Together

Both eyes can be controlled together by enabling the **Control Eyes Together** parameter. When this option is selected, any change made to one eye, will automatically be made to the other eye.

Default: Disabled

Light Reactivity Speed

The **Light Reactivity** parameter determines the speed at which the eyes react to light when the **Reactive pupils** parameter is set to **Yes. Light Reactivity** can be set to **None**, **Brisk**, **or Sluggish**.

Default: Brisk

Condition

Eight (8) eye conditions are available for selection in the neurological patient controls window.

Default: None

Options:

- ° Normal (both eyes)
- ^o Jaundice (both eyes)
- Bloodshot (both eyes)
- ^o Hemorrhage (right eye)
- ^o Hemorrhage (both eyes)
- Keyhole Pupil (right eye)
- ^o Droopy Eyelids (both eyes)
- ° Cataracts (both eyes)

Light Reactivity Speed

The **Light Reactivity** parameter determines the speed at which the eyes react to light when the **Reactive pupils** parameter is set to **Yes. Light Reactivity** can be set to **None**, **Brisk**, **or Sluggish**.

Default: Brisk

Panning

Panning enables random eye movement (left to right).

Default: ON

Pupil Control

The pupil control parameters are used to control the diameter of the pupils in the eyes.

Default: 5 mm

Pupil Diameter

The pupil diameter parameters are used to control the diameter of the pupils in the eyes. Settings allow the user to fix one or both pupils to a specific size.

Default: 5 mm

Range: 2 mm -6mm

Intracranial Pressure (ICP)

The ICP parameter is used to set the ICP displayed as a numeric value on the Patient Status Display and on the CAE Patient Monitor.





Default: 9.3mmHg

Range: 0 mmHg - 65 mmHg

Temperature: Body

The temperature measured at the body surface can be set using this parameter and can be displayed on the Patient Status Display and the CAE Patient Monitor. Body temperature is not linked to the physiologic models, however, changes can be made "on the fly" or scripted using the Scenario Designer. The set change can also be programmed to occur over time using the onset control.

Default: 36.5°C

Range: 32.0°C - 42.0°C

Temperature: Blood

The arterial blood temperature can be set using the **Temperature: Blood** parameter. The arterial blood temperature can then be displayed on the Patient Status Display and the CAE Patient Monitor. The set change can also be programmed to occur over time using the onset control.

Default: 37°C

Range: 32.0°C - 42.0°C

Fluids

Bleeding

Fluid Loss Blood

When used, the **Fluid Loss Blood** parameter allows users to apply a decrease in total blood volume. Blood loss proportionally decreases both the red blood cell volume and the plasma volume according to the current hematocrit.

Range: 0 mL - 6000 mL

Fluid Loss Plasma

When used, the **Fluid Loss Plasma** parameter reflects a decrease in plasma volume. Plasma loss decreases the plasma volume without changing the red blood cell volume. It refers collectively and generically to all fluid losses, including evaporative, transcellular, bowel and third space fluid losses.

Range: 0 mL - 6000 mL

Infusion

Colloid Infusion

When used, the **Colloid Infusion** parameter reflects an addition to the plasma volume without changing the red blood cell volume. Colloids include modified fluid gelatin starch solutions, dextran and human albumin.

Range: 0 mL - 6000 mL

Crystalloid Infusion

When used, the **Crystalloid Infusion** parameter reflects an addition to the plasma volume without changing the red blood cell volume. The term crystalloid is used to describe salt solutions for infusion (e.g., normal saline, dextrose in water, Ringer's Lactate).

Range: 0 mL - 6000 mL

PRBC Infusion

Packed red blood cells are a preparation of 70% red blood cells and 30% liquid plasma, often administered in severe anemia to restore adequate levels of hemoglobin and red cells without overloading the vascular system with excess fluids.

Range: 0 mL - 6000 mL

Whole Blood Infusion

The term whole blood is used to refer to blood that has not been separated into its various components. It represents a preparation of 40% red blood cells and 60% liquid plasma.

```
Range: 0 mL - 6000 mL
```

Sounds

A variety of simulated sounds are available to create a realistic experience. Click the **Sounds** button on the Run screen to access the Sounds controls.

Heart Sounds

Heart sounds can be adjusted by clicking on the Sound icon on the Run screen. When the Sounds panel appears, select **Heart Sounds**. The Heart Sounds menu will appear.

Independent control of the type and volume of heart sounds may be selected in each anatomical region.



Locations:

- ° Aortic
- ° Mitral
- Pulmonic
- ° Tricuspid

To affect the heart sounds simultaneously in all anatomical regions, select **All** and the desired sound.

Default: Normal

Type:

- ° Normal
- ° S3
- ° S4
- ° S3 and S4
- ° Early Systolic Murmur

Note: The volume control slider can be used to adjust the amplitude of the sound.

Lung Sounds

Normal and abnormal breath sounds are selected using this parameter. Breath sounds are synchronized with ventilation of the left and right lungs.

Breath sounds can be adjusted by clicking on the Sound icon on the Run screen. When the Sounds panel appears, select **Lung Sounds**. The Lung Sounds menu will appear.

Independent control of the type and volume of heart sounds may be selected in each anatomical region.

Locations:

- Left Upper
- ° Left Lower
- ° Right Upper
- ° Right Lower

To change breath sounds, select the desired sound from the **Lung Sounds** menu.

Default: Normal

Type:

- Normal
- ° Crackles
- ° Diminished
- ° Gurgling
- Pleural Rub

NOTE: The volume control slider can be used to adjust the amplitude of the sound.

Bowel Sounds

Bowel sounds can be adjusted by clicking on the Sound icon on the Run screen. When the Sounds panel appears, select **Bowel Sounds**. The Bowel Sounds menu will appear.

Normal, **Hypoactive**, **Hyperactive** and absent bowel sounds (**None**) are selected using this parameter.

Type:

- Normal
- Hypoactive
- Hyperactive

Independent control of the type and volume of bowel sounds may be selected in each anatomical region.

Locations:

- Left Upper
- ° Right Upper
- ° Left Lower
- ° Right Lower

To affect the bowel sounds simultaneously in all anatomical regions, select All Bowel Sounds and the desired sound.

Default: Normal

Note: The volume control slider underneath each area may be used to adjust the amplitude of the sound. The volume control slider is only enabled while connected to a simulator.



Korotkoff

Default: OFF

Pulse

All pulses are enabled by default, unless altered by an SCE. To change a pulse setting, click the **Pulse** icon on the Run screen. Click the desired pulse location on the image.

Pulse	Default	Range	
Brachial Right	ON	Not Applicable	
Brachial Deficit	80 mmHg	0 - 300	
Left Femoral	ON	Not Applicable	
Right Femoral	ON	Not Applicable	
Femoral Deficit	70 mmHg	0 - 300	
Carotid	ON	Not Applicable	
Carotid Deficit	60 mmHg	0 - 300	
Radial Right	ON	Not Applicable	
Radial Right Deficit90 mmHg		0 - 300	

The pulse can be turned ON or OFF, and the pulse intensity can be set to:

- ° Absent
- ° Weak
- ° Normal
- ° Bounding

The pulse deficit can be changed from the default to another value. When the systolic pressure falls below the set pulse deficit, the pulse will turn off.



For more information about CAE products, contact your regional sales manager or the CAE distributor in your country, or visit caehealthcare.com. Tel +1 941-377-5562 or 866-233-6384

For customer service, please contact CAE.

Customer Service Headquarters - United States

Monday - Friday from 7:00 a.m. to 6:00 p.m. ET Phone 1-866-462-7920 Email: srgcustomerservice@cae.com

Customer Service - Canada Monday - Friday from 8:00 a.m. to 5:00 p.m. ET Phone 1-877-223-6273 Email: can.service@cae.com

Customer Service - Europe, Middle East, and Africa

Monday - Friday from 8:00 a.m. to 5:00 p.m. CET Phone +49 (0) 6131 4950354 Email: international.service@cae.com

Customer Service - United Kingdom and Ireland

Monday - Friday from 9:00 a.m. to 5:00 p.m. GMT Phone +44 (0)800-917-1851 Email: uk.service@cae.com

Customer Service - Latin America

Monday - Friday from 9:00 a.m. to 5:00 p.m. BRT/BRST Phone +55 11 5069-1510 Email: la.service@cae.com

Customer Service - Asia Pacific

Monday - Friday from 8:00 a.m. to 5:00 p.m. CET Phone +49 (0) 6131 4950354 Email: ap.service@cae.com

©2021 CAE

