
# **Replay**<sup>m</sup> Ultraportable Getting Started Guide

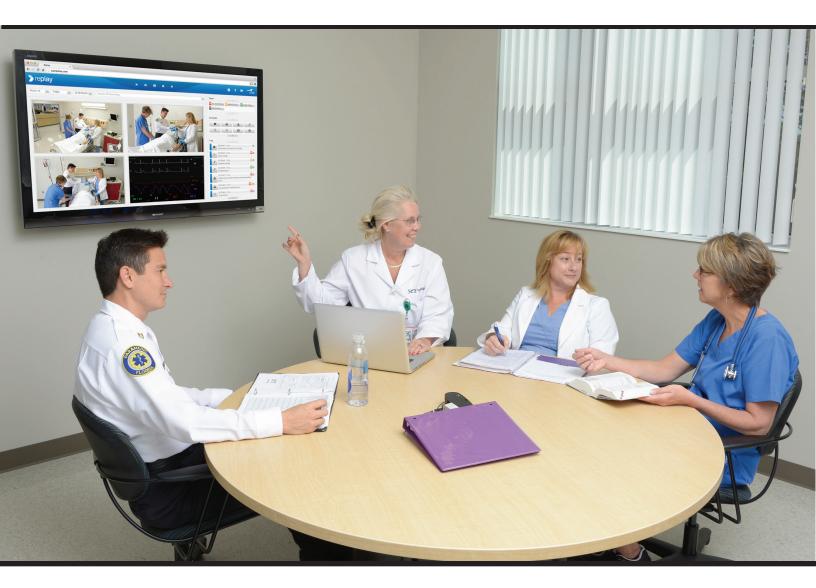



Copyright © CAE Healthcare Academy 2016. All rights reserved. No part of this document may be reproduced without written consent from CAE Healthcare Academy. 905K463252 v1.0



CAE Healthcare 6300 Edgelake Dr Sarasota, FL 34240 USA

Tel +941-377-5562 caehealthcare.com

Dear Valued Customer:

Congratulations and thank you for your recent purchase of the Replay system. This guide will help you get started using Replay - intelligent recording, easy debriefing. In addition, the Replay integrated Help offers a quick reference to Replay Ultraportable the various areas of Replay and how they are used.

The Replay Ultraportable system includes the following components:

Replay software (web-based, installed on laptop) Laptop (Toughbook) Cameras (including tripods) Storage Case Display Capture Unit Framegrabber

If you have any questions or comments, please email Replay Customer Service in North America at **avs-support@cae.com** (phone 1-866-462-7920).

International Customer Service information is available at http://www.caehealthcare.com/eng/support/contacts

**IMPORTANT:** When contacting customer service, please have the serial number and ID available. See *Appendix on page 35* for locating the serial number and ID.



# REPLAY ULTRAPORTABLE GETTING STARTED GUIDE

## Welcome to Replay!

This guide will help you get started using Replay - intelligent recording, easy debriefing. It will explain how to set up, connect, use and debrief with Replay.






# **Table of Contents**

Equipment Components	5
Equipment Setup	6
Initial Network Setup and Login	
HTML5 Video Support	12
Setting Up Rooms	
Setting Up User Access	14
General Replay Access	
Room View	
Adding and Removing Team Members	
Annotations	20
Filter Log View	22
Settings Page Overview	
Video Feed on Room Setup Tab	24
System Overview Tab	25
Chapters	
Sharing Access to Chapters	
Withdrawing Sharing	
Viewing a Shared Chapter	
Help Display	
Search	
Delete Recordings	
Automatic Recording Deletion	
Debrief	
Appendix	
Serial Number and ID Location	
Troubleshooting - Replay Access	






## **Equipment Components**

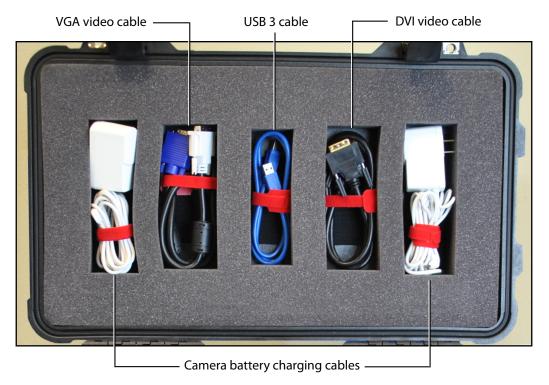

Replay Ultraportable equipment in the upper side of the case

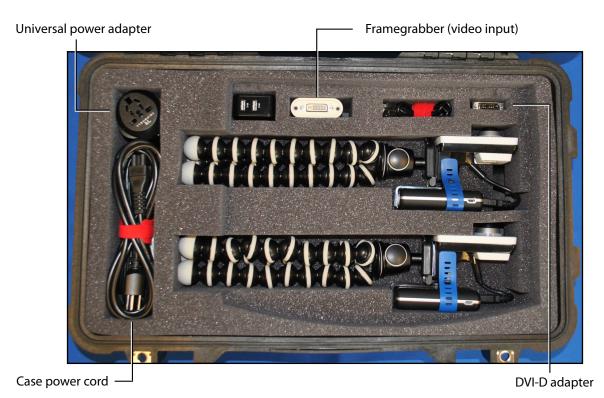

Replay Ultraportable equipment in the lower side of the case




# **Equipment Setup**

#### **IMPORTANT:**

- All power and recharging cords should remain in the storage case. Use the case power cord for all items.
- Be sure to unplug camera from battery prior to storing in the case to avoid possible damage.

To unpack and set up equipment:

- 1. Unwind the case power cord (black) and plug into a power outlet.
- 2. Remove the Toughbook (laptop).

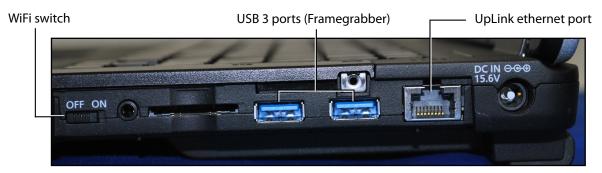

Right side of the laptop

- Plug the laptop power cord into the laptop (right side).
- Plug an UpLink ethernet cable into the UpLink ethernet port on the right side of the laptop.
- Ensure the USB WiFi adapter is plugged into the back of the laptop (the USB WiFi adapter should always remain plugged into laptop).



Back of the laptop

3. Power on the Toughbook.

**IMPORTANT:** Make sure that the WiFi switch is set to ON so that recording with the wireless cameras and connection to the unit on its WiFi hotspot are possible.





#### TO SET UP CAMERAS:

1. Remove the cameras and connect each camera cord to the **2A** port on the battery pack.

**TIP:** The camera should always remain strapped to battery pack.



Camera and battery pack


2. Press the **ON** button on the front of the battery pack to power on the camera. Cameras are connected to WiFi when front and back indicator lights are green and the video feed appears on the Replay screen.



*IMPORTANT:* To power off the camera, remove the cord from the 2A port on the battery pack.



Batteries can be recharged by plugging into the battery recharging cables in the case.

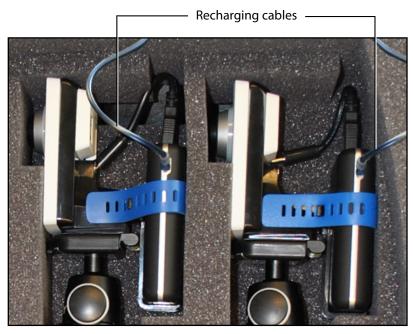

Camera battery recharging cables

- 3. Connect the **Framegrabber** to one of the USB 3 port of the laptop with the USB 3 cable.
- 4. Using the appropriate video cable (VGA, DVI, HDMI), connect the other end of the Framegrabber to the display port of the simulator you want to use.

#### TO SET UP WIRELESS CONNECTION TO A CAE SIMULATOR (after logging in):

1. Disconnect wired simulator from simulator interface if connected.

On the **Room setup** tab of the **Settings** page, all the wireless simulators within range will be listed.



Wireless simulators within range



2. Add the desired simulator to the room by drag and drop or via the **Add device** button.

Room panel -		List of unallocated devices —						
≫ Replay					<b>:</b> ?	€	CAE	
Room setup Sy	/stem overview							
Initial Room				Unallocated	ſ .			
CAE-DCU #DDA9	× ►	Available to:	<ul> <li>Auto-record</li> </ul>					
■	× 🕨	replay@caereplay.com		🛉 🔊 CAESAR99	999 #9999			
■	× 🕨	+ Add email address		🛉 🕈 MMN0316	#0316			
+ Add device		্ৰিক APP0408 #0408			*			
			✓ DONE	Wireless sim	nulator			

Adding by drag and drop

3. The simulator will start connecting.


✔ Initial Room			
CAE-DCU #DDA9	× ►	Available to:	<ul> <li>Auto-record</li> </ul>
AXIS M1034-W #037E	× ►	replay@caereplay.com	
AXIS M1034-W #DCFC	× ►	+ Add email address	
🛉 🔊 APP0408 #0408	× 🗳		
+ Add device			
		Wireless sim	ulator connecti

Room panel with added wireless simulator



# **Initial Network Setup and Login**

1. When the laptop is powered on, the Replay initial setup screen appears.

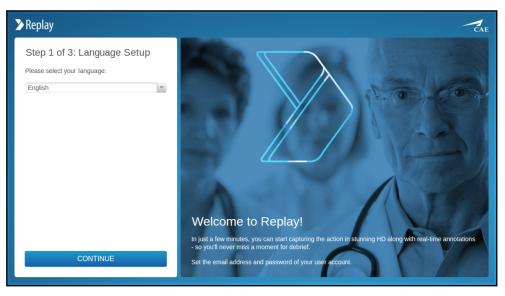


Replay Initial Setup Screen - Step 1

#### STEP 1

1. Select your language from the drop-down and click **CONTINUE**.

The Replay initial setup screen **Step 2** appears.

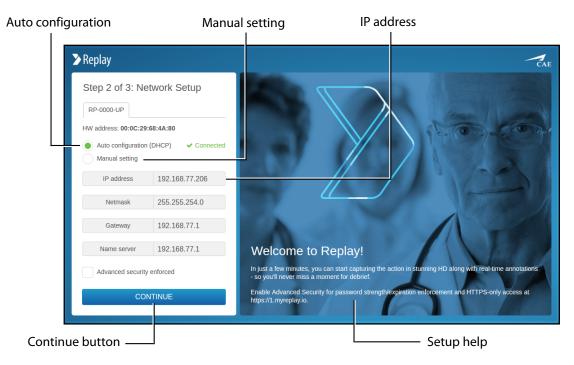

Replay Initial Setup Screen - Step 2



#### STEP 2

1. Select Auto configuration (DHCP).

OR

2. Manual Setting to configure Replay for use with your local area network (LAN).

**IMPORTANT:** Copy or make a note of the IP settings for future reference. If unsure of the settings for your LAN, select Auto configuration.

3. When finished, click **CONTINUE**.

**NOTE:** If anything is edited on the Step 2 page, the **CONTINUE** button toggles to **APPLY** and **CANCEL** options. When done with editing, clicking **APPLY** navigates to Step 3. Clicking **CANCEL** toggles back to the **CONTINUE** button.

Email field ——	Passwor	d fields Time 	e Zone drop-down 
> Replay			CAE
Email Passwo	password		
		Welcome to Replay! n just a few minutes, you can start capturing the is so you'll never miss a moment for debrief. Set the email address and password of your user	action in stunning HD along with real-time annotations account.

The Replay initial setup screen Step 3 appears.

**Replay Initial Setup Screen - Step 3** 

#### STEP 3

- 1. Create your account by entering your email and a password.
- 2. Select the desired Time Zone from the corresponding drop-down.
- 3. Click CONTINUE.

Setup is completed. The room view appears and the Replay system is ready for use.



## **HTML5 Video Support**

Replay has implemented HTML5 video tag support so that in Chrome and Edge browsers, recordings can be viewed without additional browser plug-ins.

**NOTE:** In Replay Ultraportable, Chrome is set as default browser.

**IMPORTANT:** Safari, Internet Explorer and Firefox still requires SPlayer for playing videos.

#### TIPS:

- The three initial setup steps are only performed once.
- Additional users can be added at any time after this initial system setup.
- To change the language selection at any time after initial setup, go to the System overview tab of the Settings page and use the Setup options.

> Replay	
Room setup	System overview
Version:	CAE Replay 1.2.1
Last backup:	✓ a few seconds ago Details
Disk status:	✓ Details
Available recording capacity:	at least 30 minutes
Delete video recordings after:	off 💌 days
Setup options:	Network Setup Language Setup


System Overview tab



# **Setting Up Rooms**

To set up a Replay room:

1. Click the **Settings** icon to open the Settings page.

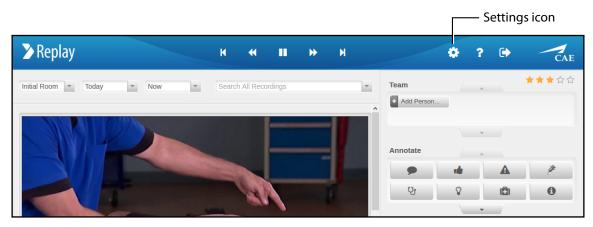

Room view (top)

2. Edit the room name in the title bar.

*TIP:* Click the pencil icon for the title bar to appear.

- 3. Click the **Add device** button to add camera and audio, or drag and drop from the **Unallocated** column into and out of a room.
- 4. Click the Add email address button to assign user access.

Room setup tab	Room title bar	Unallocated column
> Replay		
Room setup	System overview	
Room #1		Unallocated
AXIS M1054 #9EB7	Available to: Available to:	AXIS 212 PTZ #BC06
AXIS P5514 #F245	* replay.ultraportable@cae.com *	AXIS P5512 #3D58
AXIS P8221 #554D	×	•
+ Add device		
Add device	e button	Add email address button

#### Settings page

*TIP:* Any equipment, such as cameras, not assigned to a room will be shown in the **Unallocated** column. Drag and drop equipment listed in the **Unallocated** column into or out of any room.

5. Click **Done** when finished. Replay returns to the room view.

*TIP:* Click the **Settings** icon at any time to manage and edit your room and devices.



# **Setting Up User Access**

Various groups of users can be set up to allow access, control and viewing of Replay. A user who has access to a room can access both the live view and all the previous recordings for that room only.

**NOTE:** For control purposes, discretion is advised when assigning access to Replay.

To give access to users:

- 1. Click the Settings icon to go to the settings page.
- 2. Click the Add email address button within the Room setup panel.

	—— Room setup tab		Settings icon	
🏷 Replay			• = ? 🗗 🗭 -	CAE
Room setup	System overview			
Room #1			Unallocated	
AXIS M1054 #9E	37 × Available to:	Auto-record	AXIS 212 PTZ #BC06	
AXIS P5514 #F24	5 replay.ultrap	ortable@cae.com ×	AXIS P5512 #3D58	
AXIS P8221 #554	D replay@cae	replay.com × Enter email ×	-	
+ Add device				
	User already added	Ado	d email address button clicked	

Settings page

3. Enter the user's email address and press enter (on keyboard).

The user will be displayed and an email will be auto-generated and sent to that user with a link to setup their password and view Replay.

**NOTE:** The user will need to use that email address and password for any subsequent login and access to Replay.

*TIP:* After a user has been added to a room, drag and drop that user to add them to any other room.



# **General Replay Access**

Users who have been set up in your Replay system have access to Replay.

To access Replay:


1. Switch on your Ultraportable.



**Replay login screen** 

2. Enter your email and password and click **Login**.

The Replay screen will appear and is ready for use.

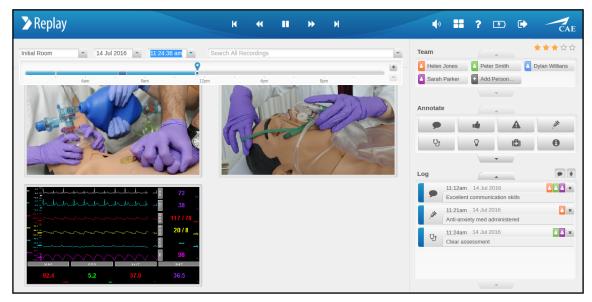

**Replay Room view** 



## **Room View**

Time-line Time drop-down Annotations panel Team panel > Replay • ₩ И × ۰. ? 5 C CAL Room #1 Team 14 Jul 2016 -🧧 Helen Jones 🛛 🔄 Peter Smith 🛛 🚺 Dylan Willians + Sarah Parker + Add Person... Annotate Ų 6 **•** + Log A 🕹 🕹 🗶 11:12am ommunicat × 11:21am 14 Jul 2016 ty med admir 🔺 📥 🗙 11:24am 14 Jul 2016 Cameras in room -Log list

The **Room view** screen displays the recordings and all features used to manage, facilitate and debrief.

Room view

A time-line bar indicates the current live time and the location of the recording at any given moment.

- The time-line bar is accessed by clicking in the time window or drop-down.
- The current location indicator can be dragged to any point in the recording.
- Click the time-line + or buttons to zoom the time-line in or out.

Any input event (annotation, team member added or removed, patient started on simulator) marked as a chapter on the time-line bar will be dark blue.

Chapter r	marker ———	Г	Time window				Live time indicator			
Room #1	<ul> <li>Today</li> </ul>	• 12:16:37	pm 💌	Search	All Rec	ordings			~	
							<b>9</b>		+	
	4am	8am	12pm	, , , , , , , , , , , , , , , , , , , ,	4pm		8pm	1		
Recording	current location	indicator ——					Time	e-line zoo	 om buttons	

Room view - time-line bar

**IMPORTANT:** To ensure recordings are saved, mark them as chapters by making any input event. See more information under *Chapters on page 27*.



If a CAE patient monitor or **Display Capture Unit** with display device is configured with Replay, that display will also be shown in the Room view.

	и 4	• •	₩	М		<b>(</b> )	? 🖸	۲	CAI
11:24:36 am			8pm		_	Helen Jones		n 🔤	★★★☆☆
	G	( )	Ser /	P		Annotate		▲ (Ĉ)	<i>*</i> 0
						11:12 Excel	ent communication am 14 Jul 2016		) ها × ۱۰۰ × ۲۰۰
20/8 , 20/8 , 98 , 36.5						[]0			4 4 ×
	an 22m	124.36 arr     2m       Image: Search All Records       Image: Search All Records	124.36 arr   200 201 201 201 201	1.24.36 cm       Search All Recordings         Image: Search All Recordings       Image: Search All Recordings         Image:	124.36 arr       Search All Recordings         128.36 arr       1000         128.36 arr       1000         129.00       1000         1000       1000	12836 arr       Search Al Recordings         128       128         128       128         128       128         128       128         128       128         128       128         128       128         128       128         128       128         129       128           129       128	1221 36 am   Search All Recordings   Image: Search All Recordings     Image: Search All Recordings <td>1221 306 arr       Search All Recordings         120m       6m         120m       14         1212m       14         14       14</td> <td>1221 30 arr       Search All Recordings         Image: Search All Recordings       Image: Search All All All All All All All All All Al</td>	1221 306 arr       Search All Recordings         120m       6m         120m       14         1212m       14         14       14	1221 30 arr       Search All Recordings         Image: Search All Recordings       Image: Search All All All All All All All All All Al

Monitor view selection

*Room view - with monitor display* 

The Star Rating system allows users to set a star rating for every Replay recording chapter. Click to highlight a star from one to five.

Recordings can be searched by star ratings, for example, by entering \*\*\* in the search field.





## Adding and Removing Team Members

To indicate who is visible or otherwise relevant at any point in a recording, add or remove team members to a recording as they enter or leave the room.

#### To add a team member to a recording:

1. Click the **Add Person** button in the **Team** panel.

> Replay		н н	∎ >>	N		<b>(</b> ) <b>=</b>	? 🖸 (	CAE
Initial Room • 14 Jul 201	16 💌 11:24:36 am 💌 Se	earch All Recordings			<b>.</b>	Team	Peter Smith	★★☆☆☆
4am	8am 12pm	4pm	8	pm i	+	-	+ Add Person	Oyian Willians
Chapter indicator —				Ado	d pers	son button	<u>,                                    </u>	



2. Enter the name and press **Enter** on keyboard (or click anywhere outside of the box after entering name).

The team member will be added and identified as a chapter (dark blue indicator) at that point in the recording.

Added team members with color identifier	Team		★★☆☆☆	
	Dylan Williams	🔺 Mia James	Gabe Walker	
	Sarah Parker	🔺 Lucas Smith	Enter name	
	-	-		——— Entering new member

#### Team panel

**NOTE:** Individual color identifiers are automatically assigned per person when added.

*TIP:* To find a team member, enter a name (or partial name) in the search box and press enter on the keyboard. Then click a selection from the results.

Click the time window or drop-down to show or hide the time-line bar.

Time drop-down ———			Search bar				
> Replay	K	<b>↔ II</b>	₩ H		<b>(</b> )	? 🖸 (	CAE
Initial Room • 14 Jul 2016 • 11:24:36 am	Search All Re	ecordings			Team	<pre>/</pre>	★★★☆☆
	?			+	Helen Jones	Peter Smith	Dylan Willians
4am 8am	12pm	4pm	8pm	-	Sarah Parker	+ Add Person	
				No.		-	





#### To remove a team member from a recording:

1. Go with the cursor over the team member to be deleted. An **X** appears next to the member's name.



Team panel

2. Click the **X** to remove the member.

A confirmation of removal message appears, including a link offering an undo option.

	Undo link ———					
🏷 Replay	K	•		₩	М	
	🟮 Tear	m member "	Sarah Park	er" has be	en removed.	Undo

**Confirmation message** 

**IMPORTANT:** The undo link is accessible only for approximately ten seconds.



## Annotations

There are various ways to add and adjust annotations within a recording.

#### To add annotations:


1. Click a button in the **Annotate** panel. The annotation appears in the **Log** and is ready for text entry.

**NOTE:** This adds the annotation with no associated team member.

2. Drag and drop a **Team** member button onto an **Annotate** button.

The annotation appears in the Log with the associated team member's color icon and is ready for text entry.

3. To remove a team member from an annotation, drag and drop that team member's color icon off the annotation (a red x will indicate removal).

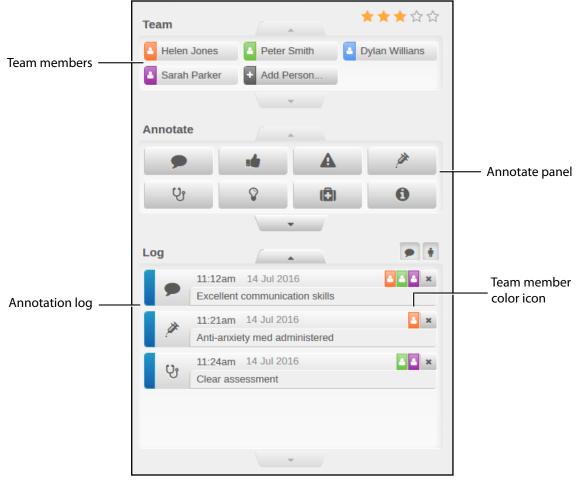

Room view (panels)

4. When finished typing an annotation, click **Enter** on keyboard or click anywhere outside of the annotation box.




#### Other annotation features:

• Drag and drop a button in the **Annotate** panel onto an existing annotation in the **Log** panel to change the category of that annotation.

**NOTE:** If the dragged button crosses over the scroll arrow of the **Log** panel, all annotations in the system appear in the log.

- Drag and drop a button in the **Annotate** panel into the **Search** bar to get search results containing the category criteria of that annotation.
- Click an annotation in the **Log** to jump to that point on the recording time-line.
- Buttons in the **Annotate** panel can be re-arranged by dragging and dropping them within the panel. Any arrangement made will be automatically saved.

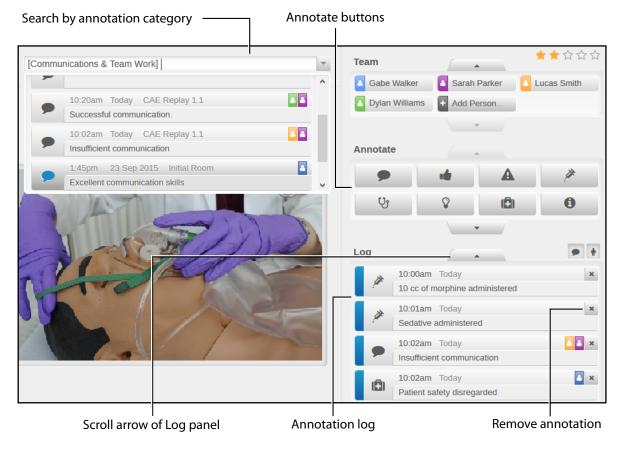

#### Room view with active search

#### To remove an annotation:


• Click the **X** on the right side of the annotation in the Log.

**IMPORTANT:** When an annotation is deleted, it is removed from the recording time-line and can no longer be targeted in search.




## **Filter Log View**

The added annotations and simulator events listed in the **Log** panel can be displayed selectively. By default, both buttons are activated, therefore the items of both categories are listed in the **Log**.

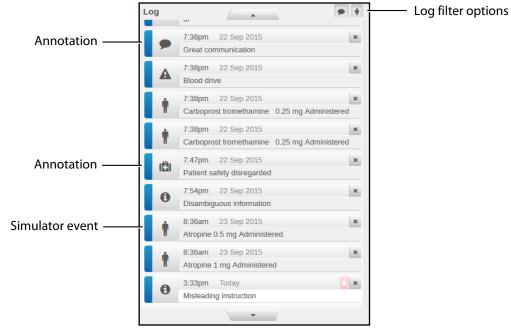

Log view unfiltered

- To view only the annotations, click thus deactivate the **Simulator events** icon (human figure).
- To have only the simulator events listed, click the **Annotations** icon (all other icons depending on the annotation categories).

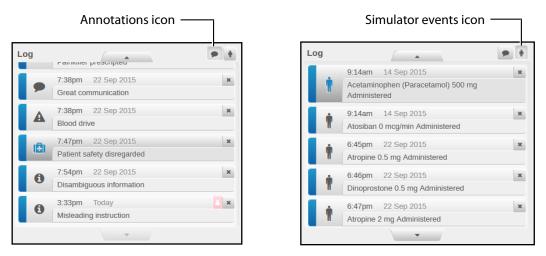

Only annotations or only simulator events displayed



## **Settings Page Overview**

The **Settings** page consists of two tabs:

**Room setup tab:** displays the settings for rooms, including cameras, equipment and users; as well as a panel for **Unallocated** equipment.

**System overview tab:** information on the Replay system.


Room setup tab —	System	overview tab	Room	view —	Battery icon
> Replay				<b>●</b> » <b>■</b> ?	E CAE
Room setup	System overview				
Room #1				Unallocated	
AXIS M1054 #9EB7	×	Available to:	Auto-record	AXIS 212 PTZ #	.BC06
AXIS P5514 #F245	× ►	replay.ultraportable@cae.com	n ×	AXIS P5512 #3	D58
AXIS P8221 #554D	× 📢	replay@caereplay.com ×	+ Add email address		•
Add device					
Room panel					Unallocated equipment

Room Setup tab of Settings page

TIP: Click the Room view icon to return to the Room view screen.

The **Battery** icon indicates the battery level or the charging status of the Toughbook.

*TIP*: Hovering over the icon prompts a percentage information to appear.

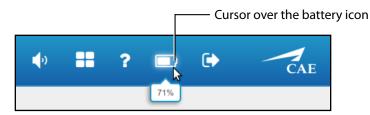

Battery level in percentage

*IMPORTANT:* The Toughbook shuts down before reaching 0% to be able to save all data.



## Video Feed on Room Setup Tab

The video feed view is accessible via the **Room setup** tab of the **Settings** page.

1. Click the **Play** icon next to the camera you wish to view.

AXIS M1034-W #DCFC	× ►	Available to:	Auto-record
■ 9 AXIS M1034-W #D6D9	× ►	replay@caereplay.com ×	rp3@caereplay.com
+ Add device		+ Add email address	
Play button			

Room panel on the Room setup tab

The camera view appears in a pop-up window.



Camera pop-up window on the Settings page

- 2. Use the volume slider on the right below the camera screen to adjust the volume.
- 3. Click anywhere on the screen outside the pop-up window to close it.

## System Overview Tab

As a new feature, the **System overview** tab is available through the **Settings** page.

System overview tab –	Version	number 	Last backu	p details			
> Replay			<b>(</b> )	<b>**</b> ?	5	•	CAE
Room setup Sy:	stem overview						
Version: CAE	Replay 1.2.1						
Last backup: 🗸 a	minute ago Details						
Disk status:	etails						
Available recording capacity: at le	ast 2 hours						
	rork Setup uage Setup						
Setup options ——	Available	e recording capacity	,	Disk st	tatus c	letails	

System overview tab on the Settings page

The tab includes information about the **Last backup**, that is, the last date and time of transferring data to the cloud.

- If the backup has been successful, it is indicated by a green check mark.
- If anything has failed, it is indicated by a red X before the **Details** link, and listed in the details accessed by clicking the link.

Last backup:	✓ a minute ago Details				
	RP-0230-MA	15 Jul 2016 (a minute ago)	~		

Last backup details

**NOTE:** Make sure that the Internet connection is uninterrupted.

The **Disk status** provides information about the available storage space. It is displayed with a green check mark before the **Details** link.

Disk status:	✔ Details		
	RP-0230-MA	6.94 GB	~


Disk status details

**NOTE:** If the system runs out of disk space, a message appears in Room View and a warning icon is displayed on the Settings page.

Room setup	System overview		You are not able to record any new video until you free up space on your disk. You can delete recordings via the recording tootlip(s) on
🖉 Initial Room			your time-line at the top of the room view.
AXIS 212 PTZ #7DD0	×	Available to:	Auto-record 0 AXIS M1054 a Warning icon

Icon on Settings page



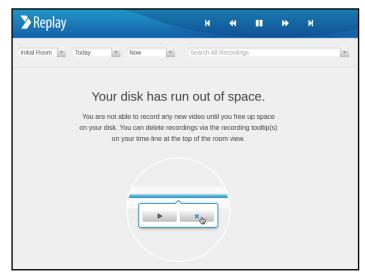

Message in Room View

The **Available recording capacity** is to be considered informative only; it provides an approximate about the minimum capacity.

Available recording capacity:	at least 21 hours					
Available recording capacity						

The **Setup options**, including the **Network Setup** and **Language Setup** links, allow for modifications in the relevant setting after the initial setup.

Setup options:	Network Setup
	Language Setup

Setup options



# Chapters

Any input event (annotation, team member added or removed, patient started on simulator) will be marked as a chapter on the time-line bar, distinguished by dark blue.

**IMPORTANT:** To ensure recordings are saved, mark them as chapters by making any input event.

Initial Room	T	Today	*	6:31:02 am	T	Search All Recordings							~
_				0									+
1	6am					7am	1	7:30am	1	8am	1	8:30am	-
		Chapte	er —								Re	ecording	

Chapter indicated on the time-line

Input events that can mark the beginning of a new chapter include:


- Adding a team member(s).
- Starting a patient on an associated simulator.
- Adding an annotation to a portion of the recording that has no other previous input events, see **NOTE** below.

This will create chapter starting just before the annotation was added and extend to just after the annotation.

- Adding a star rating to a portion of the recording that has no other previous input events, see **NOTE** below.
- Removing all team members and then adding new team members. For example, to keep the simulation running, but replacing the team working on the patient.

**NOTE:** Adding an annotation or creating a star rating will only create a new chapter if that portion of the recording has no other previous input events. For example, if a team member is added and then an annotation is made within a few minutes, that would be considered the same chapter.

Chapters can be downloaded in mp4 format. This option is available via the Chapter tooltip which can be generated by hovering over the desired portion of the time-line.

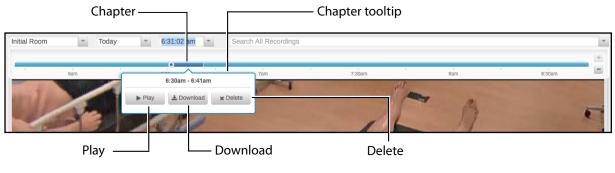

Chapter settings





## **Sharing Access to Chapters**

- 1. Specific chapters can be shared with addressees via email.
- 2. Move your cursor over the section of the time-line marking the chapter desired to be shared.

The chapter tooltip window appears.

Initial Room 💌 Today	▼ 11:41:07 am ▼	[Communications & Team Work]	•
	11.10	<b>Q</b>	+
4am		4pm 8pm	E
	10:42am - 10:53am ▶ Play ▲ Download	x Delete	
	Can view:		
	lucas.gordon@cae.com	×	
	+ Add email address		-
dd email address —		Already add	ed email

Chapter tooltip appearing below the time-line

- 3. Click the +Add email address field.
- 4. Enter the email address of the person you wish to share the chapter with.



Chapter tooltip when typing in email address

5. Press Enter.

A message appears informing about successful sharing of the chapter; and the addressee receives an email with a link of the recording.

**IMPORTANT:** The user the chapter has been shared with will have limited access; only to the chapter itself and search options within the chapter.





## Withdrawing Sharing

- 1. Search for the chapter you wish to withdraw access to.
- 2. In the chapter tooltip window, click the **X** next to the email address you wish to cease sharing with.


Can view:			
lucas.gordon@	)cae.com	×	— Withdraw access
+ Add email a	ddress		

Withdraw sharing in the chapter tooltip

Upon removing an email address, the link will no longer be available to the recipient.

## **Viewing a Shared Chapter**

 From the email received, click the link provided. The shared chapter recording appears.

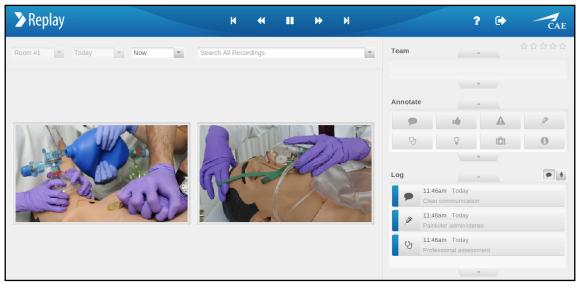

#### Shared chapter

- 2. Click anywhere on the accessible line marking the chapter to navigate back and forth within the recording.
- 3. Use the filters at the annotation **Log** panel to display annotations and simulator events selectively.




# Help Display

		Help icon				
	<b>(</b> )	? 🖸	CAE			
	Team		★★★☆☆			
+	A Helen Jones	Peter Smith	Dylan Willians			
Help icon						

Click the **Help** icon to display an overlay that highlights the various areas and how they are used.

Help can be displayed on both the **Room view** screen and the **Settings** page.

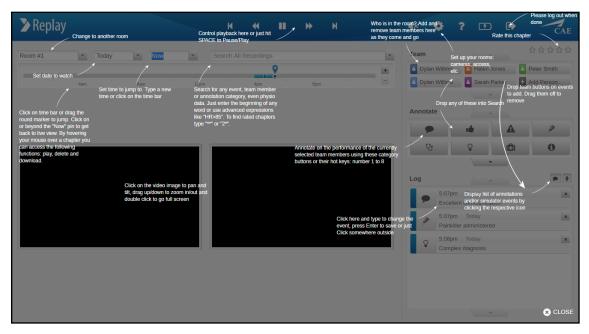

Help display on Room view

TIP: Click anywhere on the screen to exit the overlay.



# Search


The Search bar on the **Room view** allows you to find and navigate to chapters or events. Use search to find a team member, annotation or simulator events.

Options to search:

a. Enter the search term and press Enter on keyboard.

A list of results will be displayed. Click any result to go to that event or chapter.

**TIP:** Search for a partial word or name to increase search results. Use advanced terms, such as HR>85, to search for an event where the heart rate was greater than 85. Enter a star rating such as \*\*\* to search for a chapter that was given a three-star rating.

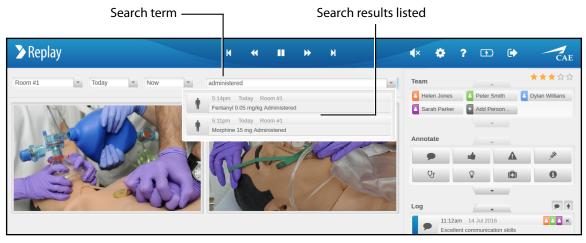

Room view - search by term

b. Drag and drop a button of the **Annotate** panel into the Search bar to get a list of results relating to that annotation category.

Communications & Team Work]	-	Team			* * ☆ ☆ ☆
-	^	Gabe Walke	er 🚺 Sarah	Parker 🔒	Lucas Smith
10:20am Today CAE Replay 1.1     Successful communication	44	Dylan Williams + Add Person			
10:02am Today CAE Replay 1.1     Insufficient communication		Annotate		•	
1:45pm 23 Sep 2015 Initial Room     Excellent communication skills		•		A	Ø
		Ċ,	Ŷ		0

Search results for annotation category



c. Drag and drop a **Team** member into the Search bar to get a list of results relating to that team member.

**TIP:** Clear search criteria before entering or dropping a new search.



Room view - search by team member

*TIP:* Click the **Help** icon to get an overlay of help items, including search tips.




# **Delete Recordings**

Recordings (light blue) or chapters (dark blue) can be deleted from the time-line.

NOTE: When a recording or chapter is deleted, it can no longer be used to debrief.


To delete a chapter or recording:

- 1. Find the recording or chapter in the time-line bar. Options to find the recording or chapter include:
  - Click in the time window or drop-down to display the time-line bar.
  - Click in the calendar window or drop-down to select the recording date.
  - Click in the search window to search for the recording or chapter and click on a result.

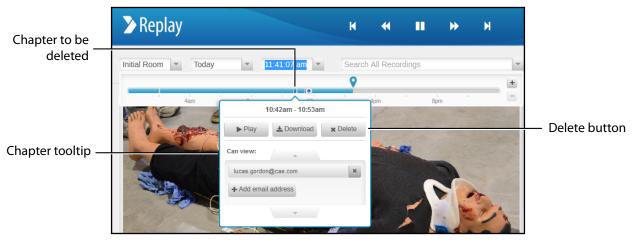

Room view - time-line bar

2. Hover your mouse over the recording or chapter on the time-line bar to be deleted and the recording or chapter tooltip will appear.

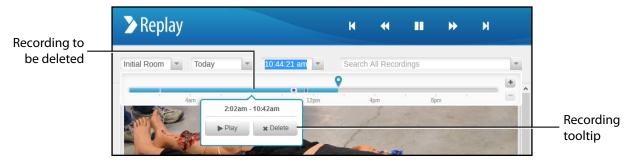

Room view - time-line bar

- 3. Click Delete.
- 4. A warning pop-up will ask to verify deletion.



# **Automatic Recording Deletion**

In order to maintain a sufficient amount of free space, Replay automatically deletes recordings when no input event has been detected.


- Recordings that do not contain chapters (input event) **and** are older than 1 day **and** no motion has been detected are automatically deleted.
- If motion is detected **and** the recordings do not contain chapters (input event) **and** are older than 7 days, they are automatically deleted.

To ensure recordings are saved, mark them as chapters by making any input event (annotation, team member added or removed, patient started on simulator, etc).

# Debrief

Chapters are used to debrief. Navigate to the desired chapter and it will play automatically.

- Use the **playback controls** to pause, play, rewind, fast forward or skip chapters forward or back.
- Use the **search** bar or drag and drop team members or annotations into the search field to show events and results.
- Use the **date and time selections** to navigate to a desired time.

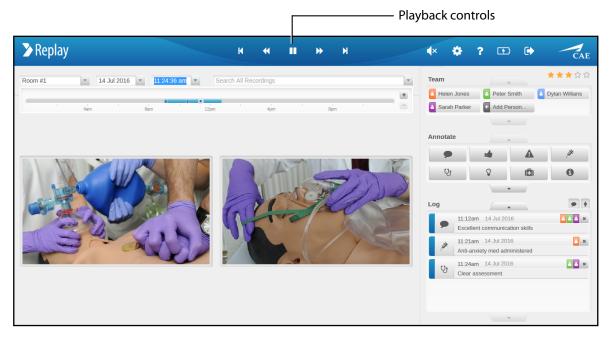

#### **Room View**



# Appendix

## **Serial Number and ID Location**

The serial number and ID are located on the bottom of the laptop.



Serial number and ID

Bottom of the laptop



## **Troubleshooting - Replay Access**

If you experience difficulties while accessing Replay remotely (through *serialnumber.myreplay.io*), you may try accessing it by using the network IP address from setup.

In case that does not work, check if Replay is accessible on the Toughbook.



**Replay Network Setup screen** 



### caehealthcare.com

For more information about CAE Healthcare products, contact your regional sales manager or the CAE Healthcare distributor in your country, or visit caehealthcare.com. Tel +1 941-377-5562 or 866-233-6384

For customer service, please contact Replay Customer Service by email. **avs-support@cae.com** 

#### **North American Time Zones**

Eastern Standard hours:Monday - Friday, 8:00 AM-8:00 PMCentral Standard hours:Monday - Friday, 7:00 AM-4:00 PMMountain Standard hours:Monday - Friday, 6:00 AM-3:00 PMPacific Standard hours:Monday - Friday, 5:00 AM-2:00 PMPhone 1-866-462-7920Monday - Friday, 5:00 AM-2:00 PM

International

Monday - Friday from 7:00 a.m. to 6:00 p.m. ET Phone + 1-514-341-6780

©2016 CAE Healthcare 905K463252 v1.0